# ***Jumbo Screen Premium Digital Blood Pressure Wrist Monitor***

Model 01-516

**INSTRUCTION MANUAL • ENGLISH & ESPAÑOL**

**STOP!**  
**PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING**  
**COMPONENTS BEFORE USING YOUR DIGITAL BLOOD**  
**PRESSURE MONITOR**


Digital Monitor  
with Attached Cuff

2 AAA Batteries

Canvas Storage  
Case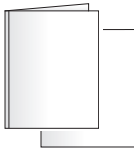

Instruction Manual  
Quick Start Guide  
Blood Pressure Reading Tracking Log  
FAQ Information Sheet

**IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR  
INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF  
PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.**

---

# INDEX

---

Introduction .....	4
Getting Started .....	5
Features of Digital Blood Pressure Monitor .....	6
Battery Installation and Replacement .....	7
Setting Date and Time .....	8-9
Applying Your Blood Pressure Cuff.....	10-11
Taking Your Blood Pressure Reading .....	12-14
Interpreting your Blood Pressure Reading .....	15-17
Memory Function.....	18-19
Care and Maintenance .....	20
Display Symbols.....	21
Troubleshooting .....	22
Specifications.....	23
FCC Statement .....	24-25
Warranty .....	26-27
Instrucciones en Español.....	28-55

**Toll-Free Customer Care Help Line: 1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

Manufactured for:  
Veridian Healthcare  
1465 S. Lakeside Drive  
Waukegan, IL 60085  
[www.veridianhealthcare.com](http://www.veridianhealthcare.com)

Made in China

#93-1059 04/10  
©2010 Veridian Healthcare, LLC

---

# INTRODUCTION

---

**Veridian Healthcare recommends that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.**

Automatic Digital Blood Pressure Monitors use the oscillometric method to electronically measure your blood pressure. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor ideal for home use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained healthcare professional using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

**Thank you for purchasing a Veridian Healthcare Automatic Digital Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.**

---

# GETTING STARTED

---


## Before Taking a Measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

---

## FEATURES OF YOUR MONITOR

---

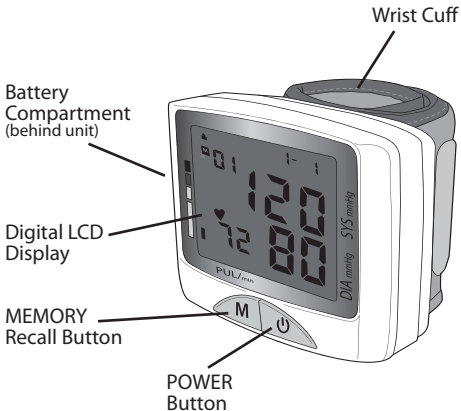**DETAILS OF THE DISPLAY AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.**

# BATTERY INSTALLATION


---

**This unit comes complete with 2 AAA batteries.**

It is necessary to replace the batteries when the low battery symbol appears on the display, or when the display does not turn on after the POWER button is pressed. 

## Replacing Battery

1. Open the battery compartment cover.
2. Insert or replace 2 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries. Do not mix new and old batteries; do not use rechargeable batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

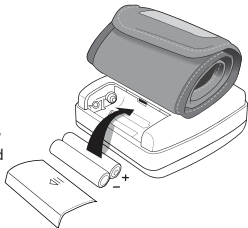It is recommended to remove the batteries if the unit will not be used for an extended period of time.

# SETTING DATE/TIME

**IT IS NECESSARY TO SET THE DATE AND TIME FOR THE UNIT EVERY TIME BATTERIES ARE INITIALLY INSTALLED OR REPLACED.**

**DATE AND TIME WILL DEFAULT TO FACTORY SETTINGS, 12:00 A.M. / 1(M)-1(D), IF YOU DO NOT CHOOSE TO SET THE CORRECT INFORMATION.**

The date/time feature is on the top right of the display screen, Fig. 1. The date and time will alternate every four seconds regardless of the operating mode.

1. When the unit is off, press and release both the  (POWER) and M button at the same time. The month will begin to flash, Fig. 2.

2. Press the M button to change until the desired number appears. To scroll through the numbers quicker, hold down the M button as the numbers advance.

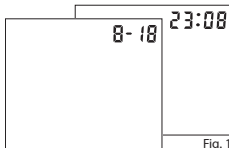
Fig. 1

Fig. 2

---

## SETTING DATE/TIME

---

3. Press the  button to confirm the month entry.
4. Next, the date will blink. Repeat steps 2 and 3 to set the date, then hours, then minutes. This unit does not track the year.
5. After setting the minutes, the unit will automatically exit out of the date/time setting mode.

Time is displayed in 24hr Military Time. This chart explains the time conversion.

### **MILITARY TIME = REGULAR TIME**

1:00 = 1am	7:00 = 7am	13:00 = 1pm	19:00 = 7pm
2:00 = 2am	8:00 = 8am	14:00 = 2pm	20:00 = 8pm
3:00 = 3am	9:00 = 9am	15:00 = 3pm	21:00 = 9pm
4:00 = 4am	10:00 = 10am	16:00 = 4pm	22:00 = 10pm
5:00 = 5am	11:00 = 11am	17:00 = 5pm	23:00 = 11pm
6:00 = 6am	12:00 = 12pm	18:00 = 6pm	24:00 = 12am

---

## APPLYING THE CUFF

---

**PROPER CUFF FIT AND PLACEMENT IS CRITICAL IN OBTAINING ACCURATE BLOOD PRESSURE MEASUREMENTS. PLEASE READ THESE INSTRUCTIONS CAREFULLY AND CONSIDER CONSULTING WITH YOUR PHYSICIAN FOR FURTHER DIRECTION.**

Before applying your blood pressure cuff, be sure your wrist is within the appropriate cuff range:

**Wrist Circumference Range Suitable for 5-1/2" – 7-5/8" (14 – 19.5 cm)**

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

If for any reason you are unable to or should not use your left wrist, please modify the instructions for cuff application to your right wrist. Your physician can help you identify which wrist is best for you to take measurements from.

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Be seated at a table or desk with your feet flat on the floor.

## APPLYING THE CUFF

3. Hold your left arm in front of you with your palm facing upward.
4. Apply the preformed cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you, Fig. 1.
5. Adjust the cuff 1/4" - 1/2" from the edge of the head of the ulna bone, Fig. 2.
6. The cuff should fit comfortably, yet snugly around your wrist.

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

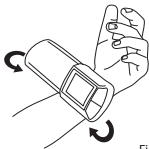
Fig. 1

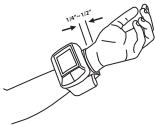
Fig. 2

# TAKING YOUR READING

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

1. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand.
2. Press the  (POWER) button to turn the unit on; the unit will run a self-test, Fig 2.
3. The display will show the current memory bank in use, 1 or 2,  at the top left of the display. Press the M button to select a different memory bank if desired.

Fig. 1

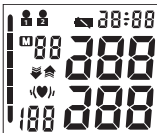
Fig. 2

## TAKING YOUR READING

- The unit will briefly display the last reading value for that memory bank. If there is no previous measurement, the unit will display '0' for all of the reading values, Fig 3.
- The cuff will automatically inflate to approximately 190mmHg or 40 mmHg higher than the last systolic pressure reading stored in memory.
- When the inflation has reached optimum level, the display will begin to show the decreasing pressure while you feel the pressure of the cuff decrease.
- Your blood pressure measurement and pulse will display simultaneously on the screen, Fig 4. Both the systolic and diastolic readings will display at the same time.

Fig. 3

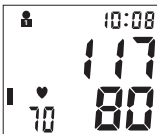
Fig. 4

## TAKING YOUR READING

- The Hypertension Indicator will indicate your reading range in the left side of the display, Fig 5. See page 17 for a chart interpreting the Hypertension Indicator results.
- If an irregular heartbeat was detected during the reading, the Irregular Heartbeat Detector indicator will appear on the display, Fig 6. See page 15 for more information on the Irregular Heartbeat Detector feature.
- The reading will automatically be stored in the memory bank selected prior to the measurement, up to 60 readings.
- Select  (POWER) to turn the unit off and conserve energy and battery life. The unit will automatically shut-off and revert to time/date mode after approximately 1 minute.

Fig. 5

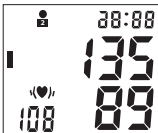
Fig. 6

**YOU CAN STOP THE INFLATION OR DEFLATION PROCESS AT ANY TIME BY PRESSING THE START/POWER BUTTON.**


---

# INTERPRETING YOUR RESULTS - IHB

---

## Irregular Heartbeat Detector (♥)

Your digital blood pressure monitor features an Irregular Heartbeat Detector. This feature allows users to accurately monitor blood pressure even if an irregular heartbeat should occur. When an irregular heartbeat is detected, the IHB icon will appear on the display.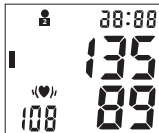

Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently.

---


# INTERPRETING YOUR RESULTS - HYPERTENSION INDICATOR

---

## Hypertension Indicator

This unit features our unique Hypertension Indicator. The World Health Organization has established globally accepted standards for the assessment of high or low blood pressure readings.

The Hypertension Indicator is located on the left side of the display

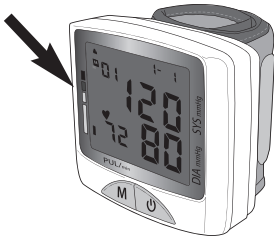# INTERPRETING YOUR RESULTS - HYPERTENSION INDICATOR

This chart should be considered only as a guideline, always consult with your physician or health care professional to interpret your individual results.

## Stage 3 Hypertension

Systolic pressure:  $\geq 180$  mmHg  
*or*  
Diastolic pressure:  $\geq 110$  mmHg

## High Normal

Systolic pressure:  $\geq 130$ -139 mmHg  
*or*  
Diastolic pressure:  $\geq 85$ -89 mmHg

## Stage 2 Hypertension

Systolic pressure:  $\geq 160$ -179 mmHg  
*or*  
Diastolic pressure:  $\geq 100$ -109 mmHg

## Normal

Systolic pressure:  $\geq 120$ -129 mmHg  
*or*  
Diastolic pressure:  $\geq 80$ -84 mmHg

## Stage 1 Hypertension

Systolic pressure:  $\geq 140$ -159 mmHg  
*or*  
Diastolic pressure:  $\geq 90$ -99 mmHg

## Optimal

Systolic pressure:  $< 120$  mmHg  
*or*  
Diastolic pressure:  $< 80$  mmHg

# MEMORY FUNCTIONS

## Recalling Measurements in Memory

You can recall up to 120 total measurements in memory, 60 per memory bank. Using the included blood pressure log, these readings can be shared with your physician or trained healthcare professional.

1. Press and release the M (MEMORY) button. The display will show the current memory bank in use, 1 or 2,  at the top left of the display. Press the M button to select a different memory bank if desired.

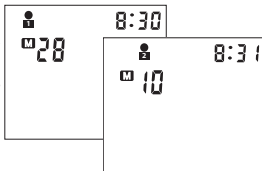
Fig. 1

2. The unit will briefly show how many measurements are stored within the bank, Fig. 1.
3. The display will next show the average of the last 3 readings. A flashing A will appear on the display, the hypertension indicator bar will also flash Fig 2.

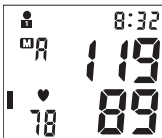
Fig. 2

## MEMORY FUNCTIONS

4. Press the M (MEMORY) button to begin scrolling through the saved measurements. The measurement number and hypertension bar indicator will flash, Fig 3. After scrolling through all the readings, the unit will automatically turn off.
5. When the number of readings exceeds 60, the oldest data will be replaced with the new record.

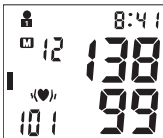
Fig. 3

### Clearing Measurements from Memory

1. Press and release the M (MEMORY) button. The unit will display how many measurements are stored within the bank.
2. Press and release the M (MEMORY) button until the first measurement is shown.
3. Press and hold the M button until two horizontal lines flash, Fig. 4. This indicates that all measurements have been erased.

Fig. 4

---

## **CARE AND MAINTENANCE**

---

- Take care not to drop the monitor or subject it to strong impact.
- Do not use any liquids on the monitor or cuff.
- Use a soft, dry cloth to clean your monitor and LCD screen.
- Do not store in direct sunlight, dust or excessive humidity.
- Remove batteries if monitor will not be used for long periods of time.
- Avoid extreme temperatures.
- Do not disassemble the monitor or cuff.

## DISPLAY SYMBOLS

---

Pulse rate

Measurement error

Irregular heartbeat detected during measurement

Appears flashing when cuff is unstable or it has too much air left in it. Once the arrow stops flashing, the unit is ready to take a measurement.

Appears when a measurement error has occurred; restart the measurement.

Measurement in memory

Memory bank user indicator

Low-battery symbol

# TROUBLESHOOTING

<b>PROBLEM</b>	<b>POSSIBLE CAUSE</b>	<b>SOLUTION</b>
Display shows an abnormal result	Cuff position was not correct or was not tight enough	Apply the cuff properly and attempt a new measurement
	Body posture was not correct during measurement	Review the 'Taking a Measurement' portion of this manual and attempt a new measurement
	Speaking, excessive movement, angry, excited or nervous anxiety during measurement	Wait a period of time and attempt a new measurement after calm has returned; do not speak or move during measurement
Display is blank when power is on	Batteries may be expired or installed improperly	Check the polarity of the batteries and reinstall if necessary; replace batteries with new batteries

---

# PRODUCT SPECIFICATIONS

---

<b>NAME</b>	Jumbo Screen Automatic Digital Blood Pressure Wrist Monitor
<b>MODEL NUMBER</b>	01-516
<b>DISPLAY SYSTEM</b>	Digital display/LCD
<b>MEASURING METHOD</b>	Oscillometric Method, Automatic Air Inflation and Measurement
<b>POWER SOURCE</b>	2 AAA batteries
<b>MEASURING RANGE</b>	Pressure: 0-300 mmHg Pulse: 40-180 beats/minute
<b>ACCURACY</b>	Pressure: $\pm 3$ mmHg
<b>DEFLATION</b>	Automatic pressure release valve
<b>MEMORY</b>	Built-in memory enabling display of up to 120 measurements ( 60 per bank)
<b>AUTOMATIC SHUT-OFF</b>	Approximately 1 minute after last button operation
<b>BATTERY LIFE</b>	Approximately 1 month with normal usage
<b>OPERATION ENVIRONMENT</b>	Temperature 41–F – 104oF (5oC – 40oC) Humidity < 80% RH max
<b>OPERATION ENVIRONMENT</b>	Temperature -4oF – 131oF (-20oC – 55oC) Humidity < 95%
<b>MONITOR DIMENSIONS</b>	3-1/3" x 3" x 1-1/4"
<b>WEIGHT</b>	4.9 oz. (without batteries)
<b>WRIST CIRCUMFERENCE RANGE</b>	Suitable for 5-1/2" – 7-5/8" (14 – 19.5 cm)
<b>ACCESSORIES</b>	Storage case, Instruction Manual, two AAA alkaline batteries, Quick Start Guide, Blood Pressure Log

*Specifications are subject to change without notice*

---

---

## **FCC STATEMENT**

---

### **NOTE:**

#### POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital

device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

---

## **FCC STATEMENT**

---

### POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

---

## LIFETIME LIMITED WARRANTY

---

Congratulations on your purchase of a Veridian Digital Blood Pressure Monitor. Your Digital Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

Veridian Healthcare warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for as long as the original purchaser owns the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of Veridian, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid along with \$5.00 for return shipping, handling and insurance to: **Veridian Healthcare, Attn: Repair Department, 1465 S. Lakeside Drive, Waukegan, IL, 60085**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and

---

## LIFETIME LIMITED WARRANTY

---

(iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, Veridian Healthcare will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL VERIDIAN HEALTHCARE BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

**Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.**

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