Jumbo Screen Premium Blood Pressure Wrist Monitor

Model 01-516

THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.

Before applying your blood pressure cuff, be sure your wrist is within the appropriate range:

wrist circumference 5-1/2" – 7-5/8" (14-19.5 cm)



A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.

Applying the Wrist Cuff

1. Be seated at a table or desk with your feet flat on the floor. Hold your left arm in front of you with your palm facing upward.


2. Apply the preformed cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you.

3. Adjust the cuff 1/4" - 1/2" from the edge of the head of the ulna bone. The cuff should fit comfortably, yet snugly around your wrist.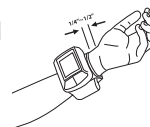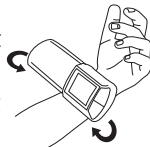

Taking the Measurement

1. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand.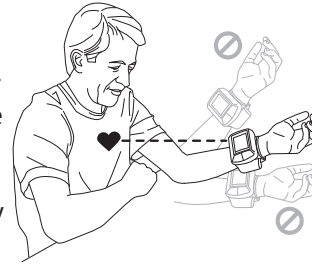

2. Press the **⏻** (POWER) button to turn the unit on; the unit will run a self-test, Fig 1.

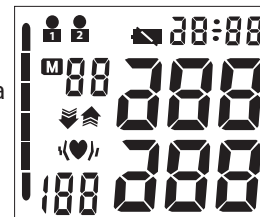
Fig. 1

3. The display will show the current memory bank in use, 1 or 2,  at the top left of the display. Press the **M** button to select a different memory bank if desired.

4. The cuff will automatically inflate to approximately 190mmHg or 40 mmHg higher than the last systolic pressure reading stored in memory.

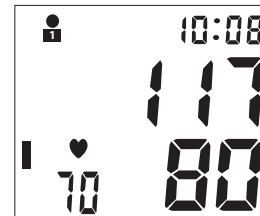
Fig. 2

5. When the measurement is complete and the cuff deflates, your blood pressure measurement and pulse will display simultaneously on the display screen. Both the systolic and diastolic readings will display at the same time, Fig 2.

6. Consult the included Instruction Manual for information on interpreting your reading results.

7. The results will automatically be stored in the memory bank selected, up to 60 readings.

8. Select **⏻** POWER to turn the unit off and conserve energy and battery life. The unit will automatically shut-off and revert to time/date mode after approximately 1 minute.

Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the **⏻** (POWER) button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.

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