
Two-Party Home Blood Pressure Kit

Model 01-5521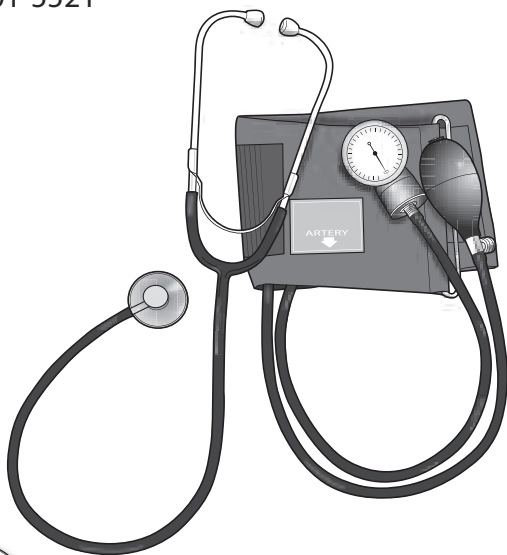

INSTRUCTION MANUAL • ENGLISH & ESPAÑOL

STOP!
PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING
COMPONENTS BEFORE USING YOUR HOME BLOOD
PRESSURE KIT

Unassembled, 3-piece Stethoscope

Gauge


Squeeze Bulb

Adult Arm Cuff

Canvas Storage Bag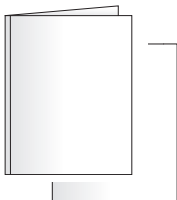

Instruction Manual
Blood Pressure Reading Tracking Log
FAQ Information Sheet
Replacement Parts Order Form

IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.

INDEX

Introduction	4
Getting Started.....	5
Features and Assembly of Your Home Kit.....	6
Applying Your Blood Pressure Cuff.....	7
Proper Stethoscope Use and Fit.....	8
Taking Your Blood Pressure Reading	9-10
Care and Maintenance	10
Warranty	11
Instrucciones en Español.....	13-24

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST


Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com

Made in China
#93-1063 06/12
©2012 Veridian Healthcare, LLC

INTRODUCTION

Veridian Healthcare recommends that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.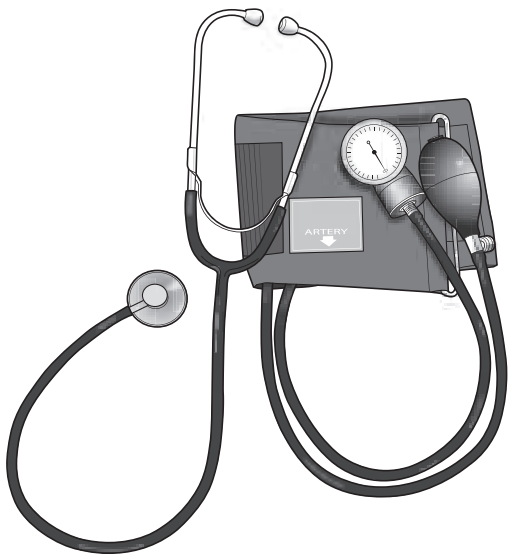


Thank you for purchasing a Veridian Healthcare Two Party Home Blood Pressure Kit. With proper care and use, your kit will provide you with many years of reliable measurements.

GETTING STARTED

Before Taking a Measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular home blood pressure monitoring will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Remain still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

FEATURES AND ASSEMBLY OF YOUR HOME KIT

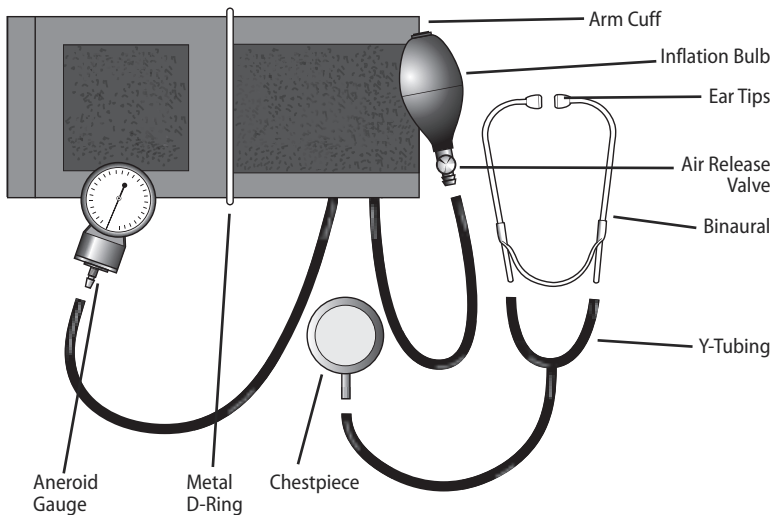Simple Assembly -

Stethoscope: Insert the single end of the “Y” tubing into the stem of the chestpiece; Insert the ends of the metal binaurals from the polybag and insert the ends into the “Y” tubing.

Cuff, Bulb and Gauge: Attach the gauge and bulb to the tubing as shown above.

APPLYING YOUR BLOOD PRESSURE CUFF

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:

Adult Cuff fits arm circumference 10" - 14" (25.4-35.5 cm)

Additional cuff sizes are available for order on the enclosed order form.

If for any reason you are unable to or should not use your left arm, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you.

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Be seated at a table or desk with your feet flat on the floor.
3. Extend your left arm. Find your brachial artery, which is at the inner aspect (bend) of your elbow, *Fig 1*. You may be able to feel your pulse. The artery patch should point to the brachial artery when the cuff is applied.
4. Open the cuff creating a circle that you can slide your arm into; do not extend the cuff beyond the metal bar.
5. Insert your left arm into the cuff. Position the bottom edge of the cuff approximately one-half inch ABOVE your elbow; when taking your reading, the stethoscope chestpiece must directly touch the inside of your arm, over your brachial artery.
6. With your arm extended, palm facing upward and tubes positioned down the center of your arm, *Fig 2-3*, use your right hand to pull the end of the cuff and secure the hook and loop material. Make sure that the artery mark is still pointed to your brachial artery.
7. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger easily between your arm and the cuff.

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

Fig. 1

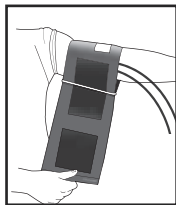
Fig. 2

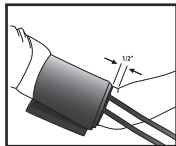
Fig. 3

PROPER STETHOSCOPE USE AND FIT

The earpieces on your Two-Party Home Blood Pressure Kit stethoscope have been carefully calibrated to provide clear and audible acoustics. To best hear the sounds to be analyzed, please keep the following tips in mind:

The earpieces should point forward when in use, *Fig 1*. The earpieces should fit in ears securely but comfortably.

The binaurals may be adjusted to ensure maximum comfort and efficacy. To increase the tension, simply squeeze the binaurals just above the joint where they meet, crossing the earpieces, *Fig 2*. Do not apply too much pressure; apply a minimum of force, slightly increasing the pressure until the earpieces fit comfortably.

Should the earpieces be too tight, pull the binaurals apart, grasping them just above the joint where they meet, *Fig. 3*. Be gentle so as not to damage the stethoscope.

Always use caution when adjusting the binaurals to avoid damage. When bringing the earpieces closer together, the maximum tension used should bring the black tubing on either side within contact. When pulling the binaurals apart, do not exceed 90 degrees between the binaurals.

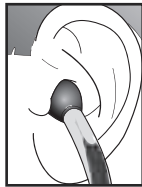
Fig. 1 -
Correct Position

Fig. 1 -
Incorrect Position

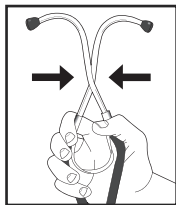
Fig. 2

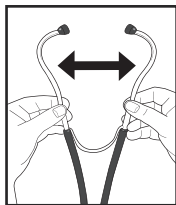
Fig. 3

TAKING YOUR BLOOD PRESSURE READING

Before taking a reading, review the section under Features & Assembly of your Home Blood Pressure Kit and ensure every piece is properly assembled. Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

NOTE: For the purposes of these instructions, the individual taking the reading will be referred to as the CAREGIVER, the person receiving the reading will be referred to as the PATIENT.

Applying the Cuff and Arm Positioning - Patient

1. Apply the cuff as instructed earlier in this manual.
2. Rest your elbow on a table with your palm and arm facing upward. Elevate your hand so that the wrist is at the same level as your heart, *Fig 1*. Relax your left hand.

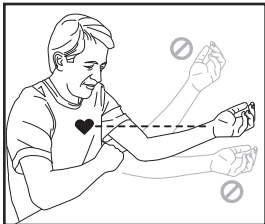
Fig. 1

Preparing to Take the Measurement - Caregiver

3. Position yourself in front of the patient, seated at the same level.
4. Place the stethoscope eartips in your ears and place the chestpiece of the stethoscope on the inside of the patient's arm. The chestpiece should be partially under the cuff directly over the brachial artery.
5. Either hold the gauge in your hand or position the gauge so it can be read without being moved, such as clipped to a thick notebook or folded towel in front of you.
6. Hold the inflation bulb with your free hand and turn the air release valve clockwise to close, *Fig 2*.
7. Begin inflating the cuff slowly and steadily by pumping the inflation bulb until the gauge reads 30 mmHG ABOVE the last SYSTOLIC measurement. If you do not know the last SYSTOLIC measurement, inflate to approximately 180-200 mmHg. NEVER INFLATE THE CUFF ABOVE 280 mmHg.

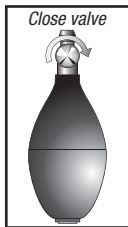
Fig. 2

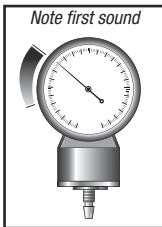
Fig. 3

TAKING YOUR BLOOD PRESSURE READING

- When you have reached the proper inflation level, slowly and steadily turn the air valve counter-clockwise at a rate of 2-3 mmHg per second. You will be able to watch the progress of the release on the gauge.
- As the pressure drops, listen closely for the pulse beat while watching the gauge. Note the first thumping sound you hear, this is the SYSTOLIC pressure, *Fig 3*.
- Allow the cuff to continue to deflate. You will hear changes in the sounds and then no sound at all. When you have heard the last sound, note the reading measurement, this is the DIASTOLIC pressure, *Fig 4*.
- Quickly turn the valve counter-clockwise to deflate the remaining air, *Fig 5*. Remove the cuff and record the measurement on the included chart.

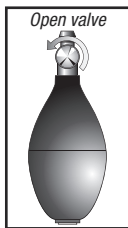
Fig. 4

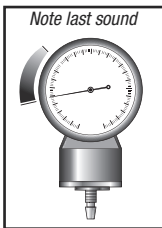
Fig. 5

IF YOU WISH TO TAKE ANOTHER MEASUREMENT, WAIT A MINIMUM OF 15 MINUTES BETWEEN READINGS TO ALLOW FOR THE BLOOD VESSELS TO RETURN TO NORMAL.

CARE & MAINTENANCE

- Do not use any liquids on the gauge or cuff.
- Do not store in direct sunlight, dust or excessive humidity.
- Avoid extreme temperatures.
- Do not disassemble the gauge or cuff.
- Avoid dropping or hitting the surface of the gauge or stethoscope chestpiece.

When the cuff is fully deflated, make sure that the needle on the gauge is within the zero square range. If the needle is outside of the zero range, it needs to be recalibrated by a qualified technician. Ship your gauge, prepaid, along with \$5.00 for return shipping and insurance, to Repair Department, 1175 Lakeside Drive, Gurnee, IL 60031.

1-YEAR LIMITED WARRANTY

Veridian Healthcare guarantees that its Two-Party Home Blood Pressure Kit will be free from manufacturing defects under normal use for a period of one-year. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty is voided if the unit is misused or abused in any manner.

If the Two-Party Home Blood Pressure Kit fails to operate within the warranty period while under the original purchaser owns the product, return it postage prepaid along with \$5.00 for return shipping and insurance to: Veridian Healthcare, Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031; include your name, address, phone number and brief description of the problem with your product. Take care to carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. Veridian Healthcare will repair or replace the defective unit at our option. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only reasonability and obligation to repair and/or replace materials or components, or refund the purchase price. Veridian will not be responsible for any indirect, incidental, special consequential or punitive damages or other loss, including, but not limited to damage to or loss of other property or liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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