SMARTHEART™

Blood Pressure Monitor

Model # 01-572


INSTRUCTION MANUAL **ENGLISH AND SPANISH**

Please read this instruction manual completely before operating this unit.


STOP!
PLEASE ENSURE YOU HAVE ALL OF THE
FOLLOWING COMPONENTS BEFORE USING
YOUR DIGITAL BLOOD PRESSURE MONITOR


Digital Monitor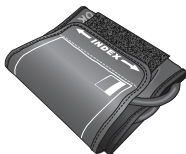

Adult Arm Cuff

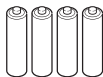Four (4) AAA
Batteries

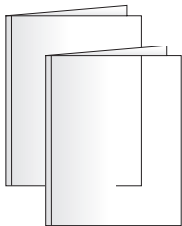Instruction Manual
Quick Start Guide
Blood Pressure Reading Tracking Log
FAQ Information Sheet
Replacement Parts Order Form

IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.

INDEX

Care & Safety Information	4-5
Introduction & Indications for Use	6
Blood Pressure Monitor Features	7
Getting Started	8
Battery Installation	9
Setting Date/Time	10-11
Fitting & Applying Your Cuff	12-13
Taking Your Blood Pressure Reading	14-15
Interpreting Your Results	16-17
Memory Function	18-19
Care & Maintenance	20-21
Device & Label Symbols	22
Display Symbols	23
Troubleshooting	24
Error Codes	25
FCC Statement	26
Electromagnetic Compatibility	27-29
Product Specifications	30
Warranty	31
Instrucciones en Español	33-63
Reference Page	64

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

SmartHeart™
Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1331 03/22
©2022 Veridian Healthcare, LLC

CARE & SAFETY INFORMATION

NOTE: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

CAUTION: Failure to read and observe all precautions could result in personal injury or equipment damage.

Improper care or use of your blood pressure monitor may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the blood pressure monitor's efficacy and long life.

GENERAL CAUTIONS AND WARNINGS

- Measurement results should be discussed with your physician or healthcare professional; never self-diagnose or attempt treatment as this may be dangerous.
- Intended for adult use only; this device is not approved for infant or child use.
- Keep out of reach of children.
- Do not use the unit if it has any damaged parts, if it has been submersed in water or dropped.
- If any abnormality occurs, discontinue use until the unit has been examined and repaired.
- Only use included attachments and parts; do not use attachments from other brands or models as these may not be properly calibrated for use with this device and may result in measurement error.
- Do not disassemble the unit or attempt to repair it; substitution of a component different from that supplied may result in measurement error and will void manufacturer warranty.
- Always follow local regulations for proper disposal of the monitor, cuff and batteries.
- Do not use this device with other diagnostic monitoring equipment.
- Take Care to note the blood circulation of the user and discontinue use if prolonged discomfort occurs.

CARE & SAFETY INFORMATION

- It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.
- Do not share this device with infected persons where there may be a risk of cross-contamination.
- Consult with your physician before use if any of the following conditions are present: arrhythmia, inflammatory diseases, intravascular therapy, arteriovenous shunts or if a mastectomy has been performed.

OPERATING CAUTIONS AND WARNINGS

- Prolonged over-inflation may cause congestion, swelling or bruising in some people.
- If you experience any discomfort or an abnormality occurs, stop using the device immediately; to stop the device during operation, press the **START STOP** button and the cuff will automatically deflate.
- Operation outside of stated operating temperature may result in measurement error or device malfunction; operation environment temperature is: 41°F – 104°F (5°C – 40°C); Humidity: 15% RH to 90%RH.
- Never use this unit while operating a vehicle or in the bathtub or shower.

STORAGE CAUTIONS AND WARNINGS

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -4°F – 140°F (-20°C – 60°C); Humidity: ≤93%RH.
- Keep the unit out of reach of children.
- Always keep the unit unplugged while not in use; remove the batteries if the unit will not be used for an extended period of time.

CLEANING CAUTIONS AND WARNINGS

- Never immerse the unit in water to clean as it may damage the unit.
- Follow the 'Cleaning and Maintenance' portion of this manual for instruction on how to clean and care for your monitor.

INTRODUCTION & INDICATIONS FOR USE

It is recommended that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.

Digital blood pressure monitors are intended for use in measuring blood pressure and heart rate for users with the arm circumference of 8.6" –12.5" (22 – 32 cm). This device is intended for indoor use only.


Automatic digital blood pressure monitors use the oscillometric method to electronically measure your blood pressure. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor ideal for home use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained healthcare professional using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

Thank you for purchasing a SmartHeart Automatic Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

**PLEASE READ THIS INSTRUCTION MANUAL
COMPLETELY BEFORE OPERATING THIS UNIT.**

BLOOD PRESSURE MONITOR FEATURES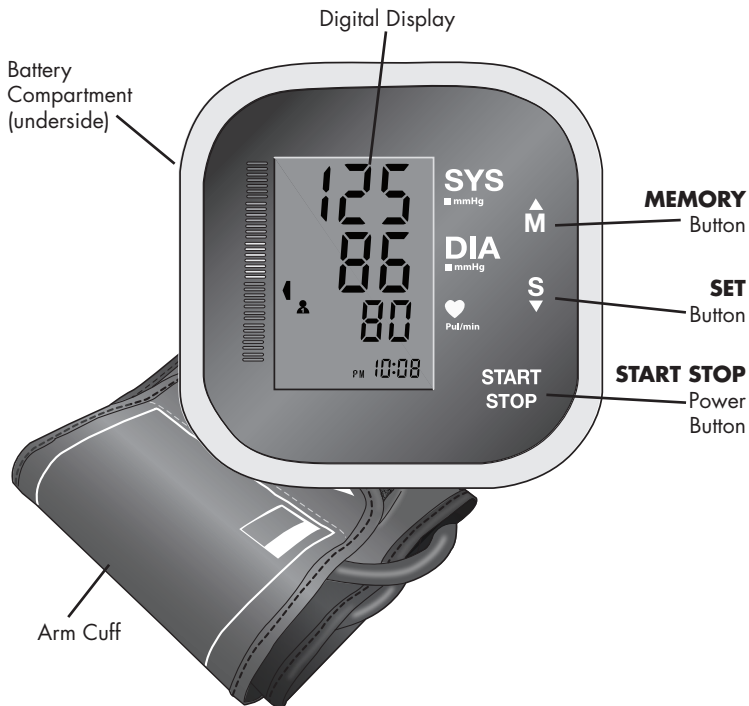

DETAILS OF THE DIGITAL DISPLAY SCREEN AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.

GETTING STARTED


Before Taking a Measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

BATTERY INSTALLATION


This unit requires 4 AAA batteries.

It is necessary to replace the batteries when the “Low Battery” symbol appears on the display or when the display does not turn on after the **START STOP** button is pressed.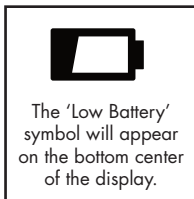

The battery compartment is located on the underside of this monitor.

1. Depress the clip on the battery compartment cover and remove the cover.
2. Insert or replace 4 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.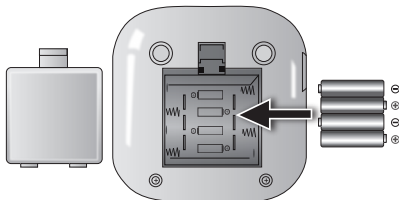


It is recommended to remove the batteries if the unit will not be used for an extended period of time.

SETTING DATE/TIME

IT IS NECESSARY TO SET THE DATE AND TIME FOR THE UNIT EVERY TIME BATTERIES ARE INITIALLY INSTALLED OR REPLACED


The date/time feature is on the lower right of the display screen. This unit stores the date and time, including year allowing for a date/time stamp on all readings in the memory recall function.

The **Memory**, **Set** and **Start•Stop** buttons are located on the right side of the display.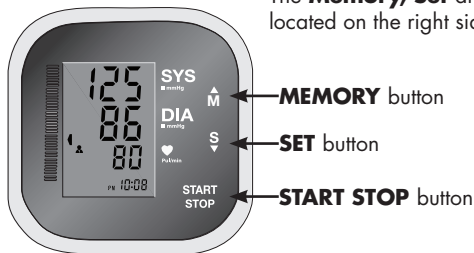


1. When the batteries are first installed, the device will automatically prompt you to set the Date/Time feature.*
*NOTE: If the batteries have already been installed—when the unit is off (a blank display screen), press and hold the **S** (SET) button for 3-5 seconds to make the unit go into clock setting mode.
2. **YEAR**—next the year will flash on the display. Press the **M** button to advance the display to the desired year, press the **S** button to confirm your selection and advance to the date settings.
3. **MONTH/DATE**—repeat the same sequence of step 2 to confirm the **MONTH** and **DATE** setting.
4. **HOURS/MINUTES**—repeat the same sequence of step 2 to confirm the **HOURL** and **MINUTE** setting.
5. After the **Date/Time** setting function is complete, the monitor is ready for use.


SETTING DATE/TIME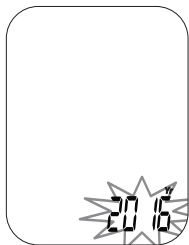

Year Selection Mode

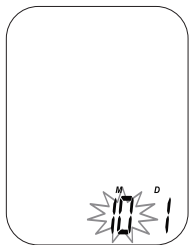Month Selection Mode

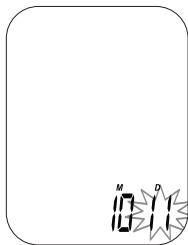
Day Selection Mode

Hour Selection Mode

Minutes Selection Mode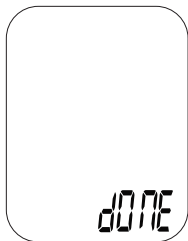


Complete

FITTING & APPLYING YOUR CUFF

PROPER CUFF FIT AND PLACEMENT IS CRITICAL IN OBTAINING ACCURATE BLOOD PRESSURE MEASUREMENTS. PLEASE READ THESE INSTRUCTIONS CAREFULLY AND CONSIDER CONSULTING WITH YOUR PHYSICIAN FOR FURTHER DIRECTION.

Before applying your blood pressure cuff, be sure the cuff included with this monitor is the appropriate size cuff.

To determine the proper cuff size, measure the circumference of your left upper arm at the point midway between inside bend of your elbow and your shoulder.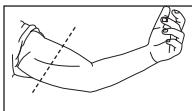

Adult Cuff fits arm circumference 8.6" – 12.5" (22–32cm)

If for any reason you are unable to or should not use your left arm, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you to take measurements from.

Assembling Your Cuff

Your cuff may need to be 'threaded' through the metal bar to form the cuff cylinder for your arm.

1. Lay the open cuff hook and loop side down, size/instruction wording side up on a flat surface.
2. Keeping the words on the inside of the cuff, thread the end of the cuff through the metal bar, Fig 1. The end of the cuff should be able to fold over onto itself for securing via the hook and loop closure, Fig 2.

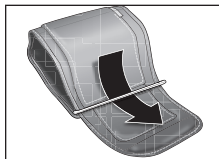
Fig. 1

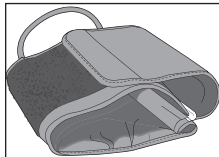
Fig. 2

FITTING & APPLYING YOUR CUFF

Applying Your Cuff

Be seated at a table or desk with your feet flat on the floor. To avoid accidentally pulling on or dropping your monitor, the cuff should not be plugged into the monitor until after the cuff is applied to your arm.

The cuff is printed with a useful guide for applying the cuff; please consult the following steps for additional direction.

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Slide the cuff cylinder onto your left arm and align the artery mark over the brachial artery on the inside of your arm, Fig 3.

The tube should follow the inside length of the arm, Fig 4.

3. The bottom edge of the cuff should be positioned approximately 0.5" above the elbow joint, Fig 5.
4. When the cuff is closed around the arm, the triangle guide imprint should be within the OK range bar marking; if the triangle is outside of the range, a different sized cuff is needed
5. The cuff should fit comfortably, yet snugly around your arm, Fig 6. You should be able to insert one finger easily between your arm and the cuff.

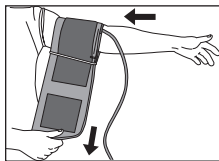
Fig. 3

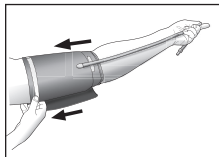
Fig. 4

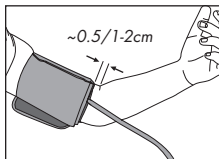
Fig. 5

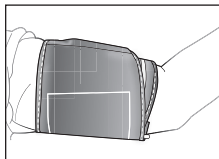
Fig. 6

TAKING YOUR BLOOD PRESSURE READING

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

Select the desired user, **1** or **2**, before measurement.

The user icon displayed represents the memory bank where the measurement result will be stored.

1. When the monitor is OFF, press and release the **S** button. The display turn on and the current user, **1** or **2**.
2. Press **S** again to toggle between user **1** or **2**.
3. Press **START STOP** to confirm the selection.

Proceed to the measurement steps below.

1. Position the monitor on a flat, stable surface with the digital display in view. Be seated with your feet flat on the floor and legs uncrossed.
2. After applying the cuff, insert the cuff tubing connector into the port on the left side of your monitor.
3. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Relax your left hand.
4. Press and release the **START STOP** button to turn the unit on; the unit will briefly flash a self-test, Fig 2.
5. The display will briefly show a '0' and down arrow before the measurement begins.
6. The cuff will automatically inflate; you will see the numbers increase on the display screen and feel the cuff inflate.

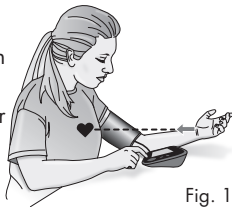
Fig. 1

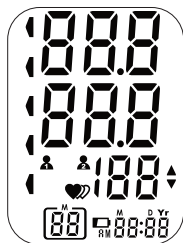
Fig. 2

TAKING YOUR BLOOD PRESSURE READING

YOU CAN STOP THE INFLATION OR DEFLATION PROCESS AT ANY TIME BY PRESSING THE POWER BUTTON.

- When the measurement is complete, the cuff will automatically deflate, then your blood pressure measurement and pulse reading results will show on the display screen at the same time, Fig 3.
- The Hypertension Indicator will indicate your reading range on the left display in comparison to the color bars on the face of the monitor, Fig 4. See page 17 for information regarding interpreting the Hypertension Indicator results.
- If an irregular heartbeat was detected during the reading, the Irregular Heartbeat Detector indicator will appear on the display, Fig 5. See page 16 for more information on the Irregular Heartbeat Detector feature.
- The reading will automatically be stored in memory, up to 60 readings per user.
- Select **START STOP** to turn the unit off and conserve energy and battery life.
- The unit will automatically shut-off approximately 1 minute after use.
- Disconnect tubing prior to storage.

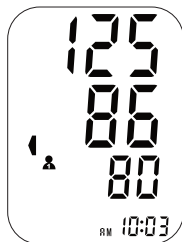
Fig. 3

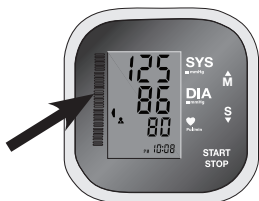
Fig. 4



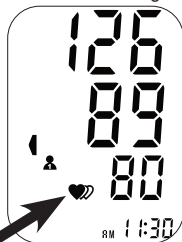
Fig. 5

INTERPRETING YOUR RESULTS

Irregular Heartbeat

An irregular heartbeat is defined as a heartbeat rhythm that is more than 25% slower or 25% faster than the average measurement detected while the monitor is conducting the measurement. When this device detects an irregular rhythm two or more times during the reading, the IHB symbol will appear on the display.

This monitor will operate if the icon appears, but the results may be compromised, especially if this symbol appears often. Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently.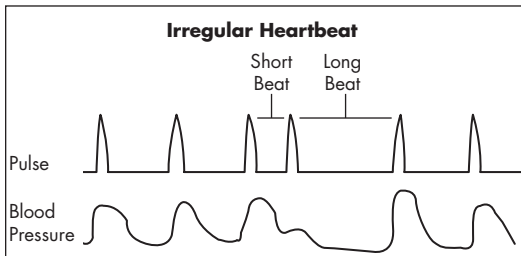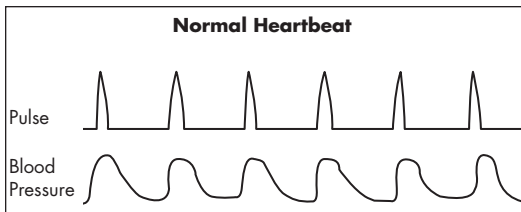


INTERPRETING YOUR RESULTS

Hypertension Indicator

This unit features our unique Hypertension Indicator. The color bars on the left side of the monitor display correspond with an icon on the digital display, indicating where the measurement results fall within the American Heart Association standards.

This chart should only be considered as a guideline; always consult with your physician to interpret your individual results. Never adjust your medication(s) dosage, or severely alter your diet or exercise routine without consulting with your physician.

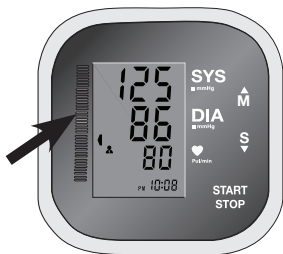Category

Systolic (upper #)

Diastolic (lower #)

Hypertensive Crisis	≥180	and/or	≥120
High Blood Pressure, Stage 2	≥140	or	≥90
High Blood Pressure, Stage 1	130-139	or	80-89
Elevated / Prehypertension	120-129	and	<80
Normal	<120	and	<80

MEMORY FUNCTION

Recalling Measurements in Memory

Select the desired user, 1 or 2, before measurement.

The user icon displayed represents the memory bank where the measurement result will be stored.

1. When the monitor is OFF, press and release the **S** button. The display turn on and the current user, **1 or 2**.
2. Press **S** again to toggle between user **1 or 2**.
3. Press **START STOP** to confirm the memory bank to review.

This monitor stores and recalls up to 60 readings per user. These measurements can be shared with your physician or trained healthcare professional.

1. When the unit is OFF, press and release the **M** button to select the memory bank for review.
2. The unit will first display the average of the last three measurements, Fig 1. If there are less than 3 records in memory, the display will show the latest record first.
3. Press the **M** button again to begin scrolling through the saved measurements. Measurements will appear on the display from most current to oldest; the memory number will appear flashing at the bottom of the screen next to the measurement that was taken.
4. All results for a given measurement will display, including measurement results, pulse rate, Hypertension Indicator and Irregular Heartbeat alert (if applicable). The date/time stamp is at the bottom right of the display, Fig 2.

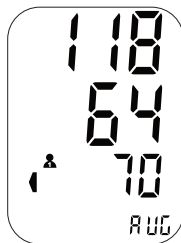
Fig. 1

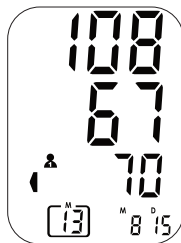
Fig. 2

MEMORY FUNCTION

- When the number of readings exceeds 60 , the oldest data will be replaced with the new record.
- Press **START STOP** to turn the monitor off at any time during review of the stored measurements.

Clearing Measurements From Memory

To delete a all results in a given bank:

- When the monitor is off, press and release the **M** button to access the memory bank feature.
- Press and hold the **M** and **S** buttons at the same time; the display will flash dEL and ALL, Fig 1.
- Press the **S** button again to confirm you would like to delete the memory bank, Fig 2; the display will show dONE and turn off.

Note: To exit out of delete mode without deleting any records, press **START•STOP** button before pressing **S** button to confirm any delete commands.

Fig. 1

Fig. 2

CARE & MAINTENANCE

Proper care and maintenance of your blood pressure monitor will help ensure long life for the product; improper use of the monitor may void the included warranty.

Always disconnect the cuff and tubing from the monitor prior to cleaning and storage.

Monitor Care and Maintenance

- Do not use any liquids on the monitor; use a soft, dry cloth to clean the monitor as needed.
- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: $-4^{\circ}\text{F} - 140^{\circ}\text{F}$ ($-20^{\circ}\text{C} - 60^{\circ}\text{C}$); Humidity: $\leq 93\%$ RH.
- Avoid high temperatures and direct sunlight.
- Always remove the batteries if the unit will not be used for an extended period of time.
- Do not attempt to disassemble your monitor; disassembling your monitor may void the manufacturers warranty.
- Do not subject the monitor to strong shocks; take care not to drop the monitor.
- No component can be maintained by user in the monitor. The circuit diagrams, component part lists, descriptions, calibrations instructions, or other information which will assist the user's appropriately qualified technical personnel to repair those parts of equipment which are designated repairable can be supplied.

CARE & MAINTENANCE

Cuff Care and Maintenance

- The cuff may be spot cleaned with a mild-detergent as needed.
- To disinfect the cuff, wipe the inside (skin contact side) of the cuff using a cloth lightly moistened with 70% - 90% ethyl rubbing alcohol and allow to air dry; it is suggested to disinfect the cuff frequently when used in clinical settings or if the cuff is contaminated in any way.
- Never attempt to iron or dry the cuff; if the cuff becomes wet, allow the cuff to air dry completely before use or storage.
- Do not attempt to remove the tube and bladder from the nylon cuff as this may damage the cuff.
- Take care not to forcefully bend or kink the tubing during storage.
- It is recommended the performance should be checked every 2 years or after a repair. Please Contact Customer care at 866-326-1313.
- It is recommended to clean the cuff every 200 uses.
- The monitor can maintain the safety and performance characteristics for minimum of 10,000 measurements of three years.
- The cuff integrity is maintained after 1,000 open- close cycles of the closure.

DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Meaning
	Read This Manual—All included manuals should be read prior to first use.
	Warning—Symbol indicates a warning, prohibition or mandatory action that mitigates a risk that is not necessarily obvious to the device operator.
	Type BF Applied Parts—Indicates that a part of this unit comes in contact with the patient in order to carry out its intended function; in the case of this device the cuff is the Type BF Applied Part.
	Environment Protection—dispose of this product properly; consult with your local recycling ordinances for proper recycling and disposal.
	Keep Dry—This device should be kept dry; never submerge the unit or cuff. Consult with the Care and Maintenance section of this manual for information on cleaning your monitor.

DISPLAY SYMBOLS

Symbol	Meaning
SYS	Systolic Blood Pressure—The ‘top’ blood pressure result
DIA	Diastolic Blood Pressure—The ‘bottom’ blood pressure result
PUL	Pulse Rate—Pulse rate per minute during the measurement
mmHg	Millimeters of Mercury—Unit of blood pressure measurement for this device
	Low Battery—Battery life indicator
	Pulse Reading—Appears when pulse reading is displayed
	Irregular Heartbeat Symbol—An irregular heartbeat was detected during the measurement
	User 1/User 2—Indicates which user is active
AUC	Average—The display shows the average of the last three measurements in memory
M	Memory—The display shows the measurements stored in memory

TROUBLESHOOTING & ERROR CODES

Problem	Possible Cause	Solution
Display shows an abnormal result.	Cuff position was not correct or was not tight enough.	Apply the cuff properly and attempt a new measurement.
	Body posture was not correct during measurement.	Review the 'Taking a Measurement' portion of this manual and attempt a new measurement.
	Speaking, excessive movement, angry, excited or nervous anxiety during measurement.	Wait a period of time and attempt a new measurement after calm has returned; do not speak or move during measurement.
Display is blank when power is on.	Batteries are expired.	Replace batteries; see Replacing Batteries section of this manual for further instruction.
Low battery symbol + Lo appear on the display.	Battery power is low.	Replace batteries; see Replacing Batteries section of this manual for further instruction.

TROUBLESHOOTING & ERROR CODES

Code	Meaning	Corrective Action
E1	The cuff is not secure.	Re-apply the cuff and attempt new measurement.
E2	The cuff is too tight.	Re-apply the cuff and attempt new measurement.
E3	The cuff pressure is in excess.	Relax for several minutes and attempt new measurement.
E10 or E11	The monitor detected motion while measuring.	Sit comfortable and remain still; attempt a new measurement.
E20	The measurement process does not detect the pulse signal.	Re-apply the cuff and attempt new measurement.
E21	The unit measured incorrectly.	Relax for a moment and then measure again.
EExx	A calibration error occurred.	Retake the measurement; if the problem persists, contact Customer Care.
OUT	Out of measurement range	Relax for a moment and then measure again. If the problem persists, it is suggested you discuss your results with your physician.

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

ELECTROMAGNETIC COMPATIBILITY

Table 1

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation."

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Technical description:

1. All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the excepted service life.
2. Guidance and manufacturer's declaration -electromagnetic emissions and Immunity.

Guidance and manufacture's declaration - electromagnetic emissions

Emissions Test	Compliance
RF emissions CISPR11	Group 1
RF emissions CISPR11	Class B
Harmonic distortion IEC 61000-3-2	Class A
Voltage fluctuations / flicker emissions IEC 61000-3-3	Comply

ELECTROMAGNETIC COMPATIBILITY

Table 2

Guidance and manufacture's declaration - electromagnetic immunity

Immunity Test	IEC 60601-1-2 test level	Compliance level
Electrostatic Discharge IEC 61000-4-2	±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV signal input/output 100 kHz repetition frequency	±2 kV for power supply lines ±1 kV signal input/output 100 kHz repetition frequency
Surge IEC 61000-4-5	±0.5 kV, ±1 kV differential mode ±0.5 kV, ±1 kV,±2 kV common mode	±0.5 kV, ±1 kV differential mode ±0.5 kV, ±1 kV,±2 kV common mode
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°.0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°.0 % UT; 250/300 cycle	0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°.0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°.0 % UT; 250/300 cycle
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz

NOTE UT is the a.c. mains voltage prior to application of the test level.

ELECTROMAGNETIC COMPATIBILITY

Table 3

Guidance and manufacture's declaration - electromagnetic immunity

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band a (MHz)	Service	Modulation	Maximum Power (w)	Distance (m)	IEC 60601-1-2 Test Level (V/m)	Compliance level (V/m)
	385	380-390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27	27
	450	380-390	GMRS 460, FRS 460	FM \pm 5kHz deviation 1kHz sine	2	0.3	28	28
	710 745 780	704-787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9	9
	810 870 930	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18Hz	2	0.3	28	28
	1720 1845 1970	1700- 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217Hz	2	0.3	28	28
	2450	2400- 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217Hz	2	0.3	28	28
	5240 5240 5785	5100- 5800	WLAN 802.11 a/n	Pulse modulation 217Hz	0.2	0.3	9	9

PRODUCT SPECIFICATIONS

Name	Automatic Blood Pressure Monitor
Model Number	01-572
Display System	Digital display/LCD
Measuring Method	Oscillometric Method, Automatic Air Inflation and Measurement
Power Source	4 x 1.5V  4 AAA Batteries
Measuring Range	Rated Cuff Pressure: 0-299mmHg; Measurement pressure: SYS 60-230mmHg; DIA 40-130 mmHg; Pulse Value: 40-199 beats/minute
Accuracy	Pressure: 5°C-40°C within ± 3 mmHg; Pulse: $\pm 5\%$
Deflation	Automatic pressure release valve
Memory	Built-in memory with 2 banks, 60 measurements per bank
Automatic Power-Off	Approximately 1 minute after last button operation
Battery Life	Approximately 270 measurements
Operation Environment	Temperature 41°F - 104°F (5°C - 40°C); Humidity 15 - 90%RH; non-condensing but not requiring a water vapor partial pressure greater than 50hPa; atmospheric pressure range of 700 hPa to 1060 hPa
Storage Environment	Temperature -4°F - 140°F (-20°C - 60°C); Humidity $\leq 93\%$ RH non-condensing at water vapor pressure up to 50 hPa
Monitor Dimensions	4.33" x 4.33" x 1.61"
Monitor Weight	5.9 oz. (without batteries)
Arm Circumference Range	Adult size cuff fits arm circumference: 8.6" - 12.5" (22-32cm)
Accessories	Instruction Manual, Quick Start Guide, FAQ, Replacement Cuff Order Form, Blood Pressure Log
Additional Options	Additional cuffs available on the enclosed order form or by calling Customer Care
Degree of Protection	Type BF Applied Part
Protection Against Ingress of water	IP21 means the device could protect against solid foreign objects of 12.5 mm and greater and protect against vertically falling water drops.
Software Version	A01

Specifications are subject to change without notice.

3-YEAR LIMITED WARRANTY

Congratulations on your purchase of a Digital Blood Pressure Monitor. Your Digital Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for a period of three years for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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