# SMARTHEART™

## Blood Pressure Monitor with Adult Wrist Cuff


Model # 01-508, 01-508EM, 01-508RB, 01-508SP


### **INSTRUCTION MANUAL** **ENGLISH AND SPANISH**

Please read this instruction manual  
completely before operating this unit.

**STOP!**  
**PLEASE ENSURE YOU HAVE ALL OF THE**  
**FOLLOWING COMPONENTS BEFORE USING**  
**YOUR DIGITAL BLOOD PRESSURE MONITOR**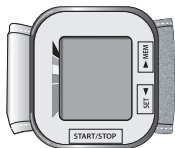


Digital Monitor

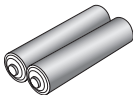2 AAA Batteries

Storage Case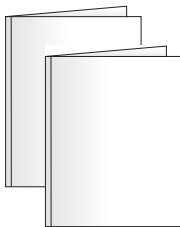

Instruction Manual  
Quick Start Guide  
Blood Pressure Reading Tracking Log  
FAQ Information Sheet

**IF YOU ARE MISSING ANY PARTS, INCLUDING  
INSERTS OR INSTRUCTION MANUALS, DO NOT  
RETURN TO PLACE OF PURCHASE. CONTACT  
CUSTOMER CARE AT 866-326-1313.**

---

# INDEX

---

Care & Safety Information.....	4-6
Introduction & Indications for Use.....	7
Digital Blood Pressure Monitor Features .....	8
Getting Started .....	9
Battery Installation .....	10
Setting Date/Time .....	11
Fitting & Applying Your Cuff.....	12-13
Taking Your Blood Pressure Reading .....	14-15
Interpreting Your Results .....	16-17
Memory Functions .....	18-19
Care & Maintenance.....	20-21
Device & Label Symbols.....	22
Description of Display Symbols.....	23
Troubleshooting .....	24
Error Codes.....	25
FCC Statement.....	26
Electromagnetic Compatibility.....	27-29
Specifications .....	30-31
Warranty .....	32-33
Instrucciones en Español .....	35-67
Reference Page.....	68

---

**Toll-Free Customer Care Help Line: 1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

SmartHeart™  
Manufactured for  
Veridian Healthcare  
1175 Lakeside Drive  
Gurnee, IL 60031

Made in China

#93-1401 10/22  
©2022 Veridian Healthcare, LLC

---

## CARE & SAFETY INFORMATION

---

**NOTE:** Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

**CAUTION:** Failure to read and observe all precautions could result in personal injury or equipment damage.

Improper care or use of your blood pressure monitor may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the blood pressure monitor's efficacy and long life.

### GENERAL CAUTIONS AND WARNINGS

- Measurement results should be discussed with your physician or healthcare professional; never self-diagnose or attempt treatment as this may be dangerous.
- Intended for adult use only; this device is not approved for infant or child.
- Keep out of reach of children.
- Do not use the unit if it has any damaged parts (including power cord or plug and cuff), if it has been submersed in water or dropped.
- If any abnormality occurs, discontinue use until the unit has been examined and repaired.
- Only use included attachments and parts; do not use attachments from other brands or models as these may not be properly calibrated for use with this device and may result in measurement error.
- Do not share the cuff with an infected person.

---

## CARE & SAFETY INFORMATION

---

- Do not disassemble the unit or attempt to repair it; substitution of a component different from that supplied may result in measurement error and will void manufacturer warranty.
- Always follow local regulations for proper disposal of the monitor, cuff and batteries.

## OPERATING CAUTIONS AND WARNINGS

- Prolonged over-inflation may cause congestion, swelling or bruising in some people.
- Wait a minimum of 2 minutes between measurements (20-minutes is recommended); excessive frequent measurements may restrict blood circulation.
- Do not use this device while connected to any monitoring equipment on the same limb.
- Consult with your healthcare professional for guidance for use if you have any of the following: intravascular shunt; mastectomy on the arm corresponding to the measurement side of the body.
- Consult with your healthcare professional before using the device in pregnancy including pre-eclampsia, or if diagnosed with arrhythmia or arteriosclerosis.
- If you experience any discomfort or an abnormality occurs, stop using the device immediately; to stop the device during operation, press the **START/STOP** button and the cuff will automatically deflate.

---

## **CARE & SAFETY INFORMATION**

---

- Operation outside of stated operating temperature may result in measurement error or device malfunction; operation environment temperature is: 41°F – 104°F (5°C – 40°C); Humidity: 15% to 90% RH (non-condensing).
- Never use this unit while operating a vehicle or in the bathtub or shower.

### **STORAGE CAUTIONS AND WARNINGS**

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -4°F – 140°F (-20°C – 60°C); Humidity: ≤93%RH (non-condensing) at water vapor pressure to 50hPa. An atmospheric pressure range of 700 hPa to 1060hPa.
- Keep the unit out of reach of children.
- Always keep the unit unplugged while not in use; remove the batteries if the unit will not be used for an extended period of time.

### **CLEANING CAUTIONS AND WARNINGS**

- Never immerse the unit in water to clean as it may damage the unit
- Follow the 'Cleaning and Maintenance' portion of this manual for instruction on how to clean and care for your monitor.

---

## **INTRODUCTION & INDICATIONS FOR USE**

---

It is recommended that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.

The Blood Pressure Monitor is digital monitors intended for use in measuring blood pressure and heartbeat rate with wrist circumference ranging from 5-1/2" to 8-1/4" inches (about 13.5 – 21.5 cm). It is intended for adult indoor use only.

Automatic digital blood pressure monitors use the oscillometric method to electronically measure your blood pressure. The monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. The oscillometric method does not require a stethoscope, making the monitor ideal for home use.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained healthcare professional using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.


Thank you for purchasing an Automatic Digital Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

**PLEASE READ THIS INSTRUCTION MANUAL  
COMPLETELY BEFORE OPERATING THIS UNIT.**

---

## BLOOD PRESSURE MONITOR FEATURES

---

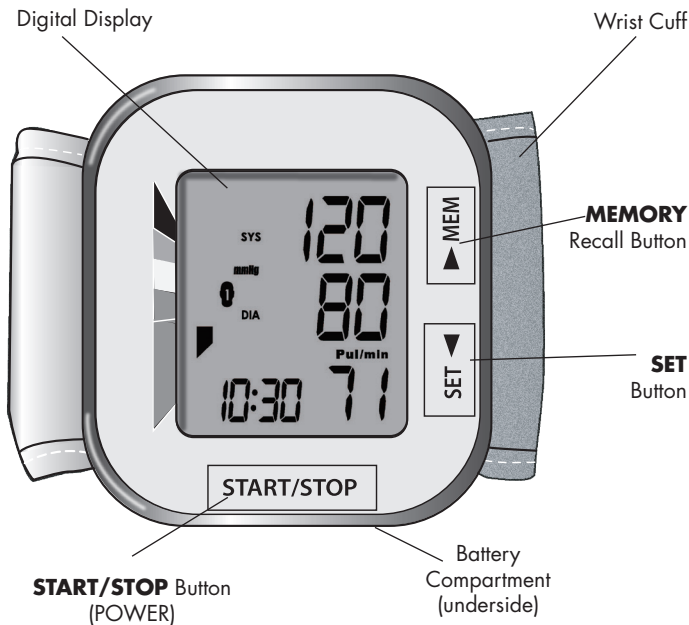**DETAILS OF THE DIGITAL DISPLAY SCREEN AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.**

---

## GETTING STARTED

---

### Before taking a measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

## BATTERY INSTALLATION

### This unit comes complete with 2 AAA batteries.

It is necessary to replace the batteries when the Low Battery symbol appears on the display, Fig 1, or when the display does not turn on after the **START/STOP** button is pressed.

The battery compartment is located on the underside of this monitor.

1. The cover is located on the bottom of the monitor, under the **START/STOP** button. Carefully release the cover to open the battery compartment.
2. Insert or replace 2 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols, Fig 2. Always use new batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

It is recommended to remove the batteries if the unit will not be used for an extended period of time.

The 'Low Battery' symbol will appear in the upper left side of the display.

Fig. 1

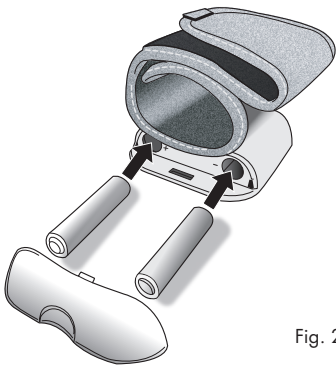
Fig. 2


## SETTING DATE/TIME

**IT IS NECESSARY TO SET THE DATE AND TIME FOR THE UNIT EVERY TIME BATTERIES ARE INITIALLY INSTALLED OR REPLACED.**

The date/time feature is on the bottom left of the display screen. This unit stores the date and time, including year.

1. When the batteries are first installed, the device will automatically prompt you to set the Date/Time feature as indicated by the flashing YEAR on the display.\*
2. Press the **MEM** button to advance the display to the desired year, press **SET** to confirm the year.
3. Next, the month will blink on the lower left of the display. Repeat step 2 to set the Month, Date, Hours, then Minutes.
4. After setting the minutes, the unit will automatically exit out of the Date/Time setting mode and briefly scroll through the Date/Time before shutting off.

\*NOTE: *If the batteries have already been installed*—when the unit is off (a blank display screen), press and hold the **SET** button. After 5-seconds, the year will begin to flash; follow the steps above to set the date and time.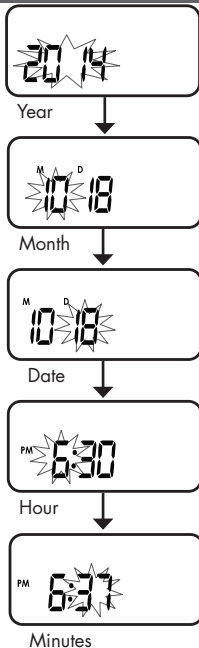

---

## **FITTING & APPLYING YOUR CUFF**

---

**PROPER CUFF FIT AND PLACEMENT IS CRITICAL IN OBTAINING ACCURATE BLOOD PRESSURE MEASUREMENTS. PLEASE READ THESE INSTRUCTIONS CAREFULLY AND CONSIDER CONSULTING WITH YOUR PHYSICIAN FOR FURTHER DIRECTION.**


Before applying your blood pressure cuff, be sure your wrist is within the appropriate cuff range: Wrist Circumference Range Suitable for 5.3" – 8.4" (13.5 – 21.5 cm).

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

If for any reason you are unable to or should not use your left wrist, please modify the instructions for cuff application to your right wrist. Your physician can help you identify which wrist is best for you to take measurements from.

### **Applying Your Cuff**

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Be seated at a table or desk with your feet flat on the floor.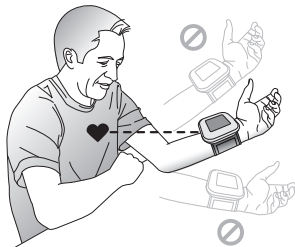

---

## **FITTING & APPLYING YOUR CUFF**

---

3. Hold your left arm in front of you with your palm facing upward.
4. Apply the preformed cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you, Fig. 1.
5. Adjust the cuff 1/4" - 1/2" from the edge of the head of the ulna bone, Fig. 2.
6. The cuff should fit comfortably, yet snugly around your wrist.

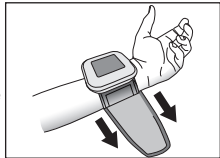
Fig. 1

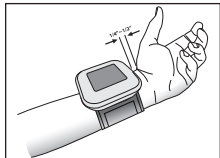
Fig. 2

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

## TAKING YOUR BLOOD PRESSURE READING

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to use.

**02** Select the desired user, **1** or **2**, before measurement. The user icon displayed represents the memory bank where the result will be stored. When the monitor is OFF, press and hold the **MEM** button for 4 seconds; the user icon will appear on the display. Press **MEM** to toggle and select the desired user. Press **SET** to confirm.

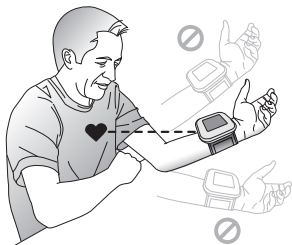
Fig. 1

1. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand.
2. Press the **START/STOP** button to turn the unit on; the unit will run a self test, Fig 2.
3. You will see the numbers increase on the screen and the heart icon flashing during measurement as the cuff inflates.

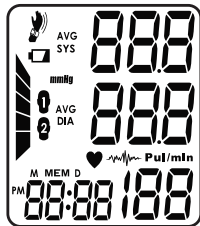
Fig. 2

**YOU CAN STOP THE INFLATION OR DEFLATION PROCESS AT ANY TIME BY PRESSING THE START/STOP BUTTON.**

## TAKING YOUR BLOOD PRESSURE READING

- When the measurement is complete, your blood pressure measurement and pulse reading results will show on the display screen at the same time, Fig 3.
- The Hypertension Indicator will indicate your reading range on the display, Fig 4. See information regarding interpreting the Hypertension Indicator results.
- If an irregular heartbeat was detected during the reading, the Irregular Heartbeat Detector indicator  will appear on the display, Fig 4.  
See more information on the Irregular Heartbeat Detector feature.
- The reading will automatically be stored in memory, up to 60 readings.
- Select **START/STOP** to turn the unit off and conserve energy and battery life. The unit will automatically shut-off.

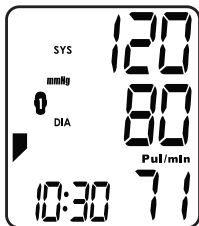
Fig. 3

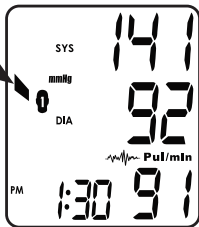
Fig. 4

### Excessive Movement

This monitor detects excessive movement during measurement. This icon appears when the monitor and wrist have moved or shaken during the reading. Though the screen will still register and display the measurement results, the results may be compromised and it is recommended that a new measurement be taken.

---



## INTERPRETING YOUR RESULTS

---

### Irregular Heartbeat Detector

An irregular heartbeat is defined as a heartbeat rhythm that is more than 25% slower or 25% faster than the average measurement detected while the monitor is conducting the measurement. When this device detects an irregular rhythm two or more times during the reading, the IHB symbol will appear on the display screen.

This monitor will operate if the icon appears, but the results may be compromised, especially if this symbol appears often. Please consult with your physician or trained healthcare professional for further information regarding an irregular heartbeat and if this symbol appears frequently.

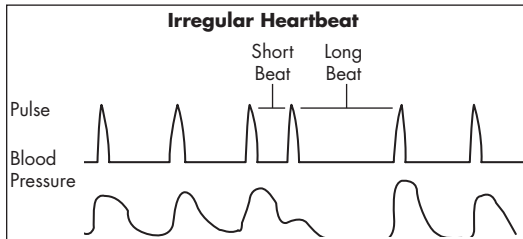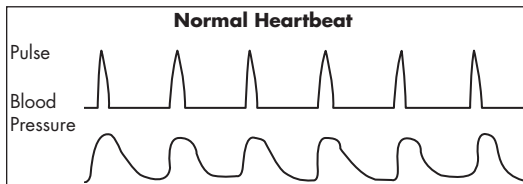
# INTERPRETING YOUR RESULTS

## Hypertension Indicator

This unit features our unique Hypertension Indicator. The color bars on the left side of the monitor display correspond with an icon on the digital display, indicating where the measurement results fall within the American Heart Association standards.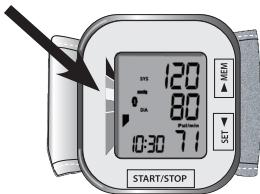

The American Heart Association has established globally accepted standards for the assessment of high or low blood pressure readings. The below chart should be considered only as a guideline, always consult with your physician or health care professional to interpret your individual results. Never adjust your medication(s) dosage, or severely alter your diet or exercise routine without consulting with your physician.

Bar Color	Category	Systolic (mmHg)		Diastolic (mmHg)
Green	Normal	<120	and	<80
Green	Prehypertension / Elevated	120-129	and	<80
Yellow	High Blood Pressure, Stage 1 Hypertension	130-139	or	80-89
Orange	High Blood Pressure, Stage 2 Hypertension	≥140	or	≥90
Red	Hypertensive Crisis	>180	and/or	>120

# MEMORY FUNCTIONS

## Recalling Measurements in Memory

**1 2** Select the desired user, **1** or **2**, to review. The user icon displayed represents the memory bank where the result will be stored. When the monitor is OFF, press and hold the **MEM** button for 4 seconds; the user icon will appear on the display. Press **MEM** to toggle and select the desired user. Press **SET** to confirm.

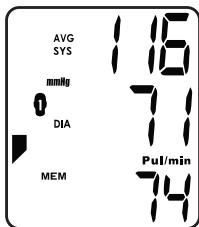
Fig. 1

This monitor stores and recalls up to 60 readings in per user. These measurements can be shared with your physician or trained healthcare professional.

1. Press and release the **MEM** button. The unit will first display the average of the last three measurements, Fig 1.
2. Press the **MEM** button again to begin scrolling through the saved measurements. Measurements will appear on the display from most current to oldest; the memory number will appear on the lower left corner.
3. All results for a given measurement will display, including measurement results, pulse rate, Hypertension Indicator and Irregular Heartbeat alert (if applicable). The date/time stamp will cycle on the lower left—memory number, time, date, repeat, Fig 2.
4. When the number of readings exceeds 60 , the oldest data will be replaced with the new record.

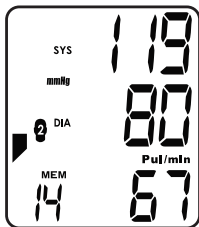
Fig. 2

---

## MEMORY FUNCTIONS

---

5. Press **START/STOP** to turn the monitor off at any time during review of the stored measurements.

### Clearing Measurements From Memory

To delete all measurements from either memory bank, follow these steps:

1. When the monitor is off, press and release the **MEM** button to access the memory bank feature.
2. Press and hold the **MEM** button for 3–4 seconds; the display will flash 'dEL ALL', Fig 1.
3. Press **SET** to confirm; the display will briefly flash 'dEL dONE' and turn off, Fig 2.

If you do not wish to delete the measurement records, press the **START/STOP** button at any time to exit the 'Clearing Measurement' mode.

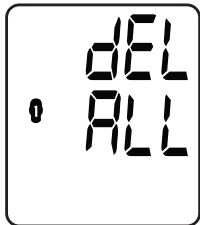
Fig. 1

Fig. 2

---

## CARE & MAINTENANCE

---

Proper care and maintenance of your blood pressure monitor will help ensure long life for the product; improper use of the monitor may void the included warranty.

### **Monitor Care & Maintenance**

- Do not use any liquids on the monitor; use a soft, dry cloth to clean the monitor as needed.
- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -4°F – 140°F (-20°C – 60°C); Humidity: ≤93%RH (non-condensing).
- Remove the batteries if the unit will not be used for an extended period of time.
- Do not attempt to disassemble your monitor; disassembling your monitor may void the manufacturers warranty.
- Do not subject the monitor to strong shocks; take care not to drop the monitor.
- No component of this device can be maintained by the user; the circuit diagrams, component part lists, descriptions, calibration instruction or other information which will assist the user's appropriately qualified technical personnel to repair those parts of equipment which are designated repairable can be supplied.

---

## **CARE & MAINTENANCE**

---

### **Cuff Care & Maintenance**

- The cuff should not be submerged or washed in a washing machine, but may be spot cleaned with a mild-detergent as needed.
- Never attempt to iron or dry the cuff; if the cuff becomes wet, allow the cuff to air dry completely before use or storage.
- To disinfect the cuff, wipe the inside (skin contact side) of the cuff using a cloth lightly moistened with 70% - 90% ethyl rubbing alcohol and allow to air dry; it is suggested to disinfect the cuff frequently when used in clinical sittings or if the cuff is contaminated in any way.

## DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Meaning
	Read This Manual—All included manuals should be read prior to first use.
	Warning—Symbol indicates a warning, prohibition or mandatory action that mitigates a risk that is not necessarily obvious to the device operator.
	Type BF Applied Parts—Indicates that a part of this unit comes in contact with the patient in order to carry out its intended function.
	Environment Protection—dispose of this product properly; consult with your local recycling ordinances for proper recycling and disposal.
	Keep Dry—This device should be kept dry; never submerge the unit or cuff. Consult with the Care and Maintenance section of this manual for information on cleaning your monitor.
	Serial Number.
	Direct Current.
	Manufacture Date.

Read This Manual—All included manuals should be read prior to first use.

Warning—Symbol indicates a warning, prohibition or mandatory action that mitigates a risk that is not necessarily obvious to the device operator.

Type BF Applied Parts—Indicates that a part of this unit comes in contact with the patient in order to carry out its intended function.

Environment Protection—dispose of this product properly; consult with your local recycling ordinances for proper recycling and disposal.

Keep Dry—This device should be kept dry; never submerge the unit or cuff. Consult with the Care and Maintenance section of this manual for information on cleaning your monitor.

Serial Number.

Direct Current.

Manufacture Date.

## DESCRIPTION OF DISPLAY SYMBOLS

Symbol	Meaning
<b>SYS</b>	Systolic Blood Pressure—The 'top' blood pressure result
<b>DIA</b>	Diastolic Blood Pressure—The 'bottom' blood pressure result
<b>PULSE/MIN</b>	Pulse Rate—Pulse rate per minute during the measurement
<b>mmHg</b>	Milligrams of Mercury—Unit of blood pressure measurement for this device
 + Lo	Low Battery—Battery life indicator
	Pulse Reading—Appears when pulse reading is displayed.
	Irregular Heartbeat Symbol—An irregular heartbeat was detected during the measurement
	User 1/User 2—Indicates which user is active
<b>AVG</b>	Average—The display shows the average of the last three measurements in memory
<b>MEM</b>	Memory—The display shows the measurements stored in memory

## TROUBLE SHOOTING

<b>Problem</b>	<b>Possible Cause</b>	<b>Solution</b>
Display shows an abnormal result.	Cuff position was not correct or was not tight enough.	Apply the cuff properly and attempt a new measurement.
	Body posture was not correct during measurement.	Review the 'Taking a Measurement' portion of this manual and attempt a new measurement.
	Speaking, excessive movement, angry, excited or nervous anxiety during measurement.	Wait a period of time and attempt a new measurement after calm has returned; do not speak or move during measurement.
Display is blank when power is on.	Batteries may be expired or installed improperly.	Check the polarity of the batteries and reinstall if necessary; replace batteries with new batteries.
No response when STOP/START is pressed or when new batteries have been loaded.	Incorrect operation; possible strong electromagnetic interference.	Remove batteries and wait a minimum of five minutes; reinstall batteries and attempt a new measurement.

## **ERROR CODES**

<b>Code</b>	<b>Meaning</b>	<b>Corrective Action</b>
E 01	The cuff is not secure/nottight enough.	Re-apply the cuff and attempt new measurement.
E 2	Device detected motion while measuring.	Remain still and attempt new measurement.
E 3	Measurement process does not detect the pulse.	Loosen any clothing on the arm; attempt a new measurement.
E4	Treatment of the measurement failed	Relax for several minutes and attempt new measurement.
Eexx	A calibration error occurred; xx may also be a digital symbol, such as 01, 02 etc. Any of these scenarios are a calibration error.	Retake the measurement. If the problem persists, contact customer service for further direction or consult the warranty in this manual for instructions on return for repair.
out	Out of measurement range.	Result is out of range; relax for a moment and attempt new measurement.

**IF AN ERROR CODE PERSISTS OR IS NOT REMEDIED,  
PLEASE CONTACT CUSTOMER CARE AT 866-326-1313.**

---

## ***FCC STATEMENT***

---

### **NOTE:**

#### POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

# ELECTROMAGNETIC COMPATIBILITY

Table 1

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments

**Warning:** Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

**Warning:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

**Warning:** Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation."

**Warning:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

## Technical description:

1. All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life.
2. Guidance and manufacturer's declaration -electromagnetic emissions and Immunity.

<b>Guidance and manufacture's declaration - electromagnetic emissions</b>	
<b>Emissions Test</b>	<b>Compliance</b>
RF emissions CISPR11	Group 1
RF emissions CISPR11	Class B
Harmonic distortion IEC 61000-3-2	Class A
Voltage fluctuations / flicker emissions IEC 61000-3-3	Comply

# ELECTROMAGNETIC COMPATIBILITY

Table 2

## Guidance and manufacture's declaration - electromagnetic immunity

Immunity Test	IEC 60601-1-2 test level	Compliance level
Electrostatic Discharge IEC 61000-4-2	$\pm 8$ kV contact $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air	$\pm 8$ kV contact $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air
Electrical fast transient/burst IEC 61000-4-4	$\pm 2$ kV for power supply lines $\pm 1$ kV signal input/output 100 kHz repetition frequency	$\pm 2$ kV for power supply lines $\pm 1$ kV signal input/output 100 kHz repetition frequency
Surge IEC 61000-4-5	$\pm 0.5$ kV, $\pm 1$ kV differential mode $\pm 0.5$ kV, $\pm 1$ kV, $\pm 2$ kV common mode	$\pm 0.5$ kV, $\pm 1$ kV differential mode $\pm 0.5$ kV, $\pm 1$ kV, $\pm 2$ kV common mode
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle	0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC 61000-4-6	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz

NOTE UT is the a.c. mains voltage prior to application of the test level.

# ELECTROMAGNETIC COMPATIBILITY

Table 3

## Guidance and manufacture's declaration - electromagnetic immunity

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band a (MHz)	Service	Modulation	Maximum Power (w)	Distance (m)	IEC 60601-1-2 Test Level (V/m)	Compliance level (V/m)
	385	380-390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27	27
	450	380-390	GMRS 460, FRS 460	FM $\pm$ 5kHz deviation 1kHz sine	2	0.3	28	28
	710 745 780	704-787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9	9
	810 870 930	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18Hz	2	0.3	28	28
	1720 1845 1970	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4,25; UMTS	Pulse modulation 217Hz	2	0.3	28	28
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217Hz	2	0.3	28	28
	5240 5240 5785	5100-5800	WLAN 802.11 a/n	Pulse modulation 217Hz	0.2	0.3	9	9

---

## PRODUCT SPECIFICATIONS

---

<b>Name</b>	Blood Pressure Monitor with Adult Wrist Cuff
<b>Model Number</b>	01-508, 01-508EM, 01-508RB, 01-508SP
<b>Display System</b>	Digital display/LCD
<b>Measuring Method</b>	Oscillometric Method, Automatic Air Inflation and Measurement
<b>Power Source</b>	2 x 1.5V  AAA Batteries
<b>Measuring Range</b>	Rated cuff pressure: 0mmHg~299mmHg(0kPa ~ 39.9kPa) Measurement pressure: SYS: 60mmHg~230mmHg (8.0kPa~30.7kPa) DIA: 40mmHg~130mmHg (5.3kPa~17.3kPa) Pulse value: (40-199) beat/minute
<b>Accuracy</b>	Pressure: 41°F – 104°F (5°C – 40°C) within $\pm 3$ mmHg; Pulse: $\pm 5\%$ of reading
<b>Pressurization</b>	Automatic pressurization by pump
<b>Deflation</b>	Automatic pressure release valve
<b>Memory</b>	Built-in memory with 2 banks, 60 measurements per bank
<b>Automatic Power-Off</b>	Approximately 1 minute after last button operation
<b>Battery Life</b>	Approximately 1 month with normal usage
<b>Operation Environment</b>	Temperature 41°F – 104°F (5°C – 40°C); Humidity 15% to 90% RH (non-condensing, but not requiring water vapor) partial pressure greater than 50hPa; Atmospheric pressure range of 700 hPa to 1060 hPa

---

## PRODUCT SPECIFICATIONS

---

<b>Storage Environment</b>	Temperature -4°F – 140°F (-20°C – 60°C); Relative humidity range of $\leq$ 93% RH (non-condensing at water vapor pressure up to 50hPa)
<b>Monitor Dimensions</b>	2-5/8" x 2-3/7" x 7/8"
<b>Monitor Weight</b>	3.5 oz. (without batteries)
<b>Wrist Circumference Range</b>	Suitable for 5.3" – 8.4" (13.5 – 21.5 cm)
<b>Accessories</b>	Instruction manual, two AAA batteries, storage case Quick Start Guide, Blood Pressure Log
<b>Degree of protection</b>	Type BF applied part
<b>Protection against ingress of water</b>	IP22
<b>Software version</b>	A01

*Specifications are subject to change without notice*

---

## 3-YEAR LIMITED WARRANTY

---

Congratulations on your purchase of a Digital Blood Pressure Monitor. Your Digital Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for a period of three years for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required

---

## **3-YEAR LIMITED WARRANTY**

---

prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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