
# Blood Pressure Monitor

Model # 01-509, 01-509-HSN, 01-509BL-HSN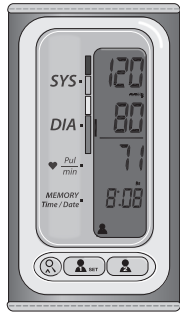

**THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.**



Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:

Adult Cuff fits arm circumference:  
8.6" to 16.5" inches (about 22 – 42 cm)

A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.

## Applying the Arm Cuff

Remove any constrictive clothing or jewelry that may interfere with cuff placement. Wrap the cuff snugly around your left arm. Position the cuff's bottom edge one inch ABOVE the elbow joint. Apply the cuff to your left arm so that the digital display face is positioned on the inside area of your arm facing you, and the word 'ELBOW' is centered over the inside bend of your elbow.

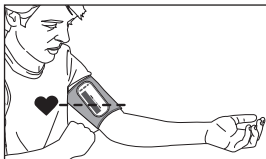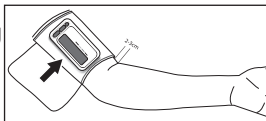## Taking the Measurement

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

The switch at the top of the device must be in the ON position to take your measurement; it will not take a reading if not in the ON position.

**Select the desired user, 1 or 2, before measurement; this selection will determine where the measurement is stored for review.**

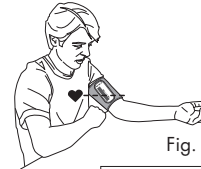
Fig. 1

- Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand.
- When the monitor is off, press the **U1/SET** or **U2 (POWER)** button to **POWER** on the monitor; the unit will run a self test,
- The display will briefly show a '0' before the measurement begins.
- You will see the numbers increase on the digital display during measurement as the cuff inflates.
- When the measurement is complete, the cuff will automatically deflate and your blood pressure measurement and pulse reading results will show on the display screen at the same time, Fig 3.
- Consult the included Instruction Manual for information on interpreting your reading results.

Fig. 2

Fig. 3

- The reading will automatically be stored in the USER Memory Bank selected at the start of the reading, up to 250 readings per user.
- Select **U1/SET** or **U2** button to turn the unit off and conserve energy and battery life.
- The unit will automatically shut-off approximately 1 minute after use.

## Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the **U1/U2** button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

**IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.**

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