# Blood Pressure Monitor

with Adult Wrist Cuff

Model # 01-508 Series



**THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.**

Before applying your blood pressure cuff, be sure your wrist is within the appropriate range: wrist circumference 5.3" – 8.4" (13.5 – 21.5 cm).

A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.

## Applying the Wrist Cuff

1. Be seated at a table or desk with your feet flat on the floor. Hold your left arm in front of you with your palm facing upward.
2. Apply the preformed cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you.
3. Adjust the cuff 1/4" - 1/2" from the edge of the head of the ulna bone. The cuff should fit comfortably, yet snugly around your wrist.

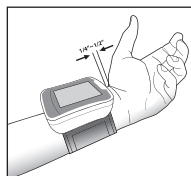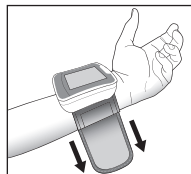## Taking the Measurement

**02** Select the desired user, 1 or 2, before measurement. The user icon displayed represents the memory bank where the result will be stored. When the monitor is OFF, press and hold the **MEM** button for 4 seconds; the user icon will appear on the display. Press **MEM** to toggle and select the desired user. Press **SET** to confirm.

Fig. 1

1. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand.
2. Press the **START/STOP** button to turn the unit on; the unit will run a self test, Fig 2.
3. You will see the numbers increase on the screen and the heart icon flashing during measurement as the cuff inflates.
4. When the measurement is complete, your blood pressure measurement and pulse reading results will show on the display screen at the same time, Fig 3.
5. Consult the included Instruction Manual for information on interpreting your reading results
6. The reading will automatically be stored in memory, up to 60 readings.
7. Select **START/STOP** to turn the unit off and conserve energy and battery life. The unit will automatically shut-off.

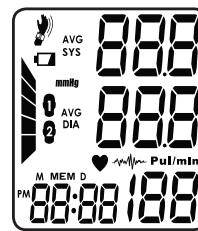
Fig. 2

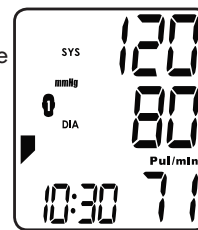
Fig. 3

## Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the **START/STOP** button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

**IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.**

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