
VH Complete Tabletop Nebulizer System

Model 11-503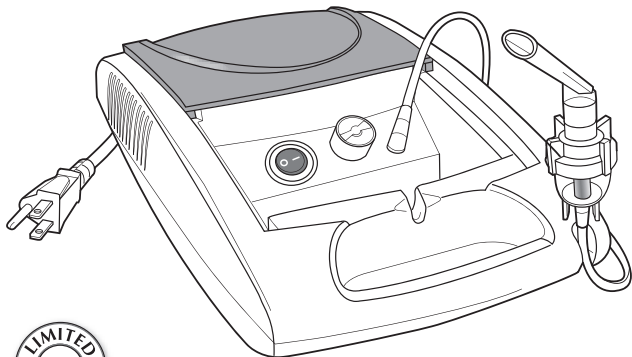

**READ THIS INSTRUCTION MANUAL
CAREFULLY BEFORE USE**

INSTRUCTION MANUAL • ENGLISH & ESPAÑOL

STOP!


PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING COMPONENTS BEFORE USING YOUR NEBULIZER

Compressor Body
with OPEN Storage
Compartment Lid

Angled
Mouthpiece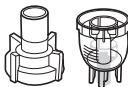


Nebulizer
Top and Cup

(5) Air Filters

Air Tubing

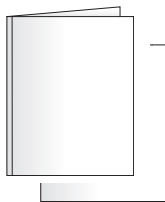Instruction Manual
FAQ Information Sheet
Replacement Parts
Order Form

IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.

TABLE OF CONTENTS

Introduction	4
Care and Safety Information	5
Product Features	6
Nebulizer Mask/Mouthpiece Selection	7
Using Your Nebulizer	8-11
Cleaning and Disinfection.....	12-13
Changing the Filter	14
Troubleshooting	15
Warnings and Cautions	16
Product Specifications	16
Warranty	17
Instrucciones en Español.....	19-35

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

Manufactured for:
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com

Made in China
#93-1066 10/15
©2015 Veridian Healthcare, LLC

INTRODUCTION

Thank you for purchasing this Veridian Compressor Nebulizer. Please read this instruction manual completely before using your unit to ensure safe and effective use, as well as long life of the product.

Compressor nebulizers are used to efficiently deliver medication in the form of a mist to the lungs. Prescribed medicine is placed in the nebulizer and the compressor blasts air through the liquid medication, converting it to an aerosol mist which is then inhaled. This method of delivery is beneficial in the treatment of asthma and other respiratory ailments, as the medicine is administered directly to the bronchial passages of the lungs to provide quick and effective relief. With proper use and care, your new nebulizer will provide you with many years of safe, effective breathing therapy treatments.

Federal law allows nebulizers to be sold by a physician's prescription only. These devices should be used only at the direction of a doctor or healthcare professional and should be used only as intended and described in this manual.

CARE AND SAFETY INFORMATION

Improper care or use of your nebulizer may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the nebulizer's efficacy and long life.

- Never allow the nebulizer to be used without adult supervision.
- Do not use the nebulizer for anything except medical nebulization treatment as directed by a physician. Do not use the nebulizer in a manner inconsistent with these instructions.
- Never use any liquid in the nebulizer other than those prescribed by your doctor. Liquids such as cough medicines or essential oils can harm both the machine and the patient.
- The nebulizer is an electronic device, keep away from water to avoid electrical shock. Do not immerse the unit in liquid and do not use while bathing. If the unit should fall into water, do not reach for it without first unplugging the unit.
- Unplug the unit before cleaning it, when filling the nebulizer, and after each use. Do not leave the unit plugged in when not in use.
- Use only the included AC power cord.
- Always ensure that the air vents are not blocked or obstructed in any way.
- Do not use the nebulizer in a very dusty area.
- Dropping the nebulizer or causing other heavy impact may damage it.
- Do not expose the unit to direct sunlight, humid environments, extreme temperatures, strong static electricity or electromagnetic waves.
- If the nebulizer is not functioning correctly, if it makes unusual sounds, or if during use you feel pain or discomfort, immediately stop use.
- Do not allow the nebulized medicine to come in contact with your eyes.
- The nebulizer and mouthpiece or mask should be replaced once per year.
- Do not use the unit if it has been dropped to the ground, exposed to extreme temperatures or high humidity, or damaged in any way.
- Do not disassemble the nebulizer or attempt to repair it.

PRODUCT FEATURES

Storage Compartment
Retracting Cover

Air Tubing
Connector Port

Accessory Storage
Compartment



Air Vents

Nebulizer
Holder

Power
Switch

Air Filter
Cover

Power cord
storage area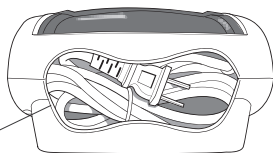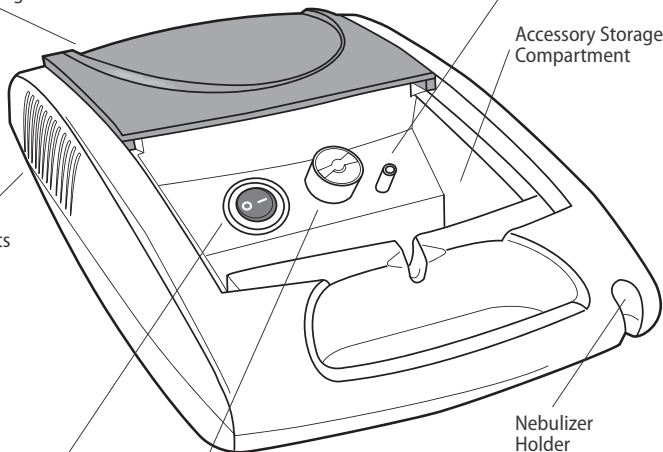

NEBULIZER MASK/MOUTHPIECE SELECTION

This unit includes a standard angled mouthpiece. Additional styles of mouthpieces and masks are available for order on the enclosed order form, by calling Customer Care at 866-326-1313 or at your local medical supply dealer.


Proper use of the mouthpiece is important in helping to ensure effective and efficient medication delivery.

Angled mouthpiece (included): put the mouthpiece into the mouth, holding gently between the teeth and close the lips around the mouthpiece; hold the nebulizer in place with a free hand.

Face mask (not included): select the appropriate size for the patient - child or adult - and place over the patient's nose and mouth. Secure in place with the elastic strap around the patient's head and hold the nebulizer in place with a free hand.

Angled mouthpiece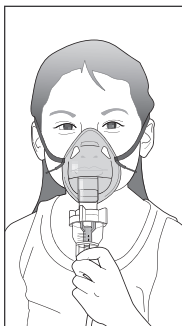

Face mask

Tee adapter mouthpiece (not included): put the mouthpiece end into the mouth, holding gently between the teeth and close the lips around the mouthpiece; hold the nebulizer in place with a free hand.

Pacifier mouthpiece (not included): put the pacifier nipple into the baby's mouth; an adult should hold the nebulizer, being certain the medicine cup remains upright.

Tee adapter mouthpiece

Pacifier mouthpiece

USING YOUR NEBULIZER

Before the first use of the nebulizer or if the unit has not been used for an extended period of time, it should be cleaned thoroughly as described in the **Cleaning and Disinfection** section of this manual.

The patient receiving treatment should be seated or slightly reclined in a comfortable position as treatments take several minutes. Keep in mind that the nebulizer cup must remain vertical during treatment.

1. Place the compressor unit on a flat, stable surface within operating range of the patient and an electrical outlet. Ensure that the air vents on the side of the unit housing are not blocked or obstructed.
2. Plug the power cord into the electrical outlet.
3. Slide the cover back to expose the Power switch and hose connector, *Fig. 1*.
4. Attach one end of the air tube to the tube connector inside the accessory compartment, *Fig. 2*. Turn on the power briefly to ensure that the unit is functioning properly and test that air is coming through the tubing, then turn the power off.

Fig. 1

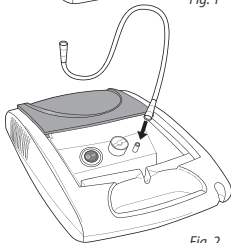
Fig. 2

USING YOUR NEBULIZER





5. Fill and assemble the nebulizer:

Step 1: Pour the medication into the nebulizer cup following the physician prescribed dosage, *Fig 1*.

Step 2: Replace the top of the nebulizer and twist to close; gently pull the top to ensure the top is secure, *Fig 2*.

6. Attach the selected mouthpiece to the top of the nebulizer, taking care not to spill the medication in the cup, *Fig 3*.

7. Attach the nebulizer to the air tubing, *Fig 4*.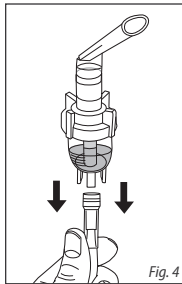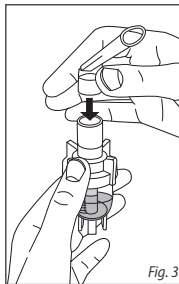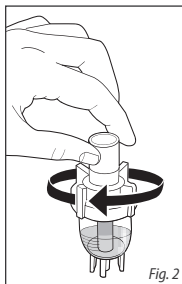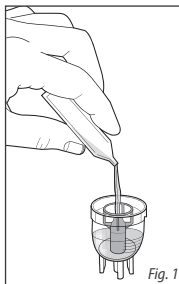


Treatment time varies depending on the type of medication and volume of medication being used. It is important to complete the entire prescribed dose of medication for maximum effectiveness. Consult with your prescribing physician or healthcare professional for information on your specific medication therapy treatment plan.


USING YOUR NEBULIZER

- Position the mouthpiece at the patient's mouth.

Remember that masks should cover the patient's mouth and nose, mouthpieces should be in the mouth with the lips closed around the end of the mouthpiece.
- Turn the Power switch on to begin treatment. Inhale deeply and exhale normally. Ensure the nebulizer is vertical during treatment for proper nebulization of the medication and to prevent spilling of the liquid.
- If the vapor output becomes sporadic or if the noise the unit makes changes during use, turn off the unit and consult the TROUBLESHOOTING section of this instruction manual.
- When the treatment is complete, turn the unit off and unplug it from the outlet.
- Disassemble, clean and store your nebulizer. Be certain that the nebulizer pieces are completely dry before storage.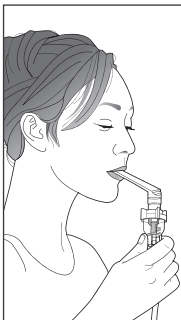

Mouthpiece position

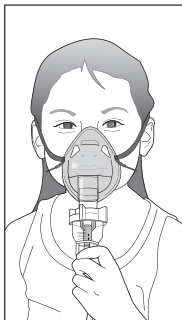Mask position

NOTE: If the vapor begins to slow and there is still medication in the cup, gently tap the side of the cup to settle the liquid at the bottom of the cup for nebulization.

USING YOUR NEBULIZER

Compressor nebulizers contain a powerful compressor motor which may heat up the unit if it is left running too long or if the vents are blocked while operating. The unit will automatically shut off if the temperature of the unit becomes too high.

If this occurs, follow these steps:

1. Turn the power off and disconnect the power cord.
2. Wait at least 30 minutes until the unit has cooled down completely.
3. Turn the unit back on.
4. If the unit heats up too quickly or emits any unusual noises beyond the normal sound of the compressor motor, contact Customer Care at 866-326-1313.

CLEANING AND DISINFECTION



The nebulizer and mouthpiece should be thoroughly cleaned after each use and disinfected each day if on a regular therapy program unless a different cleaning regimen is specified by your healthcare professional; additional hygienic precautions may be required for use with high-risk patients. Always follow the direction of your healthcare provider.

Cleaning:

1. Ensure that the power cord is disconnected from the power outlet.
2. Disconnect the air tube, nebulizer and mouthpiece. Do not wash or submerge the air tubing.
3. Disassemble the nebulizer by twisting the top and bottom and separating the pieces, *Fig 1-2*.
4. Wash the nebulizer and mouthpiece or mask in warm water, using a mild detergent. Do not wash or submerge the air tubing; if necessary, wipe the outside of the tubing with a damp cloth taking care not to get water inside the tubing.
5. Allow the nebulizer and mouthpiece to air dry completely, or dry them thoroughly with a clean, soft towel. Do not store any parts wet as remaining moisture creates a risk of bacterial growth.
6. Occasionally wipe the body of the unit with a damp cloth and allow it to air dry. Only clean the body of the unit when it is unplugged and completely cool.

Never use soap pads or powder cleanser to clean any part of the unit. Never use any cleaning products that may be harmful when swallowed or inhaled.

It is recommended that nebulizers and tubing be replaced periodically depending on the frequency of therapy treatments or when damaged, cracked or holes appear in the tubing. To order replacement parts, use the included order form, call Customer Care at 866-326-1313 or contact your local dealer.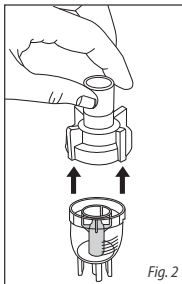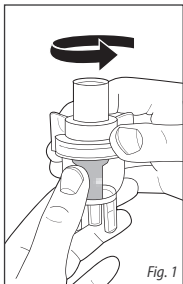

CLEANING AND DISINFECTION

Disinfection:

The nebulizer should be disinfected once daily, after the last use of the day, unless a different disinfection regimen is specified by a healthcare professional. Disinfect the nebulizer as follows:

1. Ensure that the power cord is disconnected from the power outlet.
2. Disconnect the air tube, nebulizer and mouthpiece. Do not wash or submerge the air tubing.
3. Disassemble the nebulizer by twisting the top and bottom and separating the pieces.
4. Rinse the nebulizer and mouthpiece or mask with hot water. Mix one part white vinegar with three parts distilled water, in a quantity sufficient to fully immerse the nebulizer and mouthpiece or mask.
5. Submerge the nebulizer and mouthpiece or mask in the water and vinegar mixture for thirty minutes.
6. Rinse the nebulizer and mouthpiece or mask thoroughly with water.
7. Allow the nebulizer and mouthpiece to air dry completely, or dry them thoroughly with a clean, soft towel. Do not store any parts wet as remaining moisture creates a risk of bacterial growth.

NOTE: Condensation may collect in the air tubing. If this occurs, the moisture must be completely removed, following these steps:


1. Disconnect the air tubing from the nebulizer, but keep it plugged into the air tubing connector port on the unit.
2. Turn the power on and allow the compressor to run until all moisture has dried.

CHANGING THE FILTER

- Never operate the unit without a filter.
- Replace the filter after every ten uses, or sooner if it turns gray, appears contaminated or is blocked or becomes wet.
- Use only Veridian Healthcare filters; additional filters are available for order using the included order form, calling Customer Care at 866-326-1313 or contacting your local dealer.
- Do not substitute cotton or any other materials for the filters.
- Do not attempt to clean the filters or reuse a filter.

To replace the filter, follow these steps:

1. Change the filter only when the unit is off, unplugged and completely cool.
2. Gently pull the filter cap out of the compressor.
3. Remove the filter from the filter cap.
4. Place a new filter in the filter cap.
5. Return the filter cap to its place, ensuring that it is firmly in place.

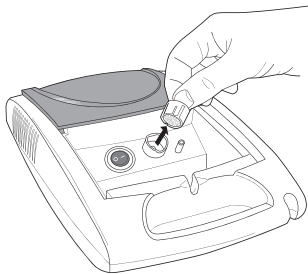TROUBLESHOOTING

The nebulizer produces no aerosol, or a small amount of aerosol.

- There may be too much or too little medicine in the nebulizer. The amount of liquid should be between 0.5 ml and 5.0 ml.
- The hose connectors may be blocked and need to be cleaned
- The nebulizer may not be held upright. Always hold the nebulizer vertically during use.
- The medication may not be suitable for nebulization. Check with your doctor.
- The filter may be in need of replacement
- The air hose may be kinked

Some medicine remains in the nebulizer at the end of my treatment.

This is normal. Once you hear a significant difference in the sound of the nebulizer, you may stop inhaling.

What medications can be used in the nebulizer?

Only medications prescribed by your doctor should be used in the nebulizer.

Should any special steps be taken for pediatric users?

Children should never use the nebulizer without adult supervision. For use with babies and children, the mask should cover the mouth and nose completely to ensure effective inhalation.

Treatments with the mask takes longer than treatments with the mouthpiece.

This is normal. Less medication is delivered per breath with the mask because the aerosol is combined with ambient air through the holes in the mask.

Can I share the nebulizer with another person?

No. Each user must have their own nebulizer to ensure proper hygiene. The compressor unit may be shared but the nebulizer, mouthpiece, and mask pieces may not.

Why must the nebulizer be replaced yearly?

Regular wear on the nebulizer can affect the composition and delivery of the medicine. Regular replacement of the nebulizer is also hygienic.

WARNING AND CAUTIONS

Nebulizer therapy is intended for use with physician prescribed medication only.

Over the counter medication, essential/aromatherapy oils or alcohol are not suitable for inhaling using a nebulizer. Highly viscous substances may damage the unit and make it unsuitable for use. Never use nebulizer therapy for anything other than physician prescribed respiratory care therapy.

Use common sense electrical caution when operating the device. This nebulizer runs on AC power. Always turn off the power and unplug the device before cleaning or if the unit is not working properly.

PRODUCT SPECIFICATIONS

Name	VH Complete Nebulizer
Model Number	11-503
Medication Capacity	Maximum 5 ml
Particle Size	0.5 to 10 μ m
Sound Pressure	Maximum 60 dBA at 1 meter
Average Nebulization Rate	Approximately 0.2 ml / minute
Compressor Pressure Range	29 to 43 Psi
Operating Pressure Range	8 to 13 Psi
Liter Flow Range	5 ~ 8 lpm
Operating Conditions	Temperature: 50°F – 104°F (10°C – 40°C); Humidity: 10% – 95%
Storage Environment	Temperature: -4°F – 158°F (-20°C – 70°C); Humidity: 10% – 95%
Unit Dimensions (W x H x D)	12" x 6.5" x 7.25"
Unit Weight	4 lbs. 1 oz. (with power cord)
Power Connection	120 V ~, 60 Hz; 180 VA
Included Accessories	Angled mouthpiece, nebulizer, air tubing, five air filters

Specifications are subject to change without notice.

5-YEAR LIMITED WARRANTY

Veridian Healthcare guarantees that its VH Complete Compressor Nebulizer will be free from manufacturing defects under normal use for a period of five-years. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty does not cover batteries or other power sources that may be provided with or used with the nebulizer unit. This warranty is voided if the nebulizer unit is misused or abused in any manner.

If the nebulizer fails to operate within the warranty period while under the original purchaser owns the product, return it postage prepaid to: Veridian Healthcare, Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031; include your name, address, phone number and brief description of the problem with your nebulizer. Take care to carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. Veridian Healthcare will repair or replace the defective unit at our option. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations or how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only reasonability and obligation to repair and/or replace materials or components, or refund the purchase price. Veridian will not be responsible for any indirect, incidental, special consequential or punitive damages or other loss, including, but not limited to damage to or loss of other property or liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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