
SMARTHEART™

Pulse Oximeter Model #11-50K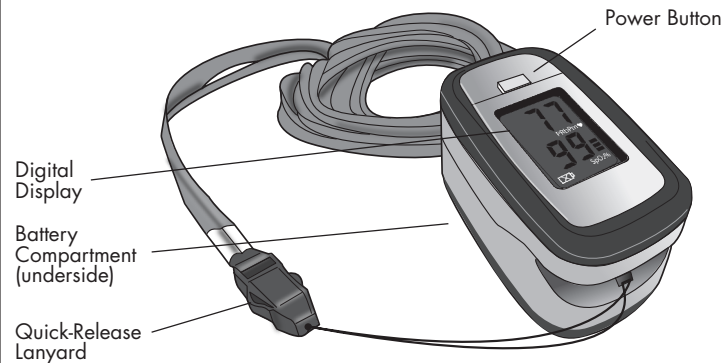

THANK YOU FOR PURCHASING A SMARTHEART PULSE OXIMETER. PLEASE READ THESE INSTRUCTIONS CAREFULLY TO ENSURE PROPER USE AND SAFE OPERATION OF YOUR PULSE OXIMETER.

INTRODUCTION

Please read all instructions carefully prior to first use.

Thank you for purchasing a SmartHeart Pulse Oximeter. With proper care and use, your SmartHeart Pulse Oximeter will provide you with many years of reliable service. The SmartHeart Pulse Oximeter measures user blood oxygen saturation (SpO2) and pulse rate information for sports and aviation activities in a small, portable, easy-to-use device.

The SmartHeart Pulse Oximeter is intended for use for persons engaging in sporting and aviation activities. Persons who need SpO2 and pulse rate measurements because of a medical condition should not use the SmartHeart Pulse Oximeter and should consult with their physician.

IMPORTANT SAFEGUARDS - CARE AND SAFETY INFORMATION

OPERATING CAUTIONS AND WARNINGS

- Keep unit and lanyard out of reach of children as the included lanyard may present an entanglement or choking hazard to small children. Adult supervision required; never leave children unattended with unit or lanyard.
- To avoid electrical shock keep the unit away from water; never immerse the unit in liquid as this will damage the unit
- Do not handle the unit with wet hands
- Do not use the unit if it has been submersed in water or dropped
- If any abnormality in product function occurs, discontinue use until the unit has been examined; discontinue use and return to the manufacturer for review and replacement under the terms of the two-year warranty
- Do not shake the unit when in operation
- Do not disassemble the unit or attempt to repair it

OPERATING CAUTIONS AND WARNINGS

- Never tie or tape SmartHeart Pulse Oximeter to your finger
- Explosion hazard: do not use the product in an explosive atmosphere

STORAGE CAUTIONS AND WARNINGS


- Never store the SmartHeart Pulse Oximeter the unit in direct sunlight, high temperature or humidity
- Keep the unit out of reach of small children

CLEANING CAUTIONS AND WARNINGS

- Never immerse the device in water to clean as it may be damaged
- Follow the included instructions for cleaning the unit after each use; if the unit has not been used for an extended period, the device should be cleaned prior to use.


HOW THE SMARTHEART PULSE OX WORKS

The Pulse Oximeter works by shining two light beams into the small vessels or capillaries of the finger to determine the user's the blood oxygen saturation percentage and pulse rate that is then displayed on the Pulse Oximeter's screen. The unit uses the phrase "SpO2" to identify the oxygen saturation level and displays it as a percentage of full capacity (100%). Pulse rate is identified by the phrase "PR bpm" and is displayed as beats per minute.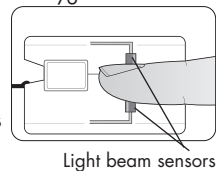

DIGITAL DISPLAY FEATURES

When in use, the Pulse Oximeter display reflects a 'live' reading of the wearers current SpO2 and PR bpm. The vertical display orientation can easily be read by the user or a caregiver that may be assisting the user.

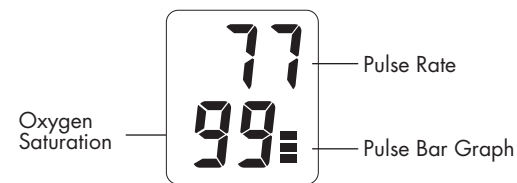USING YOUR PULSE OX

The Pulse Oximeter may be used while the user is seated, standing or lying down. The user should not be walking or running during measurements and should take care not to excessively move the arm or hand where the oximeter is attached.

PRIOR TO USE

Keep the sensors clean to help ensure accurate readings. Before and after each use, carefully swab inside the unit with a soft cloth or cotton swab with rubbing alcohol. Do not pour the alcohol directly on or into the unit.

Dirty hands or nail polish (especially dark shades) can also affect reading accuracy. Washing your hands and/or removing any nail polish prior to use to help ensure accurate readings.

Suitable for use on any finger (excluding the thumb). In most instances, the index or middle finger work best.

USING YOUR PULSE OX

1. Squeeze the hinged end of the unit to open the measurement side of the unit.
2. Insert your finger nail side up into the unit until resistance is felt. The unit should gently close below the first knuckle of the finger, and the sensors inside the unit should align with the finger nail and pad at the tip of the finger, Fig 1.
3. Press the power button to turn the unit on. Solid dashes (—) will appear on the display for a few seconds, Fig 2.
4. The display will begin to show the oxygen saturation (SpO2) and pulse rate (BPM) measurements, Fig 3.


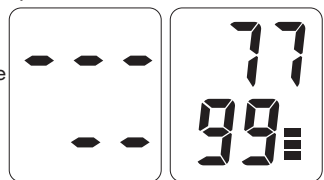
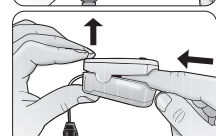
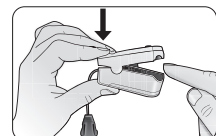
Fig. 1 Fig. 2 Fig. 3

Take care to not move excessively during monitoring. These are active readings and may fluctuate slightly as you observe the display, responding to your heart rate (bps) and other factors.

5. The Pulse Oximeter is intended for spot-checking blood oxygen saturation and pulse rate readings and should not remain on the finger for more than a few minutes at a time.
6. Remove the unit from the finger. The unit will automatically turn off after 5 seconds.
7. Clean the unit as directed before storage.

QUICK-RELEASE LANYARD

The lanyard features a quick-release clip for easy release while in use.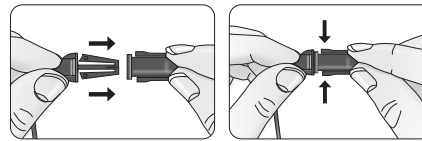


CHOKING HAZARD-The included lanyard may present an entanglement or choking hazard to small children. Adult supervision required; never leave children unattended with unit or lanyard.

BATTERY INSTALLATION AND REPLACEMENT

This unit comes complete with 2 AAA batteries.

It is necessary to replace the batteries when the Low Battery symbol is illuminated on the display or when the device does not turn on after the power button is pressed.


1. Slide off the battery cover.
2. Insert or replace 2 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols.
3. Replace the battery cover.
4. It is recommended to remove the batteries if the unit will not be used for an extended period of time.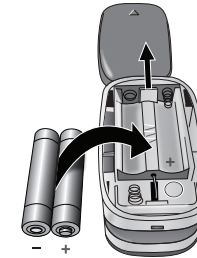

Always use new batteries. Do not mix alkaline, standard or rechargeable batteries. Please dispose of batteries according to local recycling regulations.

CARE AND MAINTENANCE

Before and after each use, carefully swab inside the unit with a soft cloth or cotton swab with rubbing alcohol. Do not pour the alcohol directly on or into the unit.

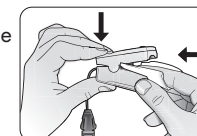The unit should be stored in a dry environment, within the ideal temperature range of -40°F-140°F (-40°C-60°C).

TROUBLESHOOTING

SpO2 or PR is not displaying a reading though the power is on

- The user's oxygen level is too low to be measured; consult your physician if the measurement cannot be read
- Reinsert the finger, remain still

SpO2 or PR display readout is unstable and fluctuating highly

- Finger may not inserted deep enough
- Finger may be trembling, try to remain still

The unit will not turn on

- Ensure the batteries have been properly installed
- Replace the batteries
- If the unit will not turn on after changing batteries, consult Customer Care

Display screen goes blank

- The unit will automatically shut-off after 5 seconds if no finger is inserted
- Check the batteries and replace if needed

PRODUCT SPECIFICATIONS

NAME	SmartHeart Pulse Oximeter
MODEL NUMBER	11-50K
DISPLAY SYSTEM	Digital display/LED
POWER SOURCE	2 "AAA" batteries
SPO2 MEASURING RANGE	0-100%
SPO2 ACCURACY	70-100%, ±2%
PULSE RATE MEASURING RANGE	30-250 bpm
PULSE RATE ACCURACY	30-250 bpm, ±2 bpm
AUTOMATIC SHUT-OFF	After 5 seconds of no indication on the sensors
OPERATION ENVIRONMENT	Temperature 50°F – 104°F (10°C – 40°C); Humidity ≤75%
STORAGE ENVIRONMENT	Temperature -40°F – 140°F (-40°C – 60°C); Humidity ≤95%
UNIT DIMENSIONS	2-1/4" x 1-1/8" x 1-1/4"
UNIT WEIGHT	2 oz. (with batteries); 1.2 oz (without batteries)

Specifications are subject to change without notice

ONE-YEAR LIMITED WARRANTY

Congratulations on your purchase of a SmartHeart Pulse Oximeter. Your SmartHeart Pulse Oximeter is covered by the following limited warranty commencing upon the date of purchase for a period of one years, and subject to the following terms and conditions:

Veridian Healthcare warrants that its SmartHeart Pulse Oximeter will be free from defects in materials and workmanship under normal consumer usage for as long as the original purchaser owns the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the SmartHeart Pulse Oximeter for commercial purposes or subjecting the SmartHeart Pulse Oximeter to abnormal usage or conditions; or (d) other acts which are not the fault of Veridian, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the SmartHeart Pulse Oximeter.

If the SmartHeart Pulse Oximeter fails to conform to this limited warranty, return the SmartHeart Pulse Oximeter to: Veridian Healthcare, Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, Veridian Healthcare will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL VERIDIAN HEALTHCARE BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

**Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

VERIDIAN HEALTHCARE

SmartHeart™
Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China
#93-1147 09/22
©2022 Veridian Healthcare, LLC

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