SMARTHEART™

# Automatic Digital Blood Pressure Wrist Monitor

Model # 01-518

**THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.**



Before applying your blood pressure cuff, be sure your wrist is within the appropriate range:

wrist circumference 5.3" – 8.5" (13.5 – 21.5 cm)

A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.

## Applying the Wrist Cuff




1. Be seated at a table or desk with your feet flat on the floor. Hold your left arm in front of you with your palm facing upward.
2. Apply the preformed cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you.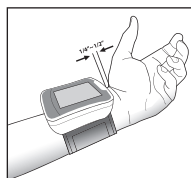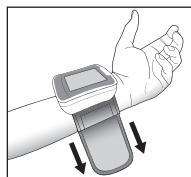

3. Adjust the cuff 1/4" - 1/2" from the edge of the head of the ulna bone. The cuff should fit comfortably, yet snugly around your wrist.

## Taking the Measurement

1. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand.
2. Firmly press the START/STOP button to turn the unit on. The display will briefly flash a test display.
3. The display will show the selected user in the upper left corner, user (1 or 2).
4. The cuff will automatically begin inflating; the user will feel the cuff tightening on the wrist.
5. The top numbers on the display screen will begin to go up; the heart icon will flash in-time with the pulse rate detected.
6. When the measurement is complete, your blood pressure and pulse reading results will show on the display screen at the same time.
7. Consult the included Instruction Manual for information on interpreting your reading results.
8. Select START/STOP button to turn the unit off and conserve energy and battery life. The unit will automatically shut-off after approximately 1 minute.

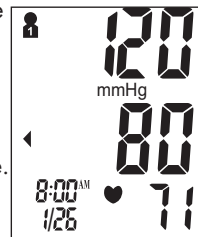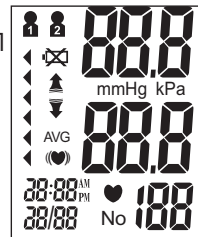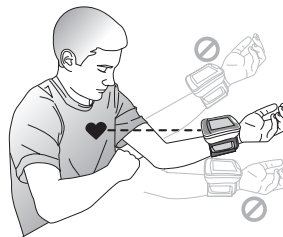## Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the START button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

**IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.**

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