CAUTION:
FOR SAFE AND SUCCESSFUL OPERATION OF
YOUR TENS+SMART CONTROL DEVICE, READ THE
INCLUDED INSTRUCTION MANUAL BEFORE USE.

To download the app, open the appropriate app store via your mobile device and search for:

TENS+EMS

NOTE: there is no 'space' between the words and + symbol.

Your Bluetooth must be ON in order for your mobile device to connect to the TENS unit. Your mobile device will automatically prompt you to accept the connection.


Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

#93-2132 09/17

TENS+EMS APP • QUICK START GUIDE

HOME SCREEN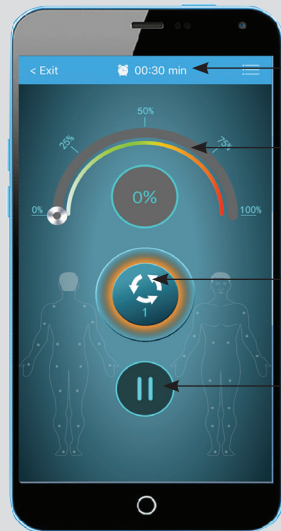

NOTE: If the Bluetooth is not connected, a box will appear at the bottom of the screen to prompt you to connect.


Treatment Timer
 Press to go to the Timer Setting screen

Treatment Intensity Slide
 The intensity will default to 0% at the start of treatment and whenever the program mode is changed

Treatment Program
 Press to go to the Program Mode Screen

Pause/Stop Treatment
 The timer will pause and remain paused until resumed or the App is closed; treatment will cease if the App is closed.
NOTE: If you leave the App, you will have to reconnect the device via Bluetooth before resuming treatment

TIMER SETTING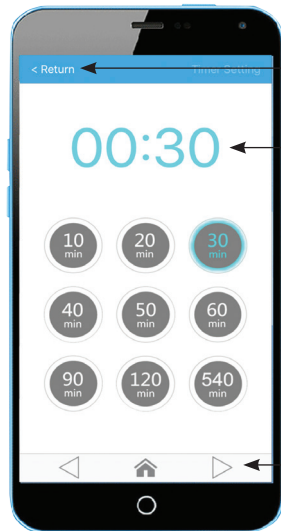

NOTE: To access the Timer Setting mode, select the menu bars at the upper right of the display for the complete menu.


Return
 Press to return to the main control screen

Selected Time
 The treatment will default to 30 minutes for each use

Timer Options
 Select the best time for your desired treatment; it is suggested to start at a lower setting and gradually increase as you become more comfortable with the treatment. The timer countdown will show on the top of the home screen

Home/Back/Forward
 Press to return to the home screen or to the other screen options

PROGRAM MODE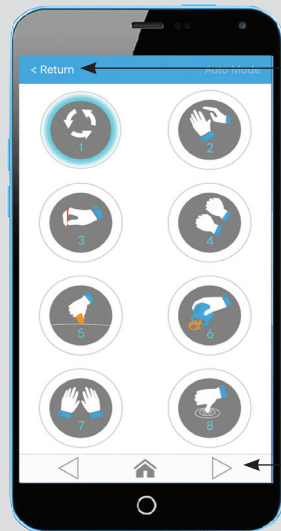

NOTE: Select the HOME button at the bottom of the display at any time to access the main screen.


Return
 Press to return to the main control screen

Program Modes
 Select the best program type for your desired treatment; detailed descriptions of each program type can be found on page 20 of the instruction manual

NOTE: When changing modes, the intensity as set on the HOME screen will default to 0% and must be adjusted to your desired level

Home/Back/Forward
 Press to return to the home screen or to the other screen options

MANUAL MODE

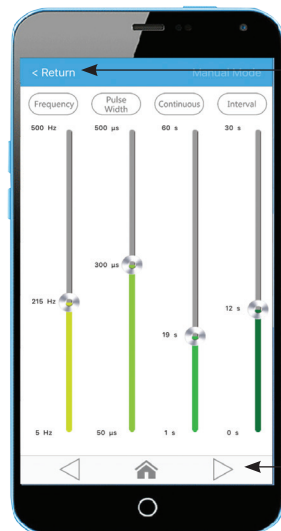NOTE: App includes extensive Warnings and Intended Use information for user direction and reference.

Return
 Press to return to the main control screen

Manual Mode
 You may manually adjust the following elements to customize the treatment:
 Frequency
 Pulse Width
 Continuous
 Interval

NOTE: When changing modes, the intensity as set on the HOME screen will default to 0% and must be adjusted to your desired level.

Home/Back/Forward
 Press to return to the home screen or to the other screen options

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