Analog Dial Weight Scale  
Model # 19-112

## **PLEASE READ BEFORE USE**

### **POINTER NEEDLE MAY SHIFT FROM ZERO (0) DURING SHIPPING AND TRANSPORT.**

When the scale is on a flat, even surface, use the dial at the base of the scale to align the pointer directly in line with Zero on the dial. Your scale is now ready for use.

## **Simple Step Instructions and Cautions**

**Correct Posture and Use**—Feet should be dry; do not wear shoes or socks during use. Take care to stand straight without excessive leaning forward or back; feet should be even on surface and not cover the dial.

**Scale Placement**— Scale should be on a solid, level surface. Never use on carpet. Do not place on wet surfaces.

**Calibration**—Pointer needle may shift during movement or transport. When the scale is on a flat, even surface, use the dial at the base of the scale to align the pointer directly in line with Zero on the dial.

**Maintenance**— Wipe with a damp cloth; a surface disinfectant may be used. Do not disassemble.

**Weight Capacity**— 286 lbs/130 kg; users outside of weight capacity will not receive an accurate reading and may damage scale.

SmartHeart™  
Manufactured for  
Veridian Healthcare  
1175 Lakeside Drive  
Gurnee, IL 60031

Made in China  
#93-1555 08/22  
©2022 Veridian Healthcare, LLC

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