Talking

Blood Pressure Arm Monitor

Model # 01-742T

THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:


Wide-Range Adult Cuff fits arm circumference: 8.6" – 16.5" (22–42 cm).

Additional cuffs are available for order on the included order form or by calling 866-326-1313.

A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.

Applying the Arm Cuff

Remove any constrictive clothing or jewelry that may interfere with cuff placement. Wrap the cuff snugly around your left arm. Position the cuff's bottom edge half an inch ABOVE the elbow joint. Position the air tubing over the main artery (on the inside of your arm). You should be able to insert one finger between your arm and cuff.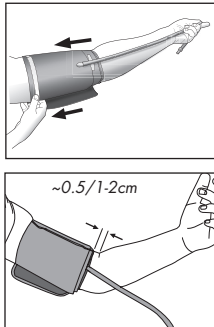

Taking Your Measurement

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

User will default to the last User Bank used; consult the manual for instructions on how to change the User to ensure accurate memory storage and review.

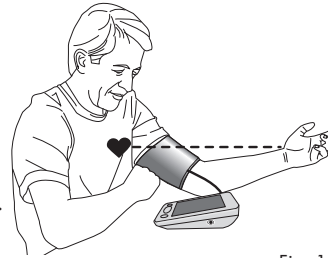
Fig. 1

1. Position the monitor on a flat, stable surface with the digital display in view. Be seated with your feet flat on the floor and legs uncrossed.
2. After applying the cuff insert the cuff tubing connector into the port on the left side of your monitor.
3. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Relax your left hand.
4. Press and release the **START/STOP** button to turn the unit on; the unit will briefly flash a self-test, Fig 2.
5. The display will briefly show a '0' and up arrow before the measurement begins, Fig 3.
6. As the inflation process begins, the display may briefly show a downward arrow indicating the cuff is releasing any air remaining in the cuff prior to measurement inflation.
7. The cuff will automatically inflate; you will see the

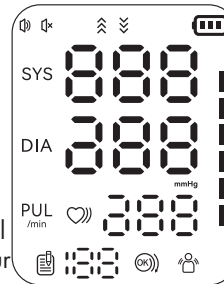
Fig. 2

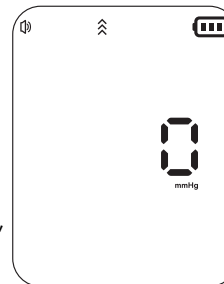
Fig. 3

numbers increase on the display screen and feel the cuff inflate.

8. When the measurement is complete, the cuff will automatically deflate, then your blood pressure measurement and pulse reading results will appear on the display screen at the same time, Fig 4.
9. Consult the included Instruction Manual for information on interpreting your reading results.
10. The reading will automatically be stored in memory, up to 199 readings.
11. Select **START/STOP** to turn the unit off and conserve energy and battery life. The unit will automatically shut-off approximately 1 minute after use.

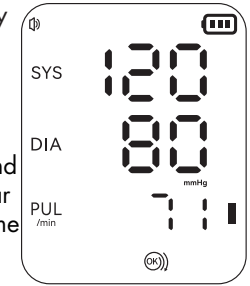
Fig. 4

Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the **START/STOP** button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.

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