

SmartHeart Talking Pulse Oximeter Simple Step Instructions.

Features of your pulse oximeter.

The talking feature of this pulse oximeter defaults to English and On. Pulse oximeters take a live reading of your blood oxygen saturation and pulse rate. Only the initial reading is automatic; to hear your results during use, press the power button for results at any time during use.

The battery compartment is on the underside of the oximeter.

There is an included lanyard inside of the storage bag.

Remove the printed protective mylar on the screen before use.

Cautions and warnings:

For best results, use on the index or middle finger. Thick nailbeds and painted nails may interfere with results.

Some studies have indicated that certain physical traits, including, but not limited to darker skin pigmentation, finger thickness, and peripheral perfusion may create variations in readings that make the oxygen level appear higher than it actually is. Consult with your healthcare professional for best guidance in interpreting your individual results.

Measuring your blood oxygen saturation:

First time use: Install two AA Type Batteries on the underside of the oximeter.

Squeeze the oximeter open at one end and insert the finger into the measurement side making sure the finger is fully inserted until resistance is felt. The nailbed should be directed up towards the top of unit and the pad of the finger down towards the battery / underside of the oximeter.

Press the power button to turn the unit on.

After several seconds, the first reading will be announced out loud.

Press the power button again to hear additional readings; remember the reading is live and may fluctuate slightly.

Remove the finger; the pulse oximeter will shut off after approximately 5 seconds.