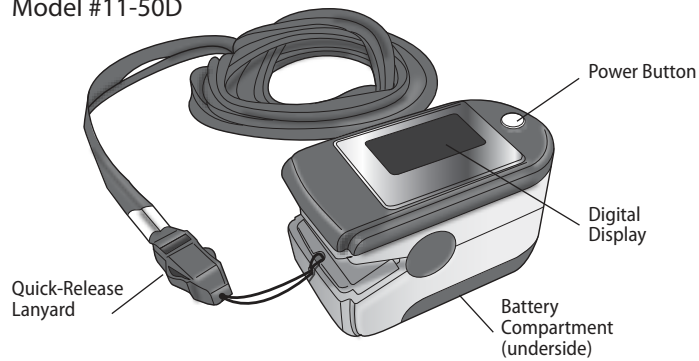
Deluxe Pulse Oximeter

Model #11-50D

THANK YOU FOR PURCHASING A PULSE OXIMETER. PLEASE READ THESE INSTRUCTIONS CAREFULLY TO ENSURE PROPER USE AND SAFE OPERATION OF YOUR PULSE OXIMETER.

INTRODUCTION

Please read all instructions carefully prior to first use.

Thank you for purchasing a Pulse Oximeter. With proper care and use, your Pulse Oximeter will provide you with many years of reliable service. The Pulse Oximeter measures user blood oxygen saturation (SpO2) and pulse rate information for sports and aviation activities in a small, portable, easy-to-use device.

The Pulse Oximeter is intended for use for persons engaging in sporting and aviation activities. Persons who need SpO2 and pulse rate measurements because of a medical condition should not use the Pulse Oximeter and should consult with their physician.

IMPORTANT SAFEGUARDS - CARE AND SAFETY INFORMATION

OPERATING CAUTIONS AND WARNINGS

- Keep unit and lanyard out of reach of children as the included lanyard may present an entanglement or choking hazard to small children. Adult supervision required; never leave children unattended with unit or lanyard.
- To avoid electrical shock keep the unit away from water; never immerse the unit in liquid as this will damage the unit
- Do not handle the unit with wet hands
- Do not use the unit if it has been submersed in water or dropped
- If any abnormality in product function occurs, discontinue use until the unit has been examined; discontinue use and return to the manufacturer for review and replacement under the terms of the warranty
- Do not shake the unit when in operation
- Do not disassemble the unit or attempt to repair it

OPERATING CAUTIONS AND WARNINGS

- Never tie or tape Pulse Oximeter to your finger
- Explosion hazard: do not use the product in an explosive atmosphere

STORAGE CAUTIONS AND WARNINGS

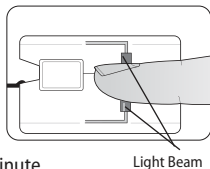
- Never store the Pulse Oximeter the unit in direct sunlight, high temperature or humidity
- Keep the unit out of reach of small children

CLEANING CAUTIONS AND WARNINGS

- Never immerse the device in water to clean as it may be damaged
- Follow the included instructions for cleaning the unit after each use; if the unit has not been used for an extended period, the device should be cleaned prior to use.



HOW THE PULSE OXIMETER WORKS

The Pulse Oximeter works by shining two light beams into the small vessels or capillaries of the finger to determine the user's the blood oxygen saturation percentage and pulse rate that is then displayed on the Pulse Oximeter's screen. The unit uses the phrase "SpO2" to identify the oxygen saturation level and displays it as a percentage of full capacity (100%). Pulse rate is identified by the phrase "PR bpm" and is displayed as beats per minute.

DIGITAL DISPLAY FEATURES

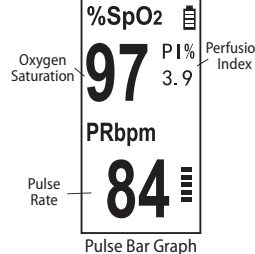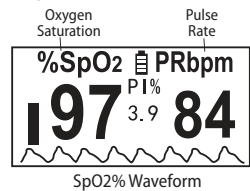When in use, the Pulse Oximeter display reflects a 'live' reading of the wearers current SpO2, PR bpm and perfusion index.

SCREEN DISPLAY ORIENTATION

The Pulse Oximeter display can be adjusted to allow for easy viewing from any angle. While the unit is ON, rotating the finger while the oximeter is on and measuring will automatically change the display orientation, as shown below.

USING YOUR PULSE OXIMETER

The Pulse Oximeter may be used while the user is seated, standing or lying down. The user should not be walking or running during measurements and should take care not to excessively move the arm or hand where the oximeter is attached.

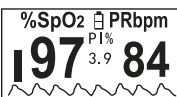
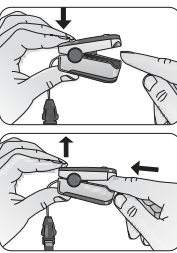
PRIOR TO USE

Keep the sensors clean to help ensure accurate readings. Before and after each use, carefully swab inside the unit with a soft cloth or cotton swab with rubbing alcohol. Do not pour the alcohol directly on or into the unit.

Dirty hands or nail polish (especially dark shades) can also affect reading accuracy. Washing your hands and/or removing any nail polish prior to use to help ensure accurate readings.


Suitable for use on any finger (excluding the thumb). In most instances, the index or middle finger work best.

- Squeeze the hinged end of the unit to open the measurement side of the unit.
- Insert finger nail side up into the unit until resistance is felt. The unit should gently close below the first knuckle of the finger, and the sensors inside the unit should align with the finger nail and pad at the tip of the finger.
- Press the power button to turn the unit on. The screen will briefly flash a test display, and then begin the measurement.
- Solid dashes (---) will appear on the display for a few seconds, then the display will begin to show the oxygen saturation (SpO2) and pulse rate (PR bpm) measurements. The display will vary depending on which display mode orientation is in use. Turning the user's hand will rotate the display for best view.
- Take care to not move excessively during monitoring. These are active readings and may fluctuate slightly as you observe the display, responding to your heart rate (bpm) and other factors.
- The pulse oximeter should only be used for spot-checking blood oxygen saturation and pulse rate and should not remain on the finger for more than a few minutes at a time.
- Remove the unit from the finger. The unit will automatically turn off after 5 seconds.
- Clean the unit as directed before storage.

QUICK-RELEASE LANYARD

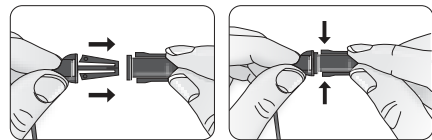The lanyard features a quick-release clip for easy release while in use.


CHOKING HAZARD-The included lanyard may present an entanglement or choking hazard to small children. Adult supervision required; never leave children unattended with unit or lanyard.

BATTERY INSTALLATION AND REPLACEMENT

This unit comes complete with 2 AAA batteries.

It is necessary to replace the batteries when the Low Battery symbol appears on the display or when the display does not turn on after the power button is pressed.

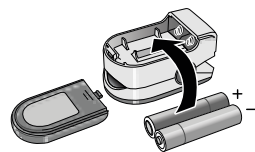
- Slide off the battery cover.
- Insert or replace 2 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols.
- Replace the battery cover.
- It is recommended to remove the batteries if the unit will not be used for an extended period of time.

Always use new batteries. Do not mix alkaline, standard or rechargeable batteries. Please dispose of batteries according to local recycling regulations.

CARE AND MAINTENANCE

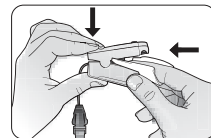Before and after each use, carefully swab inside the unit with a soft cloth or cotton swab with rubbing alcohol. Do not pour the alcohol directly on or into the unit.

The unit should be stored in a dry environment, within the ideal temperature range of -40°F-140°F (-40°C-60°C).

TROUBLESHOOTING

SpO2 or PR is not displaying a reading though the power is on

- The user's oxygen level is too low to be measured; consult your physician if the measurement cannot be read
- Reinsert the finger, remain still

SpO2 or PR display readout is unstable and fluctuating highly

- Finger may not inserted deep enough
- Finger may be trembling, try to remain still

The unit will not turn on

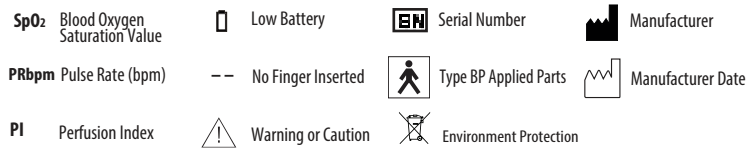
- Ensure the batteries have been properly installed
- Replace the batteries
- If the unit will not turn on after changing batteries, consult Customer Care

Display screen goes blank

- The unit will automatically shut-off after 5 seconds if no finger is inserted
- Check the batteries and replace if needed

DEVICE AND LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model:

GUIDANCE AND MANUFACTURE'S DECLARATION-ELECTROMAGNETIC EMISSION

FOR ALL EQUIPMENT AND SYSTEMS

Guidance and manufacture's declaration -electromagnetic emission		
The Pulse Oximeter is tended for use in the electromagnetic environment specified below. The user of the Pulse Oximeter should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The Pulse Oximeter uses RF energy only for their internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Pulse Oximeter is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emission IEC 61000-3-3	Not applicable	

GUIDANCE AND MANUFACTURE'S DECLARATION-ELECTROMAGNETIC IMMUNITY

FOR ALL EQUIPMENT AND SYSTEMS

Guidance and manufacture's declaration-electromagnetic immunity			
The Pulse Oximeter is tended for use in the electromagnetic environment specified below. The user of the Pulse Oximeter should assure that it is used in such an environment.			
Immunity test	IEC60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6KV contact ±8KV air	±6KV contact ±8KV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

GUIDANCE AND MANUFACTURE'S DECLARATION-ELECTROMAGNETIC IMMUNITY

FOR EQUIPMENT AND SYSTEMS THAT ARE NOT LIFE-SUPPORTING

Guidance and manufacture's declaration-electromagnetic immunity			
The Pulse Oximeter is tended for use in the electromagnetic environment specified below. The user of the Pulse Oximeter should assure that it is used in such an environment.			
Immunity test	IEC60601 test level	Compliance level	Electromagnetic environment-guidance
Radiated RF ICE 61000-4-3	3V/m 80MHz to 2.5GHz	3V/m	Portable and mobile RF communication equipment should be used no closer to any part of the Pulse Oximeter, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. recommended separation distance 80MHz to 800MHz 800MHz to 2.5GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE 1 At 80MHz and 800MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Pulse Oximeter is used exceeds the applicable RF compliance level above, the Pulse Oximeter should be observed to verify normal operation. If abnormal, performance is observed, additional measures may be necessary, such as reorienting or relocating the Pulse Oximeter.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE

RF COMMUNICATIONS EQUIPMENT AND THE EQUIPMENT OR SYSTEM

FOR EQUIPMENT OR SYSTEM THAT NOT LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the Pulse Oximeter			
The Pulse Oximeter is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Pulse Oximeter can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Pulse Oximeter as recommended below, according to the maximum output power of the communications equipment.			
	Separation distance according to frequency of transmitter (m)		
Rated maximum output power of transmitter (W)	150KHz to 80MHz	80MHz to 800MHz	800MHz to 2.5GHz
	$d = \left[\frac{3.5}{f} \right] \cdot \sqrt{P}$	$d = \left[\frac{3.5}{f} \right] \cdot \sqrt{P}$	$d = \left[\frac{7}{f} \right] \cdot \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80MHz and 800MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

PRODUCT SPECIFICATIONS

NAME	Deluxe Pulse Oximeter
MODEL NUMBER	11-50D
DISPLAY SYSTEM	Digital TFT display
POWER SOURCE	2 "AAA" batteries
SPO2 MEASURING RANGE	0-100%
SPO2 ACCURACY	70-100%, ±2%; ≤69% no definition
PULSE RATE MEASURING RANGE	30-250 bpm
PULSE RATE ACCURACY	30-250 bpm, ±2 bpm
PERFUSION INDEX MEASURING RANGE	0-20%
AUTOMATIC SHUT-OFF	After 5 seconds of no indication on the sensors
OPERATION ENVIRONMENT	Temperature 50°F – 104°F (10°C – 40°C); Humidity 30% – 75% Atmospheric Pressure: 700hPa – 1060hPa
STORAGE ENVIRONMENT	Temperature -40°F – 140°F (-40°C – 60°C); Humidity 5% – 95% Atmospheric Pressure: 500hPa – 1060hPa
UNIT DIMENSIONS	2-1/4" x 1-1/4" x 1-1/4"
UNIT WEIGHT	1.7 oz. (with batteries); 0.9 oz. (without batteries)

Specifications are subject to change without notice

TWO-YEAR LIMITED WARRANTY

Congratulations on your purchase of a Pulse Oximeter. Your Pulse Oximeter is covered by the following limited warranty commencing upon the date of purchase for a period of two years, and subject to the following terms and conditions:

The warrantor warrants that its Pulse Oximeter will be free from defects in materials and workmanship under normal consumer usage for as long as the original purchaser owns the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Pulse Oximeter for commercial purposes or subjecting the Pulse Oximeter to abnormal usage or conditions; or (d) other acts which are not the fault of the warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Pulse Oximeter.

If the Pulse Oximeter fails to conform to this limited warranty, return the Pulse Oximeter to: **Veridian Healthcare, Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031.** When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, The warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

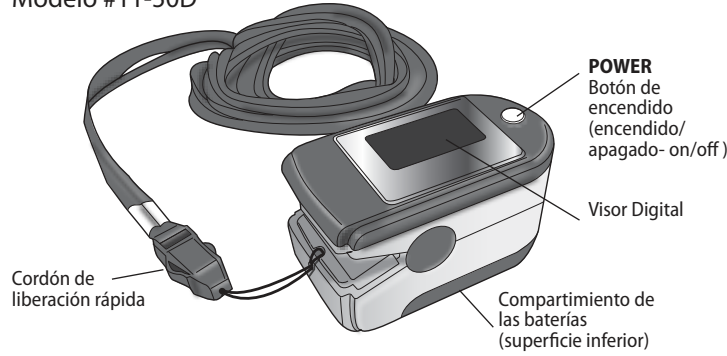
Distributed by:
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com
Made in China
#93-1094 09/22
©2022 Veridian Healthcare, LLC

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