THERA  CARE

Small Size Heating Pad

Model 24-810 series, 9"x9"

**PLEASE READ THESE INSTRUCTIONS
CAREFULLY TO ENSURE PROPER USE AND
SAFE OPERATION OF YOUR HEATING PAD.**

SAVE THESE INSTRUCTIONS

INSTRUCTION MANUAL • ENGLISH

INDEX

Introduction.....	2
Warnings & Cautions	3-5
Power/Polarized Plug	6
Electrical Specifications.....	6
How to Use Your Heating Pad	7-9
Care & Maintenance.....	10
Frequently Asked Questions.....	11
FCC Information	12
Warranty	13-14
Instrucciones en Español.....	15-28

Toll-Free Customer Care
Help Line: 1-866-326-1313

Monday – Friday
8:30 a.m. – 4:30 p.m. CST

Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1481 05/20
©2020 Veridian Healthcare, LLC

INTRODUCTION

Thank you for purchasing this Heating Pad. Please read this instruction manual completely before using your unit to ensure safe and effective use, as well as long life of the product.

Heating pads are intended for personal comfort, including temporary relief of minor aches and pains.

WARNINGS & CAUTIONS

**PLEASE READ THIS INSTRUCTION
MANUAL COMPLETELY BEFORE
OPERATING THIS DEVICE.**

	 DANGER • Do not sit on pad. • Do not use while sleeping. • Never position pad underneath the body. • Do not fold pad. • Avoid sharp folds that may damage the product.
---	--

	 DANGER Hot Surfaces.
--	--

**DANGER-- TO REDUCE THE RISK OF
BURNS, ELECTRIC SHOCK AND FIRE,
THIS PRODUCT MUST BE USED IN
ACCORDANCE WITH THE FOLLOWING
INSTRUCTIONS.**

WARNINGS & CAUTIONS

1. Read and follow all instructions before use.
2. Do not use while sleeping.
3. Do not use on an infant or on an animal.
4. The unattended use of the heating pad by children or incapacitated persons may be dangerous.
5. Do not use over sensitive skin areas or in the presence of poor circulation.
6. Do not use a heating pad on areas of insensitive skin.
7. Do not use in an oxygen enriched environment or near equipment which stores or emits oxygen.
8. Do not put this pad in a microwave.
9. Do not tamper with this pad in any way.
10. Carefully examine vinyl pad before each use. Discard if the vinyl pad shows any signs of deterioration, such as blistering or cracking.
11. Never use pad without the removable cover in place.
12. Place pad on top of and not under the part of the body needing heat.
13. Do not sit on or lean against or crush pad. Avoid sharp folds.


WARNINGS & CAUTIONS

14. Burns may occur when the power is on. Check the skin under the pad frequently to avoid burning and blistering.
15. Do not use with liniment, salve or ointment preparations, especially ones that contain heat producing ingredients. Skin burns could result.
16. Never use pins or other metallic means to fasten the pad in place.
17. Only use with a 120 volt AC circuit.
18. Never pull this pad by the supply cord. Do not use the cord as a handle.
19. Do not bend or pinch cord. Do not use if cord is damaged.
20. Unplug when not in use.
21. Loop cord loosely when storing. Tight wrapping may damage cord and internal parts.
22. There are no user-serviceable parts. If for any reason the pad does not function satisfactorily see warranty for consumer service information.

POWER / POLARIZED PLUG

This heating pad has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way.

If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. NEVER ATTEMPT TO OVERRIDE THIS SAFETY FEATURE.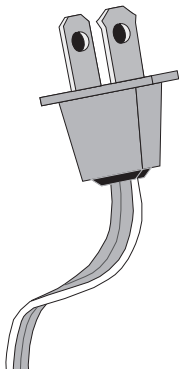

ELECTRICAL SPECIFICATIONS

VOLTAGE	FREQUENCY	WATTAGE	AMPERAGE
120 VAC	60 HZ	42 W	0.35 A

HOW TO USE YOUR HEATING PAD

Dry heat therapy—such as traditional electric heating pads or saunas—offers a fast solution for therapeutic heat and is easy to use and apply.



Moist heat therapy—such as moist heat packs, hot baths, or steamed towels—can help aid in the heat's penetration into the muscles.

Doctors, physical therapists and sports trainers often recommend both methods as a means for soothing muscles aches and pains. Consult with your physician if you have questions regarding what type of therapy is best for your ailment.

ALWAYS USE THE INCLUDED CLOTH COVER WHILE OPERATING THIS HEATING PAD. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY INCLUDING BURNS. Read the **WARNINGS AND CAUTIONS** section of this manual before use and always use common sense care when operating any electrical device.

HOW TO USE YOUR HEATING PAD


1. Insert the PVC pad fully into the cloth cover; the cord and control should extend fully outside of the cover.
2. Take care that the pad is flat and not folded or bunched within the cover.
3. The sponge pad is for use with Moist Heat therapy; if Moist Heat is desired, follow the Moist Heat instructions on the next page.
4. Plug the product into a standard 120-volt AC electrical outlet.
5. Position the heating pad for desired treatment. The pad should always be positioned on top of—NOT UNDERNEATH—the treatment site as the temperature can increase to uncomfortable levels when the heat is trapped.
6. Press the **ON/OFF** button to turn the heating pad on; the power light will illuminate.
7. Monitor your comfort level and discontinue use if heat is uncomfortable.
8. The heating pad will turn off after approximately 2 hours. Always unplug the device when not in use.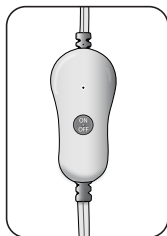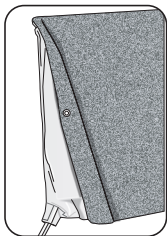

HOW TO USE YOUR HEATING PAD

MOIST HEAT OPTION

1. Saturate the SPONGE only with warm water. Do not use any gels, solvents or other liquid to saturate the sponge.
2. Lightly wring out excess water so that the sponge is not dripping.
3. Place the sponge on the PVC pad and insert the both pieces fully into the cloth cover; the cord and control should extend fully outside of the cover.
4. Take care that the pad is flat and not folded or bunched within the cover.
5. Dry hands thoroughly and plug the product into a standard 120-volt AC electrical outlet.
6. Position the heating pad for desired treatment. The pad should always be positioned on top of—NOT UNDERNEATH—the treatment site as the temperature can increase to uncomfortable levels when the heat is trapped.
7. Press the **ON/OFF** button to turn the heating pad on; the power light will illuminate.
8. Monitor your comfort level and discontinue use if heat is uncomfortable.
9. The heating pad will turn off after approximately 2 hours. Always unplug the device when not in use.

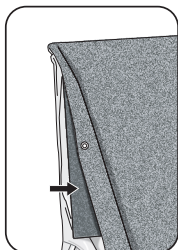CARE AND MAINTENANCE

DISCONNECT THE HEATING PAD FROM THE ELECTRICAL OUTLET AFTER USE AND BEFORE CLEANING.

Remove the pad and sponge from the outer fabric cover before cleaning.

- **VINYL PAD**—The vinyl pad may be wiped down as needed but should not be immersed in water
- **OUTER FABRIC COVER**—Machine or hand wash the outer fabric cover using a mild detergent, a gentle/delicate cycle, with cool water only. To dry, hang the outer cover or lay it flat. Do not machine dry.
- **MOIST HEAT SPONGE**—Hand wash the sponge using a mild detergent and cool water only; lay flat to dry. Do not machine dry.

Store the pad, sponge, and outer cover in a cool, dry location after each use. Do not reassemble the cover and pad until completely dry.

When storing, avoid sharply folding the pad as over time cracks may occur in the vinyl.

FREQUENTLY ASKED QUESTIONS

When should a heating pad be used?

Heating pads as recommended by a health care professional can be used for heat therapy or as desired for personal comfort.

How does heat therapy help?

Heat therapy enlarges the blood vessels of muscles, increasing the flow of oxygen and nutrients to the muscles. This therapy helps relieve painful discomfort, decreases muscle stiffness and increases flexibility.

Why did my heating pad stop working or cool down?

For safety purposes, this device will automatically shut-off after 2 hours.

What temperature does this product reach?

This heating pad is designed to reach the highest temperature limit allowed by Underwriters Laboratories, 131~140°F / 55~60°C when set on the highest setting.

How much time does it take to warm up to full heat strength?

Full Strength — Approximately 8 minutes

FCC INFORMATION

IMPORTANT INFORMATION REQUIRED BY THE FCC

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference; and (2) this device must accept any interference received, including interference that may cause undesirable operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

1-YEAR LIMITED WARRANTY

The warrantor guarantees that its Heating Pad will be free from manufacturing defects under normal use for a period of one-year. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty is voided if the unit is misused or abused in any manner.

If the Heating Pad fails to conform to this limited warranty, return the Heating Pad postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031.** When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT

1-YEAR LIMITED WARRANTY

LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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