QUICK START GUIDE

3 LANGUAGES

ENGLISH, ESPAÑOL AND LE FRANÇAIS

SMARTHEART™

Talking Blood Pressure Monitor

Model # 01-521

Please read this instruction manual
completely before operating this unit.

ENGLISH

SMARTHEART™

Talking Digital Blood Pressure Arm Monitor

Model # 01-521

**THIS QUICK START
GUIDE IS INTENDED
AS A SUPPLEMENT
TO THE INSTRUCTION
MANUAL. PLEASE READ
THE COMPLETE MANUAL
PRIOR TO FIRST USE.**

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff.

To determine the proper cuff size, measure the circumference of your left upper arm at the point midway between inside bend of your elbow and your shoulder.



Wide-Range Cuff fits arm circumference 8.6" – 16.5" (22–42 cm)

Additional cuff sizes are available for order on the enclosed order form. Additional cuffs are available for order on the included order form or by calling 866-326-1313.

A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.

Applying the Arm Cuff

Remove any constrictive clothing or jewelry that may interfere with cuff placement. Wrap the cuff snugly around your left arm. Position the cuff's bottom edge one inch ABOVE the elbow joint. Position the air tubing over the main artery (on the inside of your arm). You should be able to insert one finger between your arm and cuff.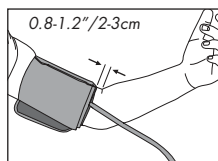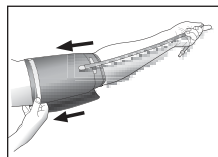

**IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING
DIFFICULTIES, PLEASE CALL THE TOLL-FREE
CUSTOMER SERVICE HELP LINE FOR FURTHER
ASSISTANCE AT 1-866-326-1313.**

ENGLISH

Please read the instruction manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

User will default to the last User Bank used; consult the manual for instructions on how to change the User to ensure accurate memory storage and review.

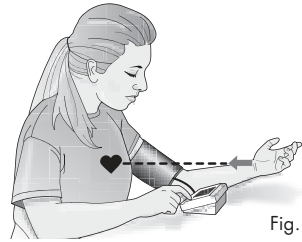
Fig. 1

1. Position the monitor on a flat, stable surface with the digital display in view. Insert the cuff tubing connector into the port on the left side of your monitor. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Relax your left hand.
2. Press the **START/STOP** button to turn the unit on; the unit will briefly flash a self-test, Fig 2. A voice tone will indicate when the unit is ready for testing..
3. The cuff will automatically inflate to approximately 180mmHg; you will see the numbers go up on the display screen.
4. When the inflation has reached optimum level, the display will begin to show the decreasing pressure while you feel the pressure of the cuff decrease.
5. When the measurement is complete, your blood pressure measurement and pulse will display simultaneously on the screen with a voice broadcast, Fig 3.
6. Consult the included Instruction Manual for information on interpreting your reading results.
7. The reading will automatically be stored in memory bank selected prior to the reading, up to 99 readings.
8. Select **START/STOP** to turn the unit off and conserve energy and battery life. The unit will automatically shut-off after 3-minutes

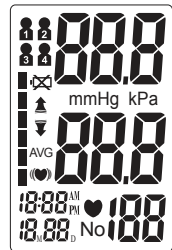
Fig. 2

Fig. 3

Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the **START/STOP** button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

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©2018 Veridian Healthcare, LLC