# QUICK START GUIDE

## 3 LANGUAGES

ENGLISH, ESPAÑOL AND LE FRANÇAIS

SMARTHEART™

## Talking

### Blood Pressure Wrist Monitor

Model # 01-521

Please read this instruction manual completely before operating this unit.

**ENGLISH**

SMARTHEART™

# Talking

## Blood Pressure Wrist Monitor

Model # 01-527

**THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.**



Before applying your blood pressure cuff, be sure your wrist is within the appropriate cuff range:

**Wrist Circumference Range** Suitable for 5.3" – 8.5" (13.5 – 21.5 cm).

*A 15 minute resting period is needed to stabilize the body prior to taking a reading. No eating, drinking or smoking while resting.*

### Applying the Cuff

Remove any constrictive clothing or jewelry that may interfere with cuff placement. Hold your left arm in front of you with your palm facing upward. Apply the preformed cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you. Adjust the cuff 1/4" - 1/2" from the edge of the head of the ulna bone. The cuff should fit comfortably, yet snugly around your wrist.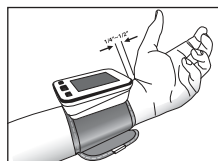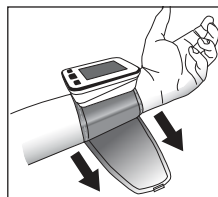

**IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.**

# ENGLISH

Please read the instruction manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

User will default to the last User Bank used; consult the manual for instructions on how to change the User to ensure accurate memory storage and review.

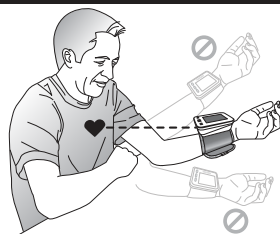
Fig. 1

1. Position the monitor on a flat, stable surface with the digital display in view. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart, Fig 1. Relax your left hand.
2. Press the **START/STOP** button to turn the unit on; the unit will briefly flash a self-test, Fig 2. A voice tone will indicate when the unit is ready for testing.
3. The cuff will automatically inflate to approximately 180mmHg; you will see the numbers go up on the display screen.
4. When the inflation has reached optimum level, the display will begin to show the decreasing pressure while you feel the pressure of the cuff decrease.
5. When the measurement is complete, your blood pressure measurement and pulse will display simultaneously on the screen with a voice broadcast, Fig 3.
6. Consult the included Instruction Manual for information on interpreting your reading results.
7. The reading will automatically be stored in memory bank selected prior to the reading, up to 99 readings.
8. Select **START/STOP** to turn the unit off and conserve energy and battery life. The unit will automatically shut-off after 3-minutes

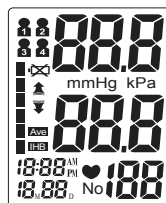
Fig. 2

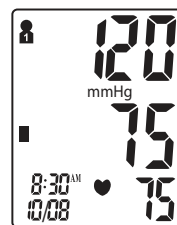
Fig. 3

## Please Note:

- You can stop the inflation or deflation process of the cuff at any time by pressing the **START/STOP** button.
- Many factors such as genetics, age, sex, altitude, physical activity, anxiety, muscular development, certain medications or even the time of day can influence blood pressure.
- Blood pressure varies from moment to moment; even readings taken after proper rest periods can vary. Home blood pressure readings may vary as much as 10-20 points from ones taken at your doctor's office.
- Variations in your individual readings should only be interpreted by your physician or trained health care professional.
- We encourage you to consult with your physician to achieve the maximum benefit from your blood pressure monitor.

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©2018 Veridian Healthcare, LLC