instruction manual

WRIST BLOOD PRESSURE MONITOR

MODEL 302-WS380

Please read this instruction manual completely before operating this unit.

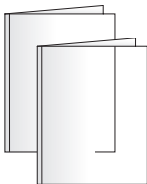
STOP!
**PLEASE ENSURE YOU HAVE ALL OF
THE FOLLOWING COMPONENTS
BEFORE USING YOUR BLOOD
PRESSURE MONITOR**

Monitor

Two (2) AAA Batteries,
Preinstalled

Instruction Manual
Quick Start Guide
Trackstar App Guide

Toll-Free Customer Care Help Line:

1-866-326-1313

Monday - Friday

8:30 a.m. - 4:30 p.m. CST

www.veridianhealthcare.com

INDEX

Care & Safety Information	4-12
Introduction.....	12-13
Monitor Features	14-16
Trackstar Wellness App	17
Getting Started	18-19
Battery Installation	20
Setting Date/Time	21-22
Applying Your Blood Pressure Cuff	23-24
Pairing	25
Taking Your Blood Pressure Reading	26-28
Irregular Pulse Rate Detection.....	29-30
Hypertension Indicator	31
Memory Recall.....	32-33
Care & Maintenance	34-36
Troubleshooting.....	37-38
Device & Label Symbols.....	39-40
Bluetooth Module	41-42
FCC Statement.....	43-44
Electromagnetic Compatibility.....	45-47
Specifications	48-49
Warranty	50-51
Instrucciones en Español	53-103
Download App	104

Trackstar™
Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1683 01/26
©2026 Veridian Healthcare, LLC

CARE & SAFETY INFORMATION

NOTE: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

CAUTION: Failure to read and observe all precautions could result in personal injury or equipment damage.

PRECAUTIONS

- Blood Pressure Monitor is intended to be operated by adults, including medical staffs and lay persons. Adult patients could also be intended users or operators.
- This device is intended for indoor, home use and is not intended for self-use in public areas.
- This device is portable, but it is not intended for use during patient transport.
- This device is not suitable for continuous monitoring during medical emergencies or operations.
- This device is intended for non-invasive measuring and monitoring of arterial blood pressure.
- It is not intended for use on extremities other than the arm, or for any purpose other than obtaining a blood pressure measurement.

CARE & SAFETY INFORMATION

- This device is intended for adults. Do not use this device on neonates or infants. Do not use it on children and adolescents unless otherwise instructed by a medical professional.
- Consult with your physician before using this monitor if you suffer from the following conditions: common arrhythmias such as premature ventricular beats or atrial fibrillation; peripheral arterial disease; pregnancy; preclampsia; implantation with electrical devices; undergoing intravascular therapy; arteriovenous shunt or mastectomy. Please note that any of these conditions may affect measurement readings and patient motion, trembling or shivering.
- If you are taking medication, consult your physician to determine the proper time to measure your blood pressure.
- This device may be used only for the intended purpose described in this manual. The manufacturer shall have no liability for any incidental, consequential, or special damages caused by misuse or abuse.
- Please use the device under the environment provided in the user manual. Otherwise, the device's performance and lifetime will be impacted and reduced.

CARE & SAFETY INFORMATION

- The device may require up to 30 minutes to warm up / cool down from the minimum/ maximum storage temperature before it is ready for use.
- The blood pressure monitor, its adapter, and the cuff are suitable for use within the patient environment.
- Do not wash the cuff in a washing machine or dishwasher.
- The device contains sensitive electronic components. To avoid measurement errors, avoid taking blood pressure measurements near a strong electromagnetic field radiated interference signal or electrical fast transient/burst signal.
- Wireless communication equipment, such as wireless home network devices, mobile phones, cordless telephones, and their base stations, walkie-talkies may cause interference that may affect the accuracy of measurements. A minimum distance of 1 foot (30 cm) should be kept from such devices during a measurement.
- Please choose the appropriate cuff according to your arm circumference and physical health.

CARE & SAFETY INFORMATION

CAUTIONS

- Do not attempt to repair the unit yourself if it malfunctions. Only have repairs carried out by authorized service centers.
- It is recommended that the performance should be checked after repair, maintenance, and every two years of use, by retesting the requirements in limits of the error of the cuff pressure indication and air leakage (testing at least at 50 mmHg and 200 mmHg). Please contact manufacturer or distributor for authorized service personnel.
- Store your device, cuff, and adapter in a clean and dry place, protect it against extreme moisture, heat, lint, dust, and direct sunlight. Never place any heavy objects on it.
- Make sure the rubber tube of the cuff is not squeezed, stretched, or kinked during storage.
- Dispose of accessories, detachable parts, and the device according to the local guidelines.

CARE & SAFETY INFORMATION

WARNINGS

- DO NOT self-diagnose the measurement results and start treatment by yourself. The measurement results given by this device is not a diagnosis. ALWAYS consult your doctor for evaluation of the results and treatment.
- NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure.
- DO NOT apply the cuff on an arm that has an intravenous drip or a blood transfusion attached.
- DO NOT kink, fold, stretch, compress, or otherwise deform the tube during measuring, as the cuff pressure might continuously increase, which could prevent blood flow and result in injury.
- Taking blood pressure measurements too frequently could disrupt blood circulation and cause injuries.
- DO NOT apply the cuff to areas on patient where the skin is delicate or damaged. Check the cuff site frequently for irritation.

CARE & SAFETY INFORMATION

- DO NOT place the cuff on the arm of a person whose arteries or veins are undergoing medical treatment, i.e. intra-vascular access or intra-vascular therapy or an arteriovenous (A-V) shunt, which could disrupt blood circulation and cause injuries.
- DO NOT place the cuff on the arm on the same side of a mastectomy (especially when lymph nodes have been removed). It is recommended to take measurements on the unaffected side.
- DO NOT wrap the cuff on the same arm to which another monitoring device is applied. One or both devices could temporarily stop functioning if you try to use them at the same time.
- Warning: Please check (for example, by observation of the limb concerned) that the operation of the device does not result in prolonged impairment of patient blood circulation.
- Warning: On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, loosen and remove the cuff immediately. Prolonged high-pressure applied to the arm (cuff pressure >300 mmHg or constant pressure >15 mmHg for

CARE & SAFETY INFORMATION

more than 3 minutes) might lead to bruising and discolored skin.

- DO NOT use this device with high-frequency (HF) surgical equipment at the same time.
- This device is not used in conjunction with oxygen rich environments, not intended for use with flammable anesthetics, not intended for use in conjunction with flammable agents.
- Excessive cuff tube lengths could cause strangulation if you don't manage them properly.
- DO NOT touch output of the batteries/ adapter and the user simultaneously.
- The power cord is considered the disconnect device for isolating this equipment from supply mains. DO NOT position the equipment so that it is difficult to reach or disconnect.
- DO NOT use this device if you are allergic to polyester, nylon, or plastic.
- Only use accessories approved by manufacturer. Using unapproved accessories might cause damage to the unit and injure users.
- If you experience discomfort during a measurement, such as pain in the arm or other

CARE & SAFETY INFORMATION

complaints, press the Power button immediately to release the air from the cuff.

- DO NOT use the device while under maintenance, or being serviced.
- Sensor degradation or looseness may reduce performance of device or cause other problems.
- The air tube poses a risk of strangulation. Furthermore, the small parts of product and batteries present a choking hazard if swallowed. They should therefore always be kept away from infants/children.

NOTICE

- You can use this device to take your own measurements, no third-party operator is required.
- Adapter is specified as a part of ME EQUIPMENT.
- At the request of authorized service personnel, circuit diagrams, component part lists, descriptions, and calibration procedures will be made available by the manufacturer or distributor.
- The expected lifetime of the cuff may vary

CARE & SAFETY INFORMATION

by the frequency of washing, skin condition, and storage state.

- Please report to the manufacturer and the competent authority of the Member State / the FDA in which you are established about any serious incident that has occurred in relation to this device.

INTRODUCTION

It is recommended that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.

Blood pressure readings determined with this device are equivalent to measurements obtained by a trained healthcare professional using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

INTRODUCTION

Thank you for purchasing an Automatic Blood Pressure Monitor. With proper care and use, your monitor will provide you with many years of reliable readings.

INDICATIONS FOR USE

This Wrist Blood Pressure Monitor is a digital monitor intended for use in measuring blood pressure and pulse rate with a wrist circumference ranging from 13.5cm to 21.5 cm (about 5.3" - 8.4"). It is intended for adult indoor use only.

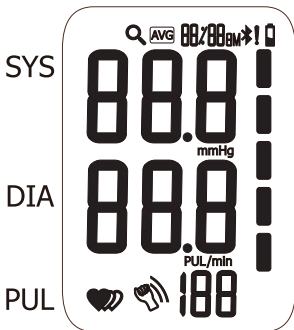
MEASUREMENT PRINCIPLE

This product uses the Oscillometric Measuring Method to detect blood pressure. Before every measurement, the unit establishes a "zero point" equivalent to the atmospheric pressure. Then it starts inflating the cuff. Meanwhile, the unit detects pressure oscillation generated by beat-to-beat pulsatile, which is used to determine the systolic pressure and diastolic pressure as well as pulse rate.

MONITOR FEATURES


MONITOR FEATURES

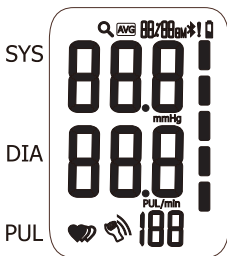SYMBOL	DESCRIPTION	EXPLANATION
SYS	Systolic Pressure	High blood pressure
DIA	Diastolic Pressure	Low blood pressure
PUL/min	Pulse Display	Pulse beats per minute
mmHg	mmHg	Unit of blood pressure measurement
 + Lo	Low Battery	Battery replacement needed
88:88	Current Time	Hour: Minute
	Pulse Rate	Pulse rate is shown

MONITOR FEATURES

SYMBOL	DESCRIPTION	EXPLANATION
	Irregular Pulse Rate	Appears when an irregular pulse is detected
	Results Average	Display shows the average of last 3 results
	Blood Pressure Level Indicator	Indicates blood pressure level
	Bluetooth	Indicates signal strength
	Memory Recall	Display is showing memory recall function
	Data Storage Alert	Data is waiting to be transferred to App
	Motion Indicator	Motion may result in an inaccurate measurement

TRACKSTAR WELLNESS APP

Installing the Trackstar Wellness app is easy.

Simply download the app, prepare your device, and start your health journey. Consult your healthcare professional before starting a new health regime, changing medication and for aid in interpreting your personal results.

1. Download the **Trackstar Wellness** app from the App Store (iOS) or Google Play (Android).
2. Launch the app and follow the instructions for account profile set up.
3. Pair your blood pressure monitor following the on-screen directions.

THIS BLOOD PRESSURE MONITOR OPERATES WITH OR WITHOUT THE TRACKSTAR APP. SOME FEATURES, SUCH AS MEMORY STORAGE AND RECALL, CAN ONLY BE ACCESSED THROUGH THE APP.

GETTING STARTED

BEFORE TAKING A MEASUREMENT

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.

GETTING STARTED

6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
- Wait at least 30 min required for ME equipment to warm from the minimum storage temperature between uses until it is ready for intended use. At least 30 min required for ME equipment to cool from the maximum storage temperature between uses until it is ready for intended use.
 - Before use, make sure the device functions safely and is in proper working condition.

BATTERY INSTALLATION

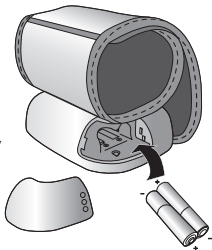
THIS UNIT REQUIRES 2 AAA BATTERIES, PREINSTALLED.

When using for the first time, remove the pull strip to activate the batteries.

It is necessary to replace the batteries when the “Low Battery” symbol + Lo’ appears on the display or when the display does not turn on after the **START STOP** button is pressed.

The battery compartment is located on the top of this monitor.

1. Slide open the battery compartment at the top of the unit.
2. Insert or replace 2 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

It is recommended to remove the batteries if the unit will not be used for an extended period of time.

SETTING DATE / TIME

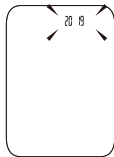
The date / time will automatically sync when connected to the Trackstar app.

If you choose to manually set the date and time, follow the instructions below.

1. When the device is off, press and release the **SET** button. The time will show on the display.
2. Press and hold the **SET** button and the YEAR will flash on the display. Press **M** to advance the YEAR; press **SET** to confirm.
3. MONTH/DATE—repeat the same sequence of step 2 to confirm the MONTH and DATE setting.
4. HOURS/MINUTES—repeat the same sequence of step 2 to confirm the HOUR and MINUTE setting.
5. The date/time feature is on the top of the display screen. The date and time will alternate while the device is in use.
6. After confirming the minutes, the unit of measurement will flash, repeat the same sequence of step 2 to confirm the unit of measurement setting.

SETTING DATE / TIME


7. The monitor will show 'donE' indicating ready to measure.

Year

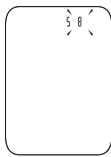Month

Date

Hour

Minutes

FITTING & APPLYING CUFF

PROPER CUFF FIT AND PLACEMENT IS CRITICAL IN OBTAINING ACCURATE BLOOD PRESSURE MEASUREMENTS. PLEASE READ THESE INSTRUCTIONS CAREFULLY AND CONSIDER CONSULTING WITH YOUR PHYSICIAN FOR FURTHER DIRECTION.

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff.

Wrist Circumference Range

Suitable for 5.3" – 8.4" (13.5 cm – 21.5 cm).

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

If for any reason you are unable to or should not use your left wrist, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you to take measurements from.

Applying Your Cuff

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.

FITTING & APPLYING CUFF

2. Be seated at a table or desk with your feet flat on the floor.
3. Hold your left arm in front of you with your palm facing upward.
4. Apply the cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you, Fig. 1.
5. Adjust the cuff approximately 0.5" from the edge of the head of the ulna bone, Fig. 2.
6. The cuff should fit comfortably, yet snugly around your wrist.

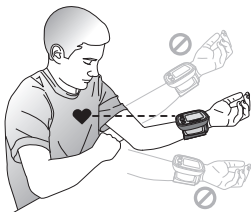
Fig. 1

Fig. 2

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

PAIRING

APP INSTALLATION AND PAIRING:

- Ensure your phone bluetooth is enabled, device is on and the app is open to establish connection.
- Follow the onscreen instructions in the app for pairing this device.
- Press and hold the **START/STOP** button for 4-seconds.
- The monitor display will show pairing is in progress; when paired, the monitor display will state the pairings is successful.

If you received this monitor as part of a health-care maintenance program, directly from your insurance or healthcare provider, or other health management program, please consult with the program paperwork prior to use and set up.

DEVICE SETTINGS:

- There is no need to set date/time on this device. Once paired with an app or assistive device, the date and time will automatically sync.
- It is not necessary to sync after every measurement; this device will store up to 60 measurements. If you do not sync after 60 readings, display will read FULL+!. Please sync at your earliest convenience.

TAKING YOUR BP READING

Please read the preceding portions of this manual prior to taking your first reading.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

1. Rest your elbow on a solid surface with your palm facing upward. Elevate your arm so that the cuff is at the same level as your heart. Be certain that cuff is elevated to the proper level; you may choose to support your hand with a folded towel or book. Relax your left hand. Be seated with your feet flat on the floor and legs uncrossed.
2. To sync results in real time, ensure App is open and bluetooth is enabled on your smart device.
3. Press and release the **START/STOP** button to start the measurement process.
4. The unit will briefly show a test display then O before the measurement begins.
5. The wrist cuff will automatically inflate; you will see the numbers increase on the display screen and feel the cuff inflate.
6. When the measurement is complete, the cuff will automatically deflate, then your blood

TAKING YOUR BP READING

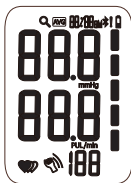
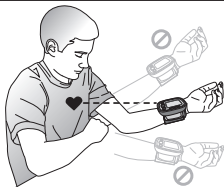
pressure measurement and pulse reading results will show on the display screen at the same time.

7. The monitor will automatically save and transmit measurement to your device if both Bluetooth and App are ON.
8. If transmission fails or Bluetooth and App are not active, results will be stored in measurement up to 60 readings.
9. Press **START/STOP** to turn off monitor; device will automatically shut off after 1 minute.

TAKING YOUR BP READING

SIMPLE STEPS:

BE SEATED, FEET FLAT ON THE FLOOR, CUFF AT THE SAME LEVEL AS YOUR HEART.

Press **START/STOP**;
test display will show

0 will briefly show
before measurement
begins

As cuff inflates, the
numbers will increase
on the display

Results will show on
the display; the blinking
bluetooth symbol
indicates results are
transmitting.

IRREGULAR PULSE RATE DETECTION

An irregular pulse rate will be detected if there is an irregular pulse rhythm while measuring systolic and diastolic blood pressure. When measurements were performed, the monitor will record all pulse intervals and calculate the average. If two or more pulse intervals were recorded, and the difference between each interval and the average is larger than $\pm 25\%$ of the average; or if four or more pulse intervals were recorded, and the difference between each interval and the average is larger than $\pm 15\%$ of the average value, the irregular pulse symbol will be displayed along with measurement results.

CAUTION

This monitor will operate if the icon appears, but the results may be compromised, especially if this symbol appears often. Please consult with your physician or trained healthcare professional for further information regarding an irregular pulse rate and if this symbol appears frequently.

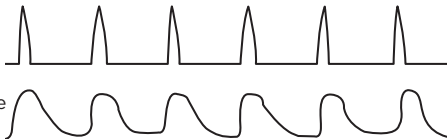
IRREGULAR PULSE RATE RATE DETECTION

Normal Pulse Rate

Pulse

Blood
Pressure


Irregular Pulse Rate

Pulse

Blood
Pressure

Short
Beat

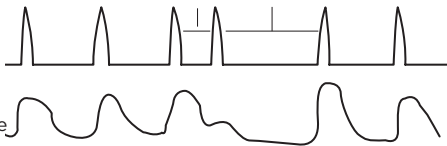Long
Beat

HYPERTENSION INDICATOR

This unit features our unique Hypertension Indicator. The color bars on the right side of the monitor display correspond with an icon on the digital display, indicating where the measurement results fall within the American Heart Association standards.

This chart should only be considered as a guideline; always consult with your physician to interpret your individual results. Never adjust your medication(s) dosage, or severely alter your diet or exercise routine without consulting with your physician.

Category	Systolic (upper #)		Diastolic (lower #)
Hypertensive Crisis	Higher than 180	&/or	Higher than 120
High Blood Pressure, Stage 2	140 or higher	or	90 or higher
High Blood Pressure Stage 1	130-139	or	80-89
Elevated	120-129	&	Less than 80
Normal	Less than 120	&	Less than 80

MEMORY RECALL

RECALLING MEASUREMENTS IN MEMORY:

This monitor stores and recalls up to 60 readings in the device and unlimited in the app. These measurements can be shared with your physician or trained healthcare professional.

1. Press and release the **M** (memory) button. The unit will show the average of the most recent 3 readings in memory, Fig 1. If less than 3 readings are available, the most current will display.

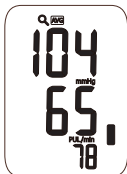
Fig. 1

2. Press the **M** button again to scroll through the saved measurements. Measurements will appear on the display from most current to oldest.
3. All results for a given measurement will display, including measurement results, pulse rate, hypertension Indicator, Irregular pulse rate alert (if applicable), and date/time stamp, Fig 2.

Fig. 2

MEMORY RECALL

4. When the number of readings exceeds 60, the oldest data will be replaced with the new record.
5. Press **START** to turn the monitor off at any time during review of the stored measurements.

CLEARING MEASUREMENTS FROM MEMORY:

To delete all results in memory:

1. Press and release the **M** (memory) button to enter memory recall mode.
2. Press and hold the **M** button; the display will show 'dEL All'.
3. Press and release the **SET** button to confirm.
4. The display will show dEL confirming all results have been deleted.

Deleting measurements on the device will not delete measurements already synced to the APP. Measurements can be deleted from the APP separately.

CARE & MAINTENANCE

Proper care and maintenance of your blood pressure monitor will help ensure long life for the product; improper use of the monitor may void the included warranty.

Monitor Care and Maintenance

- Do not use any liquids on the monitor; use a soft, dry cloth to clean the monitor as needed.
- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: $-4^{\circ}\text{F} - 140^{\circ}\text{F}$ ($-20^{\circ}\text{C} - 60^{\circ}\text{C}$); Humidity: $\leq 93\%$ RH, Atmospheric pressure: 500 hPa - 1060 hPa
- Avoid high temperatures and direct sunlight.
- Always remove the batteries if the unit will not be used for an extended period of time.
- Do not attempt to disassemble your monitor; disassembling your monitor may void the manufacturers warranty.
- Do not subject the monitor to strong shocks; take care not to drop the monitor.
- No component can be maintained by user in the monitor. The circuit diagrams, component part lists, descriptions, calibrations instructions, or other

CARE & MAINTENANCE

information which will assist the user's appropriately qualified technical personnel to repair those parts of equipment which are designated repairable can be supplied.

Cuff Care and Maintenance

- Never attempt to iron or dry the cuff; if the cuff becomes wet, allow the cuff to air dry completely before use or storage.
- It is recommended the performance should be checked every 2 years or after a repair. Please Contact Customer care at 1-866-326-1313.
- It is recommended to clean the cuff every 200 uses.
- The monitor can maintain the safety and performance characteristics for minimum of 10,000 measurements of three years.
- The cuff integrity is maintained after 1,000 open- close cycles of the closure.

Cleaning

1. Before cleaning, ensure monitor is off and disconnected from any external power sources.
2. Clean the cuff with a soft dampened cloth with mild soapy water until no visible

CARE & MAINTENANCE

contaminants remain.

3. Rinse the cuff, ensuring no cleaning solution remains.
4. Wipe off with a dry cloth and allow to air dry thoroughly before storage or next use.

Disinfection

1. Before disinfection, ensure monitor is off.
2. Disinfect the cuff with a soft dampened with 70% ethyl rubbing alcohol for at least three minutes.
3. Wipe with a clean damp towel; ensuring no disinfecting solution remains
4. Wipe off with a dry cloth and allow to air dry thoroughly before storage or next use.

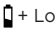
Suggestion

Frequency of Cleaning and Disinfection:

For single patient multiple use, it's recommended to clean the device surface once a month or whenever it's necessary.

For multiple patient multiple use, it's recommended to clean the device every time before and after usage. Maintenance procedures shall be taken as per instruction.

TROUBLESHOOTING

Issue	Meaning	Review	Solution
No Power	Display does not light up	Batteries are exhausted Batteries are inserted incorrectly	Replace with new batteries Check battery polarity
Low Battery	 + Lo	The battery is too low	Replace with new batteries
Error Code	E01	The cuff is not wrapped or wrapped incorrectly	Refasten the cuff and measure again
	E02	Excessive body motion (such as shaking of the arm with the cuff on) or weak pulse is detected during measuring	Relax for 5 minutes, and then keep still, measure again
	E03	Pulse is not detected during measuring	Loosen the clothing on the arm and measure again.
	E04	The measurement failed	Relax for 5 minutes and measure again.
	E12	Data communication has failed	Ensure phone is within range and bluetooth is enabled

TROUBLESHOOTING

Issue	Meaning	Review	Solution
Error Message	EExx	Calibration error occurred	Relax for a moment. Refasten the cuff and measure again. If the problem persists, contact Customer Care.
Warning Message	OUT	Out of measurement range	Relax for a moment. Refasten the cuff and measure again. If the problem persists, contact your physician.

DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Meaning
	Refer to instruction manual/booklet to signify that the instruction manual/booklet must be read. NOTE: Symbol is BLUE
	Date of manufacture
	Direct Current
	Class II Equipment
	Batch Code
	Temperature Limit
	Atmospheric Pressure Limitation
	General symbol for recovery/recyclable
	Type BF Applied Part

DEVICE & LABEL SYMBOLS

Symbol	Meaning
	Serial Number
	For Indoor Use Only
	Humidity Limit
	This symbol shows that this device can be expected to comply with part 15 of the FCC Rules
	MR Unsafe To identify an item which poses unacceptable risks to the patient, medical staff or other persons within the MR environment.
	Caution Indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	The symbol indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.

BLUETOOTH MODULE

BLUETOOTH MODULE INFORMATION

Bluetooth Module information:

RF Frequency Range: 2402 MHz to 2480 MHz

Output Power Range: ≤ 0 dBm

Supply Voltage: 1.8-3.6 V

Transmitting Distance: 10 meters

CAUTION:

- Interference may occur in the vicinity of equipment marked with the following symbol . This device may interfere the vicinity electrical equipment.
- Sensitive people, including pregnant women, including pre-eclamptic, patients and those patients with implanted, electrical devices should avoid using this product whenever possible.
- Keep the monitor at least 20 centimeters away from the human body (especially the head) when the data transmission is proceeding after the measurement.
- To enable the data transmission function, this product should be paired to Bluetooth end at 2.4 GHz.

BLUETOOTH MODULE

How to mitigate possible interference?

1. The range between the device and BT end should be reasonably close, from 1 meter to 10 meters. Please ensure no obstacles between the device and BT end so as to obtain quality connection and to lower the RF output range.
2. To avoid interference, other electronic devices (particularly those with wireless transmission / Transmitter) should be kept at least 1 meter away from the monitor. And BT end so as to obtain quality connection and to lower the RF output range.

FCC STATEMENT

FCC ID: OU9TMB1598B2

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Caution: The user is cautioned that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

-- Reorient or relocate the receiving antenna.

FCC STATEMENT

- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Radiation Exposure Statement:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

ELECTROMAGNETIC COMPATIBILITY

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation."

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment WS-380, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Technical description:

1: all necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life.

2: Guidance and manufacturer's declaration -electromagnetic emissions and Immunity.

Table 1

Guidance and manufacture's declaration - electromagnetic emissions

Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations and flicker IEC 61000-3-3	Not application

ELECTROMAGNETIC COMPATIBILITY

Table 2

Immunity test	IEC60601-1-2 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact; ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact; ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not application	Not application
Surge IEC61000-4-5	Not application	Not application
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not application	Not application
Power frequency magnetic field IEC 61000-4-8	30 A/m 50 Hz / 60 Hz	30 A/m 50 Hz / 60 Hz
Conducted RF IEC61000-4-6	Not application	Not application
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80% AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80% AM at 1 kHz

NOTE: U_i is the a.c. mains voltage prior to application of the test level.

ELECTROMAGNETIC COMPATIBILITY

Table 3

Guidance and manufacture's declaration - electromagnetic emissions

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communication s equipment)	Test Immunity MHz	Band MHz	Service	Modulation	Modulation Power (W)	Distance (m)	Immunity Test Level (V/m)	Compliance Level (V/m)
	385	380-390	TETRA 400	Pulse modulation b) 18Hz	1,8	0,3	27	27
	450	430-470	GMRS 460, FRS 460	FM c) ± 5 kHz deviation 1 kHz sine	2	0,3	28	28
	710	704-787	LTE Band 13,17	Pulse modulation b) 217 Hz	0,2	0,3	9	9
	745							
	780							
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18 Hz	2	0,3	28	28
	870							
	930							
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1,3,4,25; UMTS	Pulse modulation b) 217 Hz	2	0,3	28	28
	1845							
	1970							
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation b) 217 Hz	2	0,3	28	28
5240	5100-5800	WLAN 802.11a/n	Pulse modulation b) 217 Hz	0,2	0,3	9	9	
5500								
5785								

PRODUCT SPECIFICATIONS

Name	Wrist Blood Pressure Monitor
Model Number	302-WS380
Display System	LCD V.A. 45 mm x 33 mm
Dimensions	Approx. 86 mm × 66 mm × 22 mm (excluding the cuff)
Weight	Approx. 100g (Excluding the batteries)
Measuring Mode	Oscillographic testing mode
Mode of Operation	Continuous operation
Measuring Range	Rated cuff pressure: 0 mmHg - 299 mmHg; Measurement pressure: SYS: 60 mmHg - 230 mmHg DIA: 40 mmHg - 130 mmHg; Pulse value: (40-199) beat/minute
Accuracy	Pressure: 5°C - 40°C within ±3mmHg; Pulse value: ±5%
Operating Environment	Temperature: +5°C to +40°C Relative humidity: 15% to 90%, non-condensing, but not requiring a water vapor partial pressure greater than 50 hPa; Atmospheric pressure: 700 hPa to 1060 hPa
Storage Environment	Temperature: -20°C to +60°C; Relative humidity: ≤93%, non-condensing, at a water vapor pressure up to 50 hPa; Atmospheric pressure: 500 hPa - 1060 hPa
Wrist Measurement	5.3" - 8.4" (13.5-21.5 cm)
Degree of Protection	Type BF applied part
Device Classification	Internally Powered ME Equipment
Protection against	IP22 It means the device could be

PRODUCT SPECIFICATIONS

Ingress of water	protected against solid foreign objects of 12,5mm Φ and greater. Protected against vertically falling water-drops when encloser tilted up to 15°. Vertically falling drops shall have no harmful effects when the encloser is tilted at any angle up to 15° on either side of the vertical.
Expected Lifetime	Device: 3 years or 30,000 measurements (may vary based on usage conditions); Cuff: 10,000 uses; Battery: Approximately 200-300 times
Types of use/reuse	Single patient multiple use
Power Source	2x AAA batteries
Software Version	A01
Accessories	Instruction Manual, Quick Start Guide; Two (2) AAA Batteries

Specifications are subject to change without notice

ONE-YEAR LIMITED WARRANTY

Congratulations on your purchase of a Blood Pressure Monitor. Your Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for a period of one year for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Blood Pressure Monitor for commercial purposes or subjecting the Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Blood Pressure Monitor.

If the Blood Pressure Monitor fails to conform to this limited warranty, return the Blood Pressure Monitor postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to

ONE-YEAR LIMITED WARRANTY

avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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measure • track • share

Setting up your health monitor is fast and easy. Simply download the app, prepare your device, and start your health journey. Consult your healthcare professional before starting a new health regime, changing medication and for aid in interpreting your personal results.

1. Download the **Trackstar Wellness** app from the App Store (iOS) or Google Play (Android).
2. Launch the app and follow the instructions for account profile set up.
3. Consult the included instruction manual for detailed instructions, tips, cautions and cleaning direction.