instruction manual

INFRARED THERMOMETER

MODEL 302-11115

Please read this instruction manual completely before operating this unit.


STOP!
PLEASE ENSURE YOU HAVE ALL OF THE
FOLLOWING COMPONENTS BEFORE
USING YOUR INFRARED THERMOMETER

Thermometer

Two (2) AAA
Batteries,
Preinstalled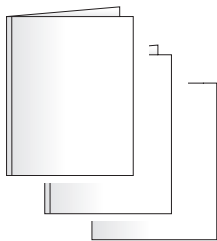

Instruction Manual
Trackstar App Guide

IF YOU ARE MISSING ANY PARTS, INCLUDING
INSERTS OR INSTRUCTION MANUALS, DO NOT
RETURN TO PLACE OF PURCHASE. CONTACT
CUSTOMER CARE AT 866-326-1313.

INDEX

Cautions & Warnings	4-5
Introduction	6
Body Temperature & Fever	7
Body Temperature Measuring Tips.....	8
Product Features	9
Device & Display Symbols.....	10
Trackstar Wellness App	11
Battery Installation	12
Switching Between °F / °C	12
Measuring by Ear	13-14
Measuring by Forehead.....	15-16
Measuring Object.....	17
Care & Safety Information.....	18
Cleaning & Storage.....	19
Troubleshooting/Error Codes.....	20-21
Device & Label Symbols.....	22
FCC Statement	23
Electromagnetic Compatibility.....	24-26
Product Specifications.....	27-28
Warranty	29
Instruccionnes en Espanol	31-59

Toll-Free
Customer Care Help Line:
1-866-326-1313
Monday – Friday
8:30 a.m. – 4:30 p.m. CST
www.veridianhealthcare.com

Distributed by:
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com

Made in China
#93-1684 11/25
©2025 Veridian Healthcare, LLC

CAUTIONS AND WARNINGS

NOTE: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

CAUTION: Failure to read and observe all precautions could result in personal injury or equipment damage.

GENERAL CAUTIONS AND WARNINGS

- Use as directed in this manual; misuse may void the warranty.
- This thermometer may be used for professional purposes or for personal home use.
- Measurement results should be discussed with your physician or healthcare professional; never self-diagnose, attempt treatment or alter medications this may be dangerous.
- Do not use the thermometer if the ear is infected with otitis or suppuration.
- The patient is the intended operator of this device.
- Wait approximately 30 minutes before measurement after strenuous physical activity, bathing, spending time outdoors or other activity which may artificially affect temperatures.
- Operation outside of stated operating temperature may result in measurement error or device malfunction; operation environment temperature is 50°F-104°F (10°C- 40°C); Humidity: 15-85%RH (non-condensing).
- If the thermometer is stored outside of operating temperature range, allow at least 30 minutes in correct range before taking a measurement.
- Keep in a clean, dry area; do not store the thermometer in direct sunlight, dusty or humid environments, or extreme temperatures.
- Clean the lens before use; do not touch the lens directly with your fingers.

CAUTIONS AND WARNINGS

- Do not expose this thermometer to electric shocks.
- Do not drop this thermometer.
- Follow the included cleaning and storage instructions; do not submerge the thermometer or wet.
- Do not use this thermometer if it is damaged.
- Do not disassemble the unit or attempt to repair it; substitution of a component different from that supplied may result in measurement error and will void manufacturer warranty.
- Remove the batteries if you do not wish to use the unit for an extended period of time.
- Do not attempt to service the ME equipment while in use with a patient.
- Portable and mobile RF communications can affect the device. The device needs special precaution regarding EMC according to the EMC information provided in the accompany documents ME equipment should not be cleaned and disinfected while in use.
- Keep the battery away from children. Remove battery from the device when not in operation for a long time.

INTRODUCTION

Thank you for purchasing this Thermometer. Please read this instruction manual completely before using your unit to ensure safe use, accurate results and long life of the thermometer.

Thank you for choosing our Infrared Thermometer. The infrared thermometer can be used to read the body temperature by measuring the ear and forehead temperature, which is suitable for adult and children (ear test mode only for the child above 3 months). Please read the instructions carefully before using the product and put it in a safe and secure place for reference.

INTENDED USE The Infrared Thermometers take human body temperature via the eardrum or forehead for both professional and home use.

EAR THERMOMETERS (also known as tympanic or aural thermometers), are a great alternative to oral, rectal and underarm thermometers. The ear is an excellent site for measuring temperature, as the part of the brain that regulates body temperature shares a blood supply with the eardrum. This thermometer provides results several times faster than the fastest oral thermometer, making it ideal for infants and fidgety children. It can even be used while the patient is sleeping. Unlike readings from oral thermometers, factors such as smoking, drinking, or speaking do not affect the measurements taken by ear thermometers.

FOREHEAD THERMOMETERS take the temperature of people and objects with no direct contact with the patient, making it less invasive and threatening than other thermometers, as well as potentially more sanitary. It can even be used when the patient is sleeping. This thermometer also measures the temperature of objects and liquids, making it useful for determining if a baby's milk or bathwater is ready.

All people and objects emit infrared energy. The forehead thermometer measures that energy, taking multiple readings quickly and converting those data to a single peak temperature, which appears on the display.

BODY TEMPERATURE AND FEVER

Body temperature can vary from 97°F(36.1°C) and 99°F(37.2°C) and still be considered normal. The medically accepted 'normal' body temperature is 98.6°F(37°C). Body temperature is commonly lower upon waking than at any point during the rest of an individual's active hours.

A fever is defined as a body temperature that is elevated above the normal for that person. It is important to determine what is normal for an individual before determining if a fever is present. Tracking an individual's temperature on a consistent basis, at the same body site and at the same time every day will help determine an individual's normal temperature.


BODY TEMPERATURE MEASURING TIPS

- Wait at least one minute between consecutive readings, to ensure accuracy.
- Attempting to take temperature readings from sites on the body other than the forehead or ear may produce inaccurate results.
- The patient should remain still while the reading is being taken.
- If an error message appears, see the Troubleshooting section of this manual for instructions on how to resolve the problem.
- Readings taken while asleep should not be compared directly to readings taken while awake, as body temperature while asleep is typically lower.
- Do not take body temperature readings within 30 minutes of being outdoors, exercising or bathing.
- Do not take temperatures while outdoors; only use indoors
- Level of activity, time of day, and other factors may affect body temperature. Keep records of your individual body temperature to use as a reference when judging a fever.

PRODUCT FEATURES

DEVICE & DISPLAY SYMBOLS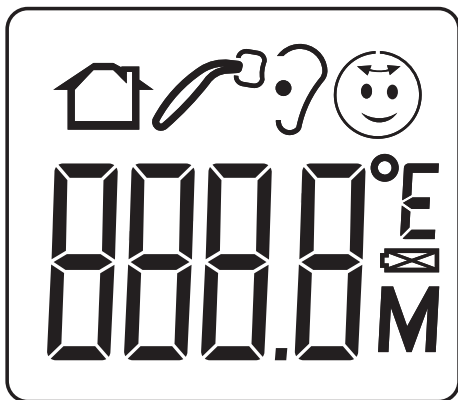

FOREHEAD
MODE

MEMORY
MODE

EAR MODE

READY TO
MEASURE

OBJECT
TEMPERATURE
MODE

BATTERY
STRENGTH

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**DETAILS OF THE DIGITAL DISPLAY SCREEN AND
INTERPRETATION OF THE SYMBOLS ARE INCLUDED
THROUGHOUT THIS MANUAL.**

TRACKSTAR WELLNESS APP

Installing the Trackstar Wellness app is fast and easy.

Simply download the app, prepare your device, and start your health journey. Consult your healthcare professional before starting a new health regime, changing medication and for aid in interpreting your personal results.

1. Download the **Trackstar Wellness** app from the App Store (iOS) or Google Play (Android).
2. Launch the app and follow the instructions for account profile set up.
3. Pair your thermometer following the on-screen directions.

THIS THERMOMETER OPERATES WITH OR WITHOUT THE TRACKSTAR APP. SOME FEATURES, SUCH AS MEMORY STORAGE AND RECALL, CAN ONLY BE ACCESSED THROUGH THE APP.


BATTERY INSTALLATION

THIS UNIT INCLUDES 2 AAA BATTERIES, PREINSTALLED.

When using for the first time, remove the pull strip to activate the batteries.

It is necessary to replace the batteries when the low battery symbol  appears on the display or the thermometer will not power on.

1. Slide open the battery compartment on the backside of the device.
2. Insert or replace 2 AAA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

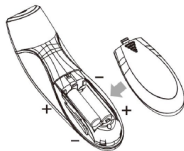It is recommended to remove the batteries if the unit will not be used for an extended period of time.

SWITCHING BETWEEN °F / °C

Your thermometer can display results in either degrees Fahrenheit (°F) or degrees Celsius (°C).

1. To switch between Fahrenheit and Celsius, while the unit is OFF press and hold the **EAR** button.
2. The thermometer will display M in the lower right corner; continue to HOLD the **EAR** button until the °F (°C) flashes.
3. Press and release **EAR** to toggle between °F and °C. When desired mode is shown, allow the thermometer to auto shut-off and the mode is confirmed.

MEASURING TEMPERATURE BY EAR ?

NOTE:

Proper placement and position of the probe is crucial for effective and accurate measurement at the ear.

The ear canal should be 'straight' in order for the sensor probe to work most effectively. Gently pull the ear slightly up and away from the head with the opposite hand that is holding the thermometer.

- For children under one year or who cannot sit on their own, lay the child flat and turn the head sideways, then gently pull the ear and place the probe.
- For older children and adults, stand at the shoulder or elbow of the seated patient and pull back the ear and place the probe.

To measure body temperature at the EAR:

1. Remove the cap from the tip of the thermometer
2. Press and release the **EAR** button to turn the thermometer on.
3. The display will illuminate and show three dashes and the ear symbol indicating it is ready for measurement symbol, Fig 1.
4. Gently insert the probe into the ear canal until a slight resistance is felt.

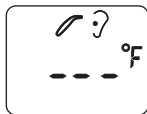
Fig. 1

MEASURING TEMPERATURE BY EAR ?

5. Press and release the **EAR** button; you will hear an audible beep. Do not remove the thermometer until you hear the beep.
6. The display will show the measurement results, Fig 2.
7. The LED fever alert light will illuminate, indicating where the results fall in the fever alert range. The thermometer will emit one long beep if the results indicate no fever, and 3 short double beeps if a fever is present.
8. After 10-seconds of non-use, the thermometer will automatically shut off.

Fig. 2

FEVER ALERT TEMPERATURE RANGE:

GREEN LIGHT— If the results below 99.5°F (37.5°C) indicating no fever is present

RED LIGHT —If the results are 99.6 °F (37.6°C) and above, a fever is considered present.

MEASURING FOREHEAD TEMPERATURE

To measure body temperature at the forehead:

1. The cap should be in place during forehead measurements. The hole in the cap allows the sensor to work effectively
2. Press and release the **HEAD** button to turn the thermometer on.
3. The display will illuminate and show three dashes and head symbol indicating it is ready for measurement symbol, Fig 1.
4. With the sensor/probe cover attached, position the thermometer at the center of the forehead, just above the eyebrow, with a distance of approximately .5-1" (1-3cm) from the skin surface.


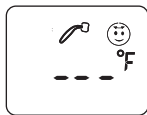
Fig. 1

NOTE:

Proper placement and position of the probe is crucial for effective and accurate measurement.

The forehead should be clean and clear of excessive lotion or oil.

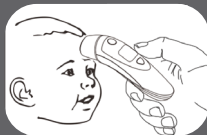5. Press and release the **HEAD** button; you will hear an audible beep. Do not remove the thermometer until you hear the beep.

MEASURING FOREHEAD TEMPERATURE

6. The display will show the measurement results, Fig 2.
7. The LED fever alert light will illuminate, indicating where the results fall in the fever alert range. The thermometer will emit one long beep if the results indicate no fever, and 3 short double beeps if a fever is present.
8. After 10-seconds of non-use, the thermometer will automatically shut off.

Fig. 2

FEVER ALERT TEMPERATURE RANGE:

GREEN LIGHT— If the results below 99.5°F (37.5°C) indicating no fever is present

RED LIGHT —If the results are 99.6 °F (37.6°C) and above, a fever is considered present.

MEASURING OBJECT/AMBIENT TEMPERATURE

This thermometer can measure objects and liquids from 32°F to 212°F (0°C - 100°C). To measure the temperature of an object or liquid, follow these steps:

1. The cap should be in place during object measurements, Fig 1. The hole in the cap allows the sensor to work effectively
2. Press and release the **HEAD/OBJECT** button to turn the thermometer on.
3. The display will illuminate and show three dash measurement symbol, Fig 1.
4. Aim the infrared lens at the object being measured, holding the thermometer approximately .5-1" (1-3cm) away.
5. Press and HOLD the **HEAD/OBJECT** button for 3 seconds to enter the object temperature mode. The object temperature reading will be displayed on the screen instantly. Press and release Head button again to measure the next object temperature before the thermometer is turned off, Fig. 2.
6. After 10-seconds of non-use, the thermometer will automatically shut off.

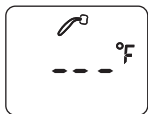
Fig. 1

Fig. 2

CARE & SAFETY INFORMATION

Improper care or use of your thermometer may result in injury, damage to the unit or inaccurate readings, which can lead to a misdiagnosed fever. Following these instructions will ensure the thermometer's accuracy and long life.

- Never allow the thermometer to be used without adult supervision.
- The infrared technology used in this thermometer is safe and is not harmful in any way. However, we recommend avoiding looking into the lens during use.
- Clean the lens before and after each use.
- Do not touch the lens with fingers.
- Inspect the thermometer for damage or cracks prior to use.
- Keep the protective cap in place when not in use.
- Dropping the thermometer or causing other heavy impact may damage it.
- Do not store the thermometer in direct sunlight, dusty or humid environments, or extreme temperatures.
- Do not disassemble the thermometer or attempt to repair it.
- If a reading cannot be taken, and the troubleshooting steps do not fix the problem, the thermometer must be replaced.

CLEANING & STORAGE

The lens should be cleaned before and after each use, to eliminate dirt and fingerprints and ensure the accuracy of your readings. Proper storage will prevent degradation and damage to the unit.

CLEANING:

- An alcohol swab, or a soft cotton cloth moistened with 70% isopropyl alcohol, should be used to gently cleanse the lens. Do not use water to wash the thermometer.
- The body of the thermometer can be cleaned with a soft, dry cloth.
- Never use a paper towel to clean the probe tip, as it may scratch the lens.
- Do not immerse the thermometer or allow liquid to enter the casing.
- Do not use cleaning agents other than alcohol to clean the thermometer; abrasives, thinners or gasoline will damage the unit.

STORAGE:

- Store the unit in an area away from sunlight, dust and humidity.
- Protect the thermometer from mechanical shock or heavy impact.
- Avoid extended exposure to extreme temperatures. The thermometer should be stored in an environment between -4°F and 122°F (-20°C—50°C), and with relative humidity of 15-95%.
- Remove the batteries during extended storage.

TROUBLESHOOTING / ERROR CODES

Performance may be degraded should one or more of the following occur:

- Operation outside stated temperature and humidity range.
- Storage outside state temperature and humidity range.
- Mechanical shock.
- Patient temperature is below ambient temperature.

The following chart indicates error messages that may appear on your display, why they appear, and how you may solve the problem.

Error	Problem or Cause	Solution
Thermometer fails to power on.	Battery level is low / battery is dead.	Replace batteries with NEW batteries.
	Battery polarity is incorrect.	Correct battery polarity.
Low battery symbol appears when powered on	Battery level is too low to operate.	Replace batteries with NEW batteries.
Error Code: Er1	Ambient temperature is lower than 10°C (50.0°F) or higher than 40°C (104°F).	Allow thermometer to come within temperature range and try again.
Error Code: ErC	Memory data reading error or correcting process is not finished.	Contact Customer Care.

TROUBLESHOOTING / ERROR CODES

Error	Problem or Cause	Solution
Results are lower than typical temperature range.	Probe lens is dirty.	Clean lens following cleaning instructions.
	Probe is too far from target.	Read instructions for proper positioning during measurement.
	Thermometer has not been allowed to come to room temperature from cold environment.	Wait for more than 30 minutes after the thermometer is moved into the measurement environment.
Results are higher than typical temperature range.	Probe is malfunctioning.	Contact Customer Care.
	Results above 212.0°F (100°C).	Ensure object being measured falls within acceptable range.
	Results below 32.0°F (0°C).	Ensure object being measured falls within acceptable range.

DEVICE AND LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Explanation	Symbol	Explanation
	Caution		Atmospheric Pressure Limitation
	Direct Current		Storage / Transport Humidity Limit; 15% ~ 95% RH
	Batch Code		Date of Manufacture
	Storage / Transport Temp Limit; -4°F-122°F -20°C-50°C)	 IP22	Manufacturer
	Type BF Applied Parts		Degree of protection against harmful ingress of water and particulate matter; 2 Protected and against solid foreign objects of 12.5mm and greater; 2 Protection against vertically falling water drops when ENCLOSURE tilted up to 15°
	Read This Manual		
	Recycling Symbol		
	DISPOSAL: Do not dispose of this product as unsorted municipal waste. Collection of such waste separately for special treatment is necessary.		

FCC STATEMENT

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Notes:

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications to this device not explicitly approved by manufacturer could void your authority to operate this equipment.

ELECTROMAGNETIC COMPATIBILITY

EMC Information-Guidance and Manufacture's Declaration

1* WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally."

2* WARNING: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation."

3* WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the ME equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result."

Table 1

Guidance and manufacture's declaration - electromagnetic emissions

Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations and flicker IEC 61000-3-3	Not application

ELECTROMAGNETIC COMPATIBILITY

Table 2

Declaration - Electromagnetic Immunity

Immunity test	IEC60601-1-2 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Not applicable
Surge IEC 61000-4-5	± 0.5kV, ± 1 kV line(s) to lines ± 0.5kV, ± 1 kV, ± 2 kV line(s) to earth	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0° 0 % UT; 250/300 cycles	Not applicable
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m

NOTE: U_i is the a.c. mains voltage prior to application of the test level.

Table 3

Declaration - Electromagnetic Immunity

Immunity test	IEC 60601 test level	Compliance level
	3 V 0.15 MHz to 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz	Not applicable
	10V/m 80 MHz to 2.7 GHz	10V/m

ELECTROMAGNETIC COMPATIBILITY

Table 4

Declaration - IMMUNITY to proximity fields from RF wireless communications equipment

Immunity test	IEC60601 test level				Compliance level
	Test frequency	Modulation	Maximum Power	Immunity level	
Radiated RF IEC 61000-4-3	385 MHz	**Pulse Modulation: 18Hz	1.8W	27 V/m	27 V/m
	450 MHz	*FM+ 5Hz deviation: 1kHz sine	2 W	28 V/m	28 V/m
	710 MHz 745 MHz 780 MHz	**Pulse Modulation: 217Hz	0.2 W	9 V/m	9 V/m
	810 MHz 870 MHz 930 MHz	**Pulse Modulation: 18Hz	2 W	28 V/m	28 V/m
	1720 MHz 1845 MHz 1970 MHz	**Pulse Modulation: 217Hz	2 W	28 V/m	28 V/m
	2450 MHz	**Pulse Modulation: 217Hz	2 W	28 V/m	28 V/m
	5240 MHz 5500 MHz 5785 MHz	**Pulse Modulation: 217Hz	0.2 W	9 V/m	9 V/m

Note* - As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

Note** - The carrier shall be modulated using a 50 % duty cycle square wave signal.

PRODUCT SPECIFICATIONS

ITEMS	STANDARDS
Model	302-11115 Infrared Thermometer
Temperature units	°C/°F, adjustable
Measurement	Forehead temperature mode: 32.0°C–42.2°C (89.6°F–108.0°F) Ear mode: 32.0°C–42.2°C (89.6°F–108.0°F) Object mode: 0.0°C–100.0°C (32.0°F–212.0°F)
Operating Environment	Temperature: 10°C~ 40°C (50°F–104°F) Humidity: 15%–85% RH, non-condensing Atmospheric pressure: 86–106 kPa
Storage/ Transportation Environment	Temperature: -20°C to 50°C (-4°F–122°F) Humidity: 15– 95% RH, non-condensing Atmospheric pressure: 50–106 kPa
Accuracy (Laboratory)	Forehead mode: $\pm 0.2^{\circ}\text{C}/\pm 4.0^{\circ}\text{F}$ Ear mode: $\pm 0.2^{\circ}\text{C}/\pm 4.0^{\circ}\text{F}$; Object mode: $\pm 1.0^{\circ}\text{C}/\pm 2.0^{\circ}\text{F}$
Accuracy (Clinical)	$\pm 0.3^{\circ}\text{C}$ ($\pm 0.6^{\circ}\text{F}$)
Display resolution	0.1°C (0.1°F)
Measuring site	Ear canal, Forehead (keep distance 1~3 cm from forehead)
Mode of operation	Adjusted mode
Measure time	About 1 second
Latency Time	About 1 second
Measuring distance	Forehead temperature: 1-3cm
Sound	Volume ≥ 50 db (the perpendicular distance from dB Volume sensor to thermometer is 10cm)

PRODUCT SPECIFICATIONS

ITEMS	STANDARDS
Automatic	The thermometer automatically powers off if it is not shutdown function used in 10 ± 1 seconds.
Low-voltage display function	The product shall display low-voltage signal if the voltage is below $2.5V \pm 0.1V$.
Memory function	Memorize 20 groups of measured temperature.
Power supply	Internal power supply
Operating voltage	DC 3V
Battery	AAA $\times 2$
Life	2 years
Battery life measurements	Alkaline dry battery for around 20000
Reference Body Site	Armpit

ONE-YEAR LIMITED WARRANTY

The warrantor guarantees that its Thermometer will be free from manufacturing defects for a period of one year from the date of purchase. This warranty does not cover batteries or other power sources that may be provided with or used with the Infrared Thermometer. This warranty is voided if the Thermometer is misused or abused in any manner.

If the Thermometer fails to operate during the time the original purchaser owns it, return it postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include your name, address, phone number and brief description of the specific problem with your unit. Take care to carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. The warrantor will repair or replace the defective unit at our option. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only responsibility and obligation to repair and/or replace materials or components, or refund the purchase price. The warrantor will not be responsible for any indirect, incidental, special consequential or punitive damages or other loss, including, but not limited to damage to or loss of other property or liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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measure • track • share

Setting up your health monitor is fast and easy. Simply download the app, prepare your device, and start your health journey. Consult your healthcare professional before starting a new health regime, changing medication and for aid in interpreting your personal results.

1. Download the **Trackstar Wellness** app from the App Store (iOS) or Google Play (Android).
2. Launch the app and follow the instructions for account profile set up.
3. Consult the included instruction manual for detailed instructions, tips, cautions and cleaning direction.