GETTING STARTED

Consult included manual for more instructions, warnings & cautions

WRIST BLOOD PRESSURE MONITOR

MODEL 302-WS380

TIPS FOR SUCCESSFUL BLOOD PRESSURE MONITOR USE

- Take care when applying wrist cuff; a too tight or loose cuff during measurement may result in inaccurate measurements.
- If possible, take the readings at the same time each day and avoid eating or drinking prior to measuring.
- Wait 20 minutes between readings before attempting a second measurement.
- If your wrist is outside of the range printed on the wrist cuff, consult your healthcare provider for advice on how best to monitor your blood pressure.


GETTING STARTED

- Follow the included paperwork for pairing with the Trackstar Wellness App before use.
- Apply wrist cuff following instructions on the following page.
- Be seated with your feet flat on the floor with arm resting on a table and device resting on a flat surface.
- Never eat or talk during measurements.
- When ready, press **START / STOP** button to begin measurement; monitor will automatically inflate and deflate, taking your measurement and storing results in app history.
- Review the included app and manual for aid in interpreting your results. Variations in results should be discussed with your healthcare professional.

CUFF APPLICATION


Wrist Cuff fits arm circumference 5.3" – 8.4" (13.5 cm – 21.5 cm)

- Remove any constrictive clothing or jewelry that may interfere with cuff placement.
- Apply the cuff to your left wrist so that the digital display face is positioned on the inside area of your wrist facing you.
- Adjust the cuff approximately 0.5" from the edge of the head of the ulna bone.
- The cuff should fit comfortably, yet snugly around your wrist.
- If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.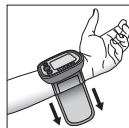

BODY POSITION

- Be seated at a table or desk with your feet flat on the floor.
- Hold your left arm in front of you with your palm facing upward.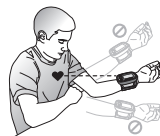


MEASUREMENT

- Press **START/STOP** button to power on; measurement will begin automatically.
- Monitor will inflate and deflate, taking your measurement and storing results in measurement history. This should take approximately one minute.


RESULTS & PAIRING

- If the APP is open, the monitor will automatically attempt to pair with your app while active. If the APP is not open, readings will store and sync the next time the device and app are connected.
- Remove cuff; the monitor will automatically shut off after one minute of non-use.

SIMPLE STEP INSTRUCTIONS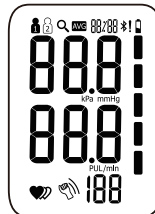

- Press **START/STOP**; test display will show

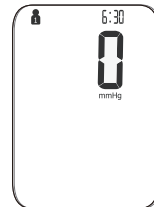- "0" will briefly show before measurement begins

- As cuff inflates, the numbers will increase on the display

- Results will show on the display; the blinking bluetooth symbol indicates results are transmitting.

Toll-Free Customer Care Help Line: **1-866-326-1313**
Monday – Friday **8:30 a.m. – 4:30 p.m. CST**
www.veridianhealthcare.com

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