
HealthWise™

# Cool Mist Ultrasonic Humidifier

Model 59-526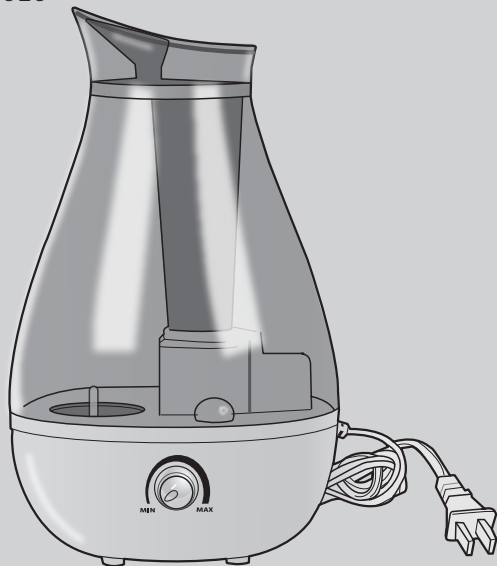

**Please read this instruction manual  
completely before operating this unit.**

**INSTRUCTION MANUAL • ENGLISH & ESPAÑOL**

**STOP!**  
**PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING COMPONENTS BEFORE USING YOUR HUMIDIFIER.**

Humidifier Unit

Instruction Manual

**IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.**

---

## **INTRODUCTION**

---

Thank you for purchasing this Cool Mist Ultrasonic Humidifier. **PLEASE READ THIS INSTRUCTION MANUAL COMPLETELY BEFORE USING YOUR UNIT TO ENSURE SAFE AND EFFECTIVE USE, AS WELL AS LONG LIFE OF THE PRODUCT.**

### **HUMIDIFIER USE TIPS**

- To maintain and prolong the lifespan of the humidifier, only use room temperature water. Never use hot or ice water.
- It is recommended to use distilled or purified water or room temperature water.
- Change / empty water daily to prevent mold and bacteria growth.
- Never add essential oils in the water tank of the humidifier; oils can damage the humidifier.
- Humidifiers should be utilized on a flat, level, and firm surface, with a minimum of five inches away from walls or any heat source.

---

## INDEX

---

Introduction .....	2
Important Safeguards - Care & Safety Information .....	4-5
Product Features .....	6
Instructions For Use .....	7
Cleaning, Maintenance & Storage .....	8-9
Troubleshooting .....	10
Product Specifications .....	11
Warranty .....	12
Instrucciones en Español.....	13-24

---

**Toll-Free Customer Care Help Line: 1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

HealthWise™  
Manufactured for:  
Veridian Healthcare, LLC  
1175 Lakeside Drive  
Gurnee, IL 60031 USA  
Made in China  
#93-1549 04/22  
©2022 Veridian Healthcare, LLC

---

# IMPORTANT SAFEGUARDS

## CARE & SAFETY INFORMATION

---

**NOTE:** Read all instructions carefully before use. The following basic precautions are needed when using an electrical product. **Failure to read and observe all precautions could result in personal injury or equipment damage.**

**Improper care or use of your humidifier may result in injury, damage to the unit or ineffective mist. Following these instructions will ensure the humidifiers efficacy and long life.**

### GENERAL CAUTIONS AND WARNINGS

- Keep out of reach of children.
- Inspect humidifier before first use and before each subsequent use; do not use the unit if it has any damaged parts (including power cord or plug), if it has been submersed in water or dropped.
- Read all instructions carefully before using the appliance.
- Before connecting the appliance, check if the voltage indicated at the bottom of the appliance corresponds to the local mains voltage before you connect the appliance.
- DO NOT operate any appliance with a damaged cord or plug, or after the appliance malfunctions, or has been damaged in any manner. If the mains cord is damaged, contact your service provider immediately.
- To protect against electrical hazards, DO NOT immerse cord, plugs, or the humidifier base in water or other liquid.
- DO NOT let cord hang over the edge of a table or counter, and do not let it touch hot surfaces.
- Regularly check the main cord if it is still in good and safe condition.
- To avoid circuit overload, do not operate another high voltage appliance on the same circuit.

---

# IMPORTANT SAFEGUARDS

## CARE & SAFETY INFORMATION

---



- Unplug from outlet when not in use and before cleaning.
- Turn the switch to "MIN/OFF" before connecting or disconnecting plug from power outlet.
- Ensure that the appliance is unplugged from the outlet when refilling water.
- Do not place the appliance on or near a hot gas or electric burner.
- Appliance should always be used on a dry and level surface.
- Close supervision is necessary when the appliance is used and near children.
- This appliance is not intended to be used by persons (including children) with reduced physical, sensory and mental capabilities. This appliance is not for persons with lack of experience and knowledge, unless they have been given supervision or instructions concerning the use of the appliance by a person responsible for their safety.
- Never add essential oils in the water tank of the humidifier. The oils can damage the humidifier.
- Do not use outdoors or for purposes other than intended use.
- DO NOT leave unit unattended while in use.
- The device is not intended to be operated with an external timer or separate remote control system.


### **CLEANING CAUTIONS AND WARNINGS**

- Never immerse the unit in water to clean as it may damage the unit.
- Disconnect the unit from the electrical outlet before cleaning.
- Only use included attachments and parts; do not use attachments from other brands or models.
- Do not disassemble the unit or attempt to repair it.
- Clean all necessary parts after each use as instructed in this instruction manual; if the unit has not been used for an extended period, all parts should be cleaned prior to use.

# PRODUCT FEATURES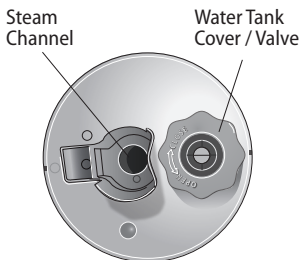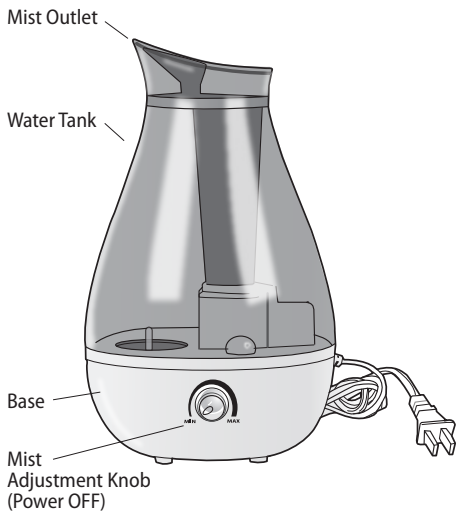

*Water Tank Underside View*

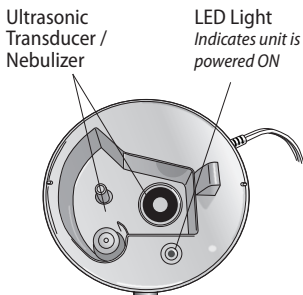*Base Top View*

**DETAILS OF INTERIOR PARTS ARE INCLUDED THROUGHOUT THIS MANUAL.**



---

## INSTRUCTIONS FOR USE

---

Place the humidifier on a flat, hard surface within reach of an outlet. Ensure the unit is off and not plugged in until ready to use.

1. Remove the water tank, turn upside down and unscrew the water tank cover.
2. Fill the water tank with room temperature water to approximately 1/4" from the opening.
3. Replace the water tank cover and install the water tank onto the humidifier base.
4. Plug in the unit, shift the unit as desired to aim the mist in the desired direction and turn the mist adjustment knob to the desired level. The LED light will illuminate indicating the unit is ON.
5. The water tank will automatically fill the base, and the mist action will start shortly.
6. Adjust the level of mist as desired.
7. When the water tank is empty, the humidifier will stop automatically and the LED light will turn OFF. Always turn the knob to MIN / completely OFF before refilling or cleaning and storing.

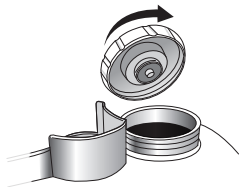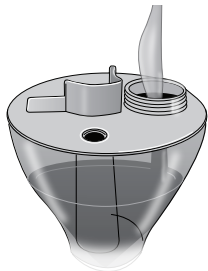### NOTE:

Tap water quality varies; it is suggested that distilled or purified water is used in this humidifier. Always use room temperature water.

---

# CLEANING, MAINTENANCE & STORAGE

---

The ultrasonic humidifier should be thoroughly cleaned daily after use, and weekly when being used frequently.

- Unplug the humidifier before cleaning and storage.
- Never allow unused water to sit in the water tank or base for an extended period.
- If the mist or humidifier produces an unpleasant smell, stop use and wash immediately before attempting use again.
- Do not submerge the base nor allow to be held under running water.

## **Daily Cleaning Procedure / After Use:**

1. Ensure that the power is set to MIN (OFF) and the power cord is unplugged.
2. Remove water tank and empty completely. Empty any remaining water in base.
3. Wipe all components with a soft, dry cloth – tank, cap, base and interior of base including the nebulizer and transducer.
4. Do not use detergents or soaps.
5. Hand dry or allow to dry completely before storage.

## **Weekly Cleaning Procedure:**

1. Remove water tank and empty completely. Empty any remaining liquid in base.
2. Prepare a solution of 50% white vinegar and 50% water.
3. Replace cap and return water tank to base; this will allow the base to fill up with the vinegar solution and allow the 'base' to soak.
4. **DO NOT TURN THE HUMIDIFIER ON.**
5. Allow to soak a minimum of 4 hours, overnight is suggested.

---

## CLEANING, MAINTENANCE & STORAGE

---

6. Empty the solution and thoroughly rinse the water tank.
7. Wipe all components with a soft, dry cloth – tank, cap, base and interior of base including the nebulizer and transducer.
8. Hand dry or allow to dry completely before storage.

**NOTE:** Humidifier parts are NOT dishwasher safe.

---

# TROUBLESHOOTING

---

## **WATER IS LEAKING**

- Check that the secondary tank is not overfilled and is at the proper fill level.
- Make sure the unit is on a flat surface and not tilted.

## **HUMIDIFIER DOES NOT POWER ON; INDICATOR LIGHT DOES NOT ILLUMINATE**

- Confirm the unit is plugged in.
- Check the knob is ON; turn toward MAX.

## **INDICATOR LIGHT IS ON, BUT NO MIST OUTPUT**

- Check for hard water / mineral deposits in the base and on transducer; clean following the included instructions and attempt use again.
- Check there is sufficient water in the tank and base.

## **STRANGE ODOR**

- Clean the water tank and base following the included instructions and allow to air dry for a minimum of 12 hours.

---

## PRODUCT SPECIFICATIONS

---

<b>NAME</b>	Ultrasonic Cool Mist Humidifier
<b>MODEL NUMBER</b>	59-526
<b>POWER CONSUMPTION</b>	120V, 60Hz, 22W
<b>TANK CAPACITY</b>	0.66 gal.
<b>RECOMMENDED ROOM SIZE</b>	Medium to Large
<b>AVERAGE HUMIDITY OUTPUT</b>	7.1 fl/hour; <i>output depends on room temperature, humidity and mist output setting</i>
<b>RUN TIME</b>	Up to 24 hours (min. setting); Up to 10 hours (max setting)
<b>SOUND LEVEL</b>	<35 dBA
<b>UNIT DIMENSIONS</b>	11-3/4" x 6-7/8" x 6-7/8" (excluding power cord)
<b>UNIT WEIGHT</b>	2 lb. 4 oz.

*Specifications are subject to change without notice.*

## SAVE THESE INSTRUCTIONS

---

## 2-YEAR LIMITED WARRANTY

---

The warrantor guarantees that its Ultrasonic Humidifier will be free from manufacturing defects under normal use for a period of two-years. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty is voided if the unit is misused or abused in any manner.

If the humidifier fails to operate within the warranty period while under the original purchaser owns the product, return it postage prepaid to: **Attn: Repair Department, 1175 Lakeside, Gurnee, IL, 60031**; include your name, address, phone number and brief description of the problem with your humidifier. Take care to carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. The warrantor will repair or replace the defective unit at our option. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

**ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT.** Some states do not allow limitations or how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only reasonability and obligation to repair and/or replace materials or components, or refund the purchase price. The warrantor will not be responsible for any indirect, incidental, special consequential or punitive damages or other loss, including, but not limited to damage to or loss of other property or liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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