
SMARTHEART™

Body Composition Digital Scale



Model # 19-102

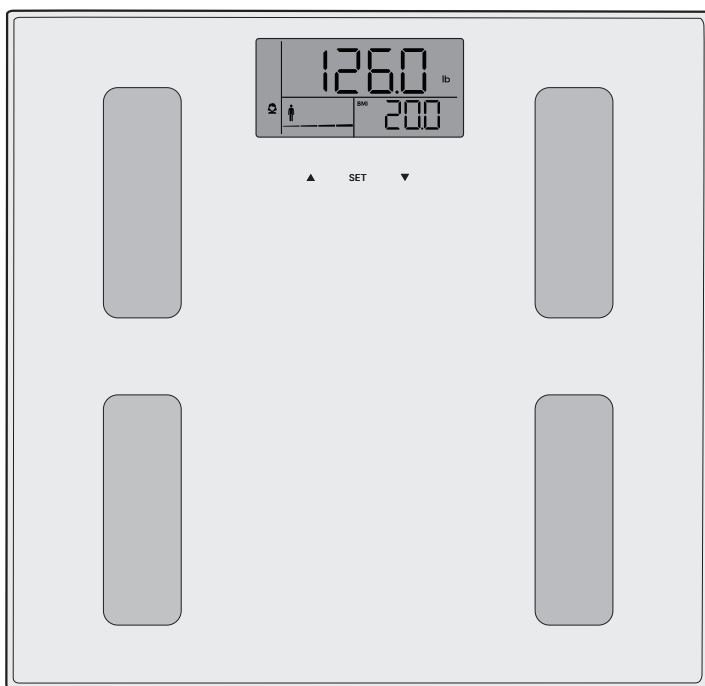
INSTRUCTION MANUAL ***ENGLISH AND SPANISH***

Please read this instruction manual
completely before operating this device.

STOP!
PLEASE ENSURE YOU HAVE ALL OF THE
FOLLOWING COMPONENTS BEFORE USING
YOUR WEIGHT SCALE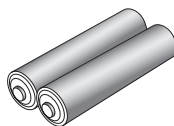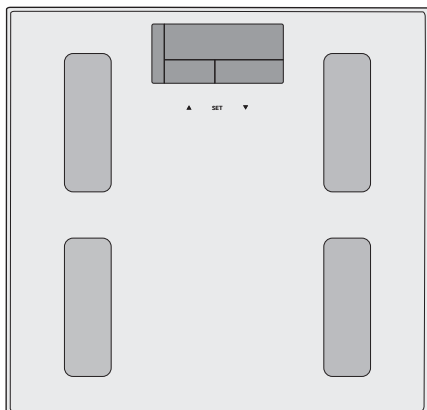

Two (2) AAA
Batteries

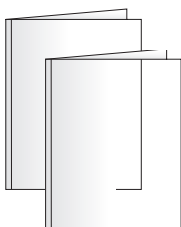Instruction Manual

INDEX

Introduction	4
Indications for Use	4
Contraindications	4
Care & Safety Information	4-5
Product Overview	6-7
Battery Installation	8
Mode Selection	8
Initial Set-Up	9
Instructions for Use	10-11
Care and Maintenance	12
Device & Label Symbols	12
Troubleshooting & Error Codes	13
Important Information About Body Composition	14-16
FCC Information	17
Specifications	18
Warranty	19
Instrucciones en Español	13-24

Toll-Free Customer Care Help Line:
1-866-326-1313
Monday – Friday
8:30 a.m. – 4:30 p.m. CST

SmartHeart™
Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1603 10/23
©2023 Veridian Healthcare, LLC

INTRODUCTION

Thank you for purchasing a Body Composition Digital Scale. Please read this instruction manual completely before using your unit to ensure safe and effective use, as well as long life of the product.

Body composition scales work by using bio-electrical impedance analysis (BIA) technology to estimate body fat, total body water percentage, bone mass and muscle mass. A slight, weak electric current travels through the metal sensors into the body, detecting the bio-impedance and estimating these attributes. BIA technology is harmless, with a current of less than 0.5mA. Users do not feel this electric current.

INDICATIONS FOR USE

This body composition digital scale is intended for use in generally healthy children 10 years and older and healthy adults who can fully support themselves standing on the scale.

This device measures: body weight, body fat, total body water percentage, bone mass and muscle mass. Results rely on user input including sex, body type, height, age and general activity category.

Intended for home use only.

CONTRAINDICATIONS

Not for use by pregnant persons or persons who may be pregnant. Results will be inaccurate due to the weight of the fetus and amniotic fluid and effects on the fetus of BIA technology is unknown.

Not for use by persons connected to a wearable or implantable electronic device or instrument such as a pacemaker or defibrillator.

CARE & SAFETY INFORMATION

NOTE: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

CAUTION: Failure to read and observe all precautions could result in personal injury or equipment damage.

SAFETY CAUTIONS AND WARNINGS

- Do not use on slippery floor such as some tiles or laminates
- Do not use with wet feet; feet should be completely dry before use
- Do not use near a cell phone or microwave oven

CARE & SAFETY INFORMATION

- Do not drop the scale; take care not to drop items onto the scale; do not jump onto the scale or while on the scale
- Always stand in the center of the scale; do not stand on the edges or allow feet to hang off the edge(s)

GENERAL CAUTIONS AND WARNINGS

- Do not use the unit if it has any damaged parts or if the top is cracked
- If any abnormality occurs, discontinue use until the unit has been examined and repaired
- Do not disassemble the unit or attempt to repair it; doing so may result in results error and will void manufacturer warranty
- Always follow local regulations for proper disposal of the digital scale and batteries
- Do not use this device simultaneously with other diagnostic monitoring equipment

OPERATING CAUTIONS AND WARNINGS

- Operation outside of stated operating temperature may result in measurement error or device malfunction; operation environment temperature is: 41°F – 104°F (5°C – 40°C); Humidity: ≤90%RH max
- DO NOT USE if pregnant or if you suspect you are pregnant
- DO NOT USE if you have an implanted or wearable electronic device such as a pacemaker or defibrillator
- Read and follow the Instructions For Use section of this manual for safe use of this device



STORAGE CAUTIONS AND WARNINGS

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is: -4°F – 140°F (-20°C – 60°C); Humidity: ≤90%RH max
- Avoid storage near water, moisture or excessive humidity
- Avoid storage in extreme temperatures, direct sunlight, dust or salt air
- Keep out of the reach of infants, children or incapacitated persons


CLEANING CAUTIONS AND WARNINGS

- Never immerse the unit in water to clean as it may damage the unit
- Follow the 'Cleaning and Maintenance' portion of this manual for instruction on how to clean and care for your scale

PRODUCT OVERVIEW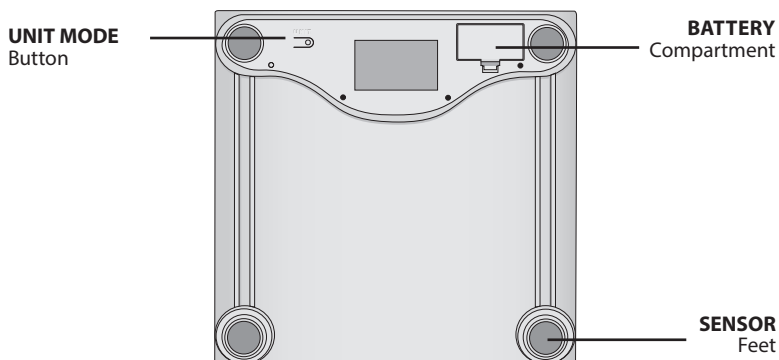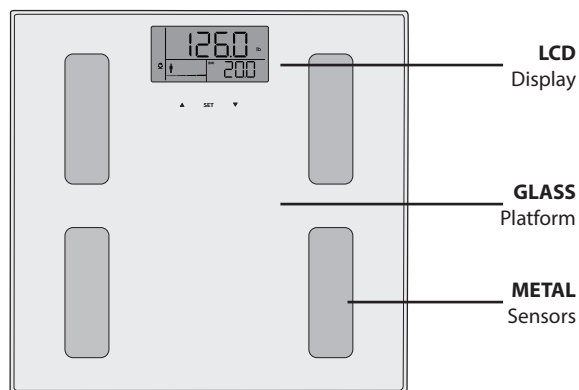

DIGITAL DISPLAY OVERVIEW

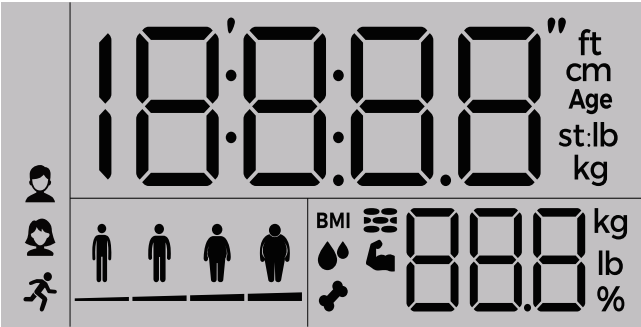 Body Fat Analysis Result	 Male
 Total Body Water Analysis Result	 Male Athlete
 Muscle Mass Analysis Result	 Female
 Bone Mass Analysis Result	 Female Athlete
st lb Stone	 Body Fat Level
lb Pound	 Body Fat Level
kg Kilogram	 Body Fat Level
% Percentage	 Body Fat Level
ft Foot	BMI Body Mass Index
cm Centimeter	age Age

DETAILS OF THE DIGITAL DISPLAY SCREEN AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.

BATTERY INSTALLATION



This unit includes 2 AAA batteries.

It is necessary to replace the batteries when the display does not turn on when being activated by stepping on the scale.


The battery compartment is located on the underside of this scale.

1. Open and remove the battery cover.
2. Insert or replace 2 AAA batteries into the battery compartment. Always use new batteries.
3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

It is recommended to remove the batteries if the unit will not be used for an extended period of time.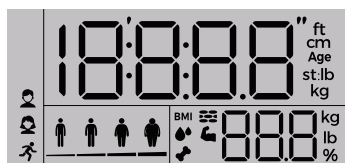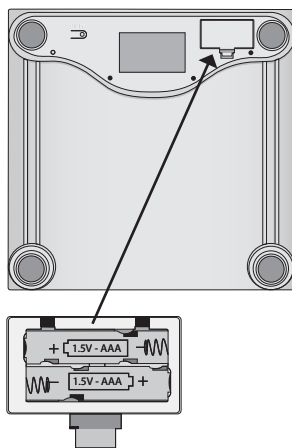

When batteries are first installed, the display will briefly show all digits.

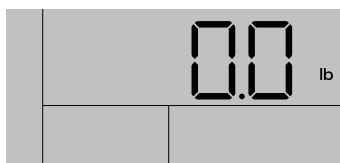When the display shows 0.0, it is ready for measurement.

MODE SELECTION-POUND (LB), KILOGRAM (KG), STONE (ST)

This scale will automatically default to pound (LB) unit of measure.

1. The unit change button is under underside of the scale to the left of the ratings label.
2. To change the unit of measure mode, press the UNIT button on the back side of the monitor.
3. The display will alternate between 'lb' 'kg' and 'st:kb'.
4. When the desired unit of measure is shown, release the button. The unit of measure mode is now set.


INITIAL SET-UP

This device supports up to 8 users. Setting individual user profiles helps ensure accuracy of the results.

- GENDER —ACTIVITY LEVEL TYPE - regular or athlete mode
- HEIGHT —AGE


Setting User ID

- When batteries are first installed and the display shows 0.0, press and hold the **SET** key to enter the setting mode.
- P1** will blink; press **SET** to confirm or press the **UP** or **DOWN** triangle to scroll to the desired user, up to U8.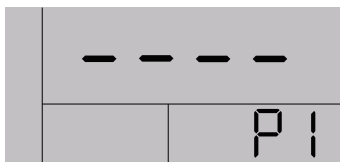


Setting Gender and Activity Level Modes

- Gender** will blink on the left side of the display; press the **UP** or **DOWN** triangle to scroll to the desired user type and press **SET** to confirm your selection.
- Gender options include: Male, Female, Male Athlete, Female Athlete
- Athlete is defined as a person who does 10 hours or more per week of aerobic activity and has a resting heart rate of 60 beats per minute.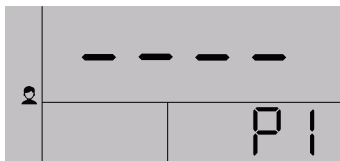


Setting Height

- The height will blink; press the **UP** or **DOWN** triangle to scroll to the user height. Press **SET** to confirm your selection.
- Press and hold the **UP** or **DOWN** triangle at any time to scroll faster to your selection.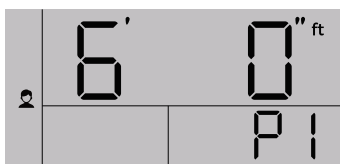

Setting Age

- The age will blink; press the **UP** or **DOWN** triangle to scroll to the users age. Press **SET** to confirm your selection.
- Press and hold the **UP** or **DOWN** triangle at any time to scroll faster to your selection.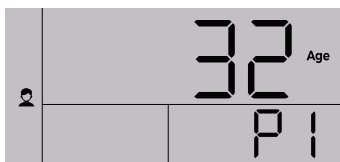

The scale will display 0.0 and the selected user and is now ready for use.

Repeat steps above to set additional users.

INSTRUCTIONS FOR USE

SENSE ON Technology—The Digital Scale will automatically turn on when you step onto the scale. It is not necessary to turn the unit ON/OFF.

FOR BEST RESULTS:

- Weigh yourself before eating
- Weigh yourself at the same time of day; ideally in the mornings after using the bathroom
- Weigh yourself naked or in minimal clothing
- The Digital Scale should be used on a flat, hard surface
- Use on soft surfaces such as carpeting may affect the accuracy of your results
- Remain still; do not hold onto or lean on anything



INITIALIZING YOUR SCALE:

- Every time a digital scale is moved it needs to be initialized (calibrated)
- Never store the scale on it's side; if stored vertically it must be initialized before each use
- When the scale is in the desired location, firmly press down on the platform to turn on with one foot
- Remove your foot; the scale will reset to 0.0
- After the scale shuts-off (no display) it is ready for use

BODY WEIGHT ONLY

SIMPLE STEP INSTRUCTIONS:

1. Step onto the scale. Stand still with your feet evenly planted on the scale surface, which will result in a more even weight distribution and more accurate results.
2. Stand still and keep full contact with the scale surface.
3. Allow the scale several seconds to calculate the final results; you will notice the numbers fluctuate multiple times before the true weight is reached.
4. The results plus **lb**, **kg** or **st** symbol will flash twice, indicating the measurement is final.
5. The results will show on the display in the selected mode – **lb**, **kg** or **st**. The results will display for approximately 15 seconds.
6. The digital scale will turn off after approximately 15 seconds.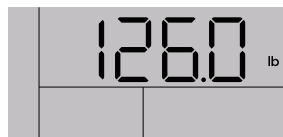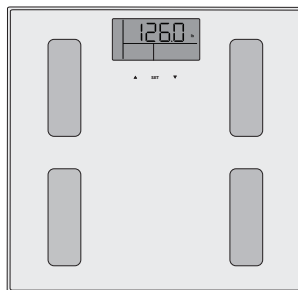


INSTRUCTIONS FOR USE

WEIGHT + BODY COMPOSITION RESULTS

MULTI USER INSTRUCTIONS:

YOU MUST BE BAREFOOT FOR BODY COMPOSITION MEASUREMENTS

1. Press down on the scale to activate the Sense On technology.
2. Press the **UP** or **DOWN** triangle to scroll to the desired user being measured.
3. Step onto the scale. Stand still with your feet evenly planted on the scale surface, which will result in a more even weight distribution and more accurate results.
4. Stand still and keep full contact with the scale surface.
5. Allow the scale several seconds to calculate the final weight results; you will notice the numbers fluctuate multiple times before the true weight is reached.
6. The results will show on the display in the selected mode – **lb, kg or st.**
7. **DO NOT STEP OFF THE SCALE.** The additional body composition results are still calculating.
8. The display will change as the calculations are complete.
9. The results will display for three rotations and then the scale will power off.

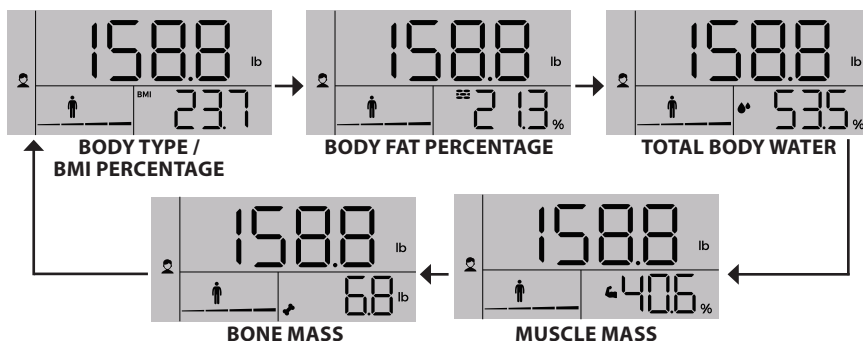NOTE: IF THE BODY COMPOSITION CANNOT BE MEASURED (EXCESSIVE MOVEMENT, POOR CONTACT WITH SENSORS, THE USER ISN'T BAREFOOT) THE SCALE WILL SHOW THE WEIGHT RESULT ONLY.

DAILY MEASUREMENTS

- After individual user profiles have been set up and a the users have completed multiple body composition measurements, it is no longer necessary to select the user profile before measurement.
- Internal technology will automatically pair the results with the user with the most similar history records and complete the measurements for that user.
- The display will show all screens as summarized in the WEIGHT + BODY COMPOSITION RESULTS portion of this manual.
- If two or more users have a similar history, the scale will prompt you to select the user by flashing the User ID symbol (P1, P2 etc). Press the **UP** or **DOWN** triangle and set to confirm selection.

CARE & MAINTENANCE

Use common sense caution when cleaning an electronic device.

- The scale may be dusted with a soft cloth, or wiped down with a damp cloth.
- Always dry the scale top after cleaning.
- Never get excessively wet or submerge the scale in water.
- Never use propellant, abrasive or other chemicals on the scale.
- Do not disassemble this device; doing so may void the manufacturer’s warranty.
- Follow the Operating and Storage Temperature ranges as indicated in the Specifications section of this manual.
- Follow the indicated weight range capacity; weight in excess of the stated range may damage the unit.
- It is recommended that the battery be removed if the scale will not be used for an extended period of time.

DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

Symbol	Meaning
	Read This Manual—All included manuals should be read prior to first use
	Warning—Symbol indicates a warning, prohibition or mandatory action that mitigates a risk that is not necessarily obvious to the device operator
	Environment Protection—dispose of this product properly; consult with your local recycling ordinances for proper recycling and disposal
	Keep Dry—This device should be kept dry; never submerge the unit or cuff.
	Type BF Applied Part—Indicates that a part of this unit comes in contact with the patient in order to carry out it’s intended function.
	Manufacture Date Symbol

TROUBLESHOOTING & ERROR CODES

ERROR	DESCRIPTION	SOLUTION
	Overload / too much weight is on the scale. The unit will shut off.	This scale is not suitable for weight in excess of 199 kg / 438.7 lbs; cease using this scale
	Low battery	Replace batteries with 2 NEW AAA batteries

PROBLEM	DESCRIPTION	SOLUTION
Blank LCD display with the device is powered on	Battery is not installed or installed incorrectly; battery is depleted.	Install 2 NEW AAA batteries; check the polarity of the batteries
The scale activates but shuts off immediately	Batteries are low	Replace batteries with 2 NEW AAA batteries
Abnormal results: • Too High • Too Low • Large different between two recent measurements	Incorrect / poor posture	Take care to stand upright and still and attempt new measurement
	Scale is not on firm, flat ground (ie a rug or carpet or ramp)	Place scale on a firm, flat surface
	Users feet are too dry (body composition measurements only)	Wipe feet with a lightly damp cloth and take care to stand entirely on the scale sensors
Scale cannot analyze body composition features (BMI, Body Fat, Muscle Mass, Bone Mass)	User is wearing socks and/or shoes	User must be barefoot to properly measure advanced features; remove socks and attempt new measurements
	Scale cannot match user ID to a current profile	Assign a user ID using instructions in this manual
	User fails to select user ID from the display prompt	Follow the display prompts; assign a user ID using instructions in this manual

IMPORTANT INFORMATION ABOUT BODY COMPOSITION METRICS

BMI AND HOW IS IT CALCULATED BMI

- Body Mass Index (BMI) is a calculation or medical screening tool that helps provide an indication of the individual's health by calculating the estimated body fat.
- BMI = weight in pounds (kilograms) divided by the square of height in inches (centimeters)
- **EXAMPLE:**
$$\text{BMI} = (\text{your weight in pounds} \times 703) \div (\text{your height in inches} \times \text{your height in inches})$$

For example, if you weigh 120 pounds and are 5 ft. 3 in. (63 in.) tall:

$$\text{BMI} = (120 \times 703) \div (63 \times 63) \text{ or } 84,360 \div 3969 = 21.3$$
- A high body fat can be in indicator of a higher risk for health problems.
- Standard accepted BMI ranges are:
Underweight: BMI is less than 18.5
Normal weight: BMI is 18.5 to 24.9
Overweight: BMI is 25 to 29.9
Obese: BMI is 30 or more

BODY FAT

- Body fat is a vital component for a healthy, functioning body. Body fat acts as a protector of vital organs, helps regulate body temperature, stores vitamins and helps the body sustain itself when food sources are scarce.
- Too high of a body fat percentage can be an indication of higher risk for health problems.
- Commonly accepted standard for body Fat Levels:

STANDARD FOR MEN

RATING	AGE RANGE				
	20-29	30-39	40-49	50-59	60+
Low	<13	<14	<16	<17	<18
Normal	13-20	14-21	16-23	17-24	18-25
Moderately High	20.1-23	21.1-24	23.1-26	24.1-27	25.1-28
High	>23	>24	>26	>27	>28

STANDARD FOR WOMEN

RATING	AGE RANGE				
	20-29	30-39	40-49	50-59	60+
Low	<19	<20	<21	<22	<23
Normal	19-28	20-29	21-30	22-31	23-32
Moderately High	28.1-31	29.1-32	30.1-33	31.1-34	32.1-35
High	>31	>32	>33	>34	>35

IMPORTANT INFORMATION ABOUT BODY COMPOSITION METRICS

BODY WATER CONTENT

- Body water is the single most important component of your overall body weight.
- Body water percentage can help indicate how hydrated your cells are, if your diet includes enough fruits and vegetables and how much water you need to drink for your overall body type.
- Body water percentage is a strong indicator of overall health; body cells only function properly if they have enough water.
- Lean body mass stores more water than fat.
- Body water content may fluctuate immediately after hydrating or excessive perspiration such as exercise. For best results, consider these measurements before excessive exertion or water consumption.

	BODY FAT % RANGE	OPTIMAL TOTAL BODY WATER % RANGE
MEN	4 - 14%	70 - 63%
	15 - 21%	63 - 57%
	22 - 24%	57 - 55%
	25 and over	55 - 37%
WOMEN	4 - 20%	70 - 58%
	21 - 29%	58 - 52%
	30 - 32%	52 - 49%
	33 and over	49 - 37%

MUSCLE MASS CONTENT

- Muscle mass is the amount of muscle in your body, including: skeletal muscles, smooth muscles, and cardiac muscles.
- Muscle mass naturally declines over time; tracking muscle mass can be beneficial in planning for activities to help slow that decline.
- Skeletal muscles are the most vital muscle group and are responsible for mobility and joint health.

	AGE	MUSCLE MASS % RANGE
MEN	18 - 35	40 - 44%
	36 - 55	36 - 40%
	56 - 75	32 - 35%
	76 and over	<31%
WOMEN	18 - 35	31 - 33%
	36 - 55	29 - 31%
	56 - 75	27 - 30%
	76 and over	<26%

IMPORTANT INFORMATION ABOUT BODY COMPOSITION METRICS

BONE MASS

- Bone mass is a function of bone size and volumetric bone mineral density (BMD).
- Bone is a living tissue that responds to exercise and nutrition to become stronger.
- For most people, bone mass peaks in their 30's; as they age people begin to lose bone.
- Those with osteoporosis or known low bone density should consult with a healthcare professional; bone density measurements with this device may not be accurate.

The ideal average bone mass percentage for both men and women is between 4 to 5%.

ATHLETE MODE / ATHLETE DEFINITION

- Athlete is defined as a person who does 10 hours or more per week of aerobic activity and has a resting heart rate of 60 beats per minute.
- Physiological variations in an athletes body may affect the composition results this device is measuring.
- Example: BMI doesn't measure body fat specifically but is a calculation of weight and height, so, an athlete with a lot of muscle, could have a higher BMI based on their weight and height.
- Example: Athletes typically will have a higher bone density outside of normal range and higher level of hydration.
-

Selecting 'Athlete' Mode adjusts the internal scale calculation to consider these physiological variances. Always consult with a healthcare professional for help in interpreting your results.

IMPORTANT INFORMATION REQUIRED BY THE FCC

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference; and (2) this device must accept any interference received, including interference that may cause undesirable operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

SPECIFICATIONS

Name	SmartHeart Body Composition Digital Weight Scale	
Model Number	19-102	
Display System	Digital display	
Scale Dimensions	11.8" x 11.8" x .9" / 300 x 300 x 22.8 mm	
Scale Weight	2.8 lbs./1.3 kg (without battery)	
Measuring Unit	Pound (lb) / Kilogram (kg) / Stone (st)	
Measuring Range	11 lb – 440 lb / 5 kg – 200 kg	
Division	0.2 lb; 0.1 kg	
Accuracy	50 kg: ± 0.3 kg	100 kg: ± 0.4 kg
	150 kg: ± 0.5 kg	200 kg: ± 0.7 kg
Operating Environment	Temperature 41°F – 104°F (5°C – 40°C); Humidity $\leq 90\%$ RH max; Atmospheric pressure 86 kPa – 106 kPa	
Storage Environment	Temperature -4°F – 140°F (-20°C – 60°C); Humidity $\leq 90\%$ RH max; Atmospheric pressure 50 kPa – 106 kPa	
Power Source	3V 2x AAA batteries	
Battery Life	Approximately 650 measurements	
Auto-On	SenseOn Technology	
Auto-Off	After 15 seconds of non-use	
Mode of Operation	Continuous Operation	
Accessories	Instruction Manual; Two (2) AAA Batteries	

Specifications are subject to change without notice

FIVE-YEAR LIMITED WARRANTY

Congratulations on your purchase of a Weight Scale. Your Weight Scale is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Weight Scale will be free from defects in materials and workmanship under normal consumer usage for a period of five years for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Weight Scale for commercial purposes or subjecting the Weight Scale to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Weight Scale.

If the Weight Scale fails to conform to this limited warranty, return the Weight Scale postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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