**R.P.M.
PRO**TM

DIGITAL WEIGHT SCALE

MODEL 210-1113C

INSTRUCTION MANUAL


Please read this instruction manual
completely before operating this unit.


STOP!
PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING
COMPONENTS BEFORE USING YOUR DIGITAL WEIGHT
SCALE

Digital Scale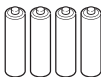

Four (4) AA
Batteries
Preinstalled

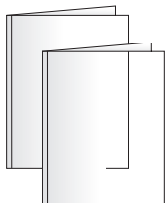Instruction Manual

INTENTION FOR USE

This product is specially designed for body weight measurement for persons who are able to stand on the scale platform for medical purposes. It should never be used for verification of goods' weight. Personal weight results should be discussed with your physician or healthcare professional; this device is not intended to diagnose, treat or prevent an illness.

INDEX

Care & Safety Information	4-5
Attention	6
Product Overview	7
Battery Installation	8
Mode Selection	9
Intention for Use	10-11
Care & Maintenance	12
Troubleshooting & Error Codes	13
Device & Label Symbols	14
FCC Information	15
Electromagnetic Chart	16-18
Specifications	19
Warranty	20
Instructions in Spanish	21-40

Toll-Free
Customer Care Help Line:

1-866-326-1313

Monday – Friday
8:30 a.m. – 4:30 p.m. CST
www.veridianhealthcare.com

R.P.M. PRO
Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1606 04/25
©2025 Veridian Healthcare

CARE & SAFETY INFORMATION

NOTE: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

NOTE: Failure to read and observe all precautions could result in personal injury or equipment damage.

SAFETY CAUTIONS AND WARNINGS

- Do not use on slippery floor such as some tiles or laminates
- Do not use with wet feet; feet should be completely dry before use
- Do not use near a cell phone or microwave oven
- Do not drop the scale; take care not to drop items onto the scale; do not jump onto the scale or while on the scale
- Always stand in the center of the scale; do not stand on the edges or allow feet to hang off the edge(s)

GENERAL CAUTIONS AND WARNINGS

- Do not use the unit if it has any damaged parts or if the top is cracked
- If any abnormality occurs, discontinue use until the unit has been examined and repaired
- Do not disassemble the unit or attempt to repair it; doing so may result in results error and will void manufacturer warranty
- Always follow local regulations for proper disposal of the digital scale and batteries
- Do not use this device simultaneously with other diagnostic monitoring equipment

CARE & SAFETY INFORMATION

OPERATING CAUTIONS AND WARNINGS

- Operation outside of stated operating temperature may result in measurement error or device malfunction; operation environment temperature is:
Temperature: 5°C to 40°C / 41°F to 104°F
Relative Humidity: ≤90% RH
Atmospheric pressure: 860hPa to 1060hPa
- Read and follow the Instructions For Use section of this manual for safe use of this device

STORAGE CAUTIONS AND WARNINGS

- Storage outside of stated storage temperature may result in measurement error or device malfunction; storage environment temperature is:
Temperature: -20°C to 60°C / -4°F to 140°F
Relative Humidity: ≤90%
Atmospheric pressure: 500hPa to 1060hPa
- Avoid storage near water, moisture or excessive humidity
- Avoid storage in extreme temperatures, direct sunlight, dust or salt air
- Keep out of the reach of infants, children or incapacitated persons

CLEANING CAUTIONS AND WARNINGS

- Never immerse the unit in water to clean as it may damage the unit
- Follow the 'Cleaning and Maintenance' portion of this manual for instruction on how to clean and care for your scale


ATTENTION!

This manual is intended for instruction on operating this weight scale as a stand-alone measurement device.

Your Weight Scale **MAY HAVE** connectivity to a Smartphone App, Digital Alert Device, Hub or other Assistive Technology.

For instructions on how to connect your Weight Scale with other devices, please consult the documentation that was provided with the device.

PRODUCT OVERVIEW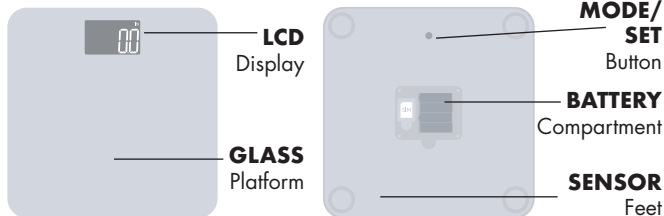

kg	Kilogram		Signal Symbol
lb	Pounds		Data Transmitting
st lb	Stone		Data Pending to Transmit
	Battery Symbol		No SIM Card Information

DETAILS OF THE DIGITAL DISPLAY SCREEN AND INTERPRETATION OF THE SYMBOLS ARE INCLUDED THROUGHOUT THIS MANUAL.

BATTERY INSTALLATION

THIS UNIT INCLUDES 4 AA BATTERIES, PREINSTALLED.

When using for the first time, remove the pull strip to activate the batteries.


It is necessary to replace the batteries when the low battery symbol appears on the display or when the display does not turn on when the scale is activated.

1. Slide open the battery compartment on the backside of the device.
2. Insert or replace 4 AA batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.

When batteries are first installed, the display will briefly show a test display.

3. Replace the battery cover.
4. Dispose of batteries according to local disposal and recycling regulations.

It is recommended to remove the batteries if the unit will not be used for an extended period of time.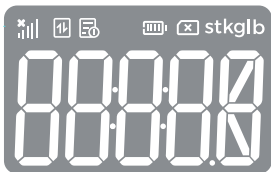


When batteries are first installed, the display will briefly show all digits.

MODE SELECTION—LB, KG, STONE

This scale will automatically default to pound (LB) unit of measure.

1. To change the unit of measure mode, press and release the UNIT button on the back side of the scale.
2. The display will alternate between 'lb', 'kg' and 'st lb'.
3. When the desired unit of measure is shown, release the button.
The unit of measure mode is now set

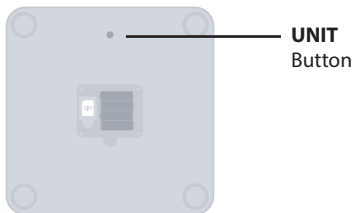INSTRUCTIONS FOR USE

SENSE ON Technology—This digital scale will automatically turn on when you step onto the scale. It is not necessary to turn the unit ON/OFF.

FOR BEST RESULTS:

- Weigh yourself before eating
- Weigh yourself at the same time of day; ideally in the mornings after using the bathroom
- Weigh yourself naked or in minimal clothing
- The Digital Scale should be used on a flat, hard surface
- Use on soft surfaces such as carpeting may affect the accuracy of your results
- Remain still; do not hold onto or lean on anything

SIMPLE STEP INSTRUCTIONS:

1. Step onto the scale. Stand still with your feet evenly planted on the scale surface, which will result in a more even weight distribution and more accurate results.
2. Stand still and keep full contact with the scale surface.
3. Allow the scale several seconds to calculate the final results; you will notice the numbers fluctuate multiple times before the true weight is reached.
4. The results will show on the display in the selected mode – **LB**, **KG**. The results will display for approximately 10 seconds.
5. Device will turn off approximately 10-seconds after data transmission is complete.

INSTRUCTIONS FOR USE

Stand still and keep feet in full contact; the results will blink twice and then lock as the final result.

The Signal symbol at the upper left will indicate that the cellular connection is successful.

If the scale fails to connect, the Signal symbol will appear with an 'x'.

After connecting, the weight results data will be transmitted; when data is successfully transmitted, OK and the data transmitting symbol will appear on the display.

If the data fails to transmit, the Data Pending symbol will appear.

CARE & MAINTENANCE

Use common sense caution when cleaning an electronic device.

- The scale may be dusted with a soft cloth, or wiped down with a damp cloth.
- Always dry the scale top after cleaning.
- Never get excessively wet or submerge the scale in water.
- Never use propellant, abrasive or other chemicals on the scale.
- Do not disassemble this device; doing so may void the manufacturer's warranty.
- Follow the Operating and Storage Temperature ranges as indicated in the Specifications section of this manual.
- Follow the indicated weight range capacity; weight in excess of the stated range may damage the unit.
- It is recommended that the battery be removed if the scale will not be used for an extended period of time.

TROUBLESHOOTING & ERROR CODES

ERROR	DESCRIPTION	SOLUTION
	Overload / too much weight is on the scale.	Retake measurement; user may be out of weight range (11 lb to 440 lb)
	SIM error	Check and reinstall the SIM card if the issue persists, contact customer support
	No signal	Move signal to stronger signal place; contact customer care
	Hardware error	Retake measurement; If the issue persists, contact customer support
	Data transmission error	Move to new area, closer to a window; use device at a location where mobile phone has strong signal
	No cellular coverage or cellular connection error	Move to new area, closer to a window; use device at a location where mobile phone has strong signal
	Preloaded SIM card is replaced or subscription no longer valid	Ensure SIM card is the original card; do not replace SIM card by yourself; contact customer support
	Server connection error	Move to new area, closer to a window; use device at a location where mobile phone has strong signal
	Certificate error	Contact customer support
	Hardware error	Retake measurement; if issue persists contact customer support

DEVICE & LABEL SYMBOLS

These symbols may appear on your device, instructions or packaging and may vary by make and model.

 Refer to the instruction manual/booklet. NOTE: The background color of this symbol is blue.	 Non-ionizing electromagnetic radiation
 Serial Number	 Date of Manufacture
 Direct Current	 General symbol for recovery / recyclable
 Type BF Applied Part	 Keep Dry
 FCC: This device can be expected to comply with part 15 of the FCC Rules	 Batch Code
IP21 It means the device could be protected against solid foreign objects for 12,5mm Φ and greater, and against vertically falling water drops.	 The symbol indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recover and recycling.
 Caution: Indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences	

FCC INFORMATION

IMPORTANT INFORMATION REQUIRED BY THE FCC

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference; and (2) this device must accept any interference received, including interference that may cause undesirable operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.

End user must follow the specific operating instructions for satisfying RF exposure compliance.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

The portable device is designed to meet the requirements for exposure to radio waves established by the Federal Communications Commission (USA).

These requirements set a SAR limit of 4.0 W/kg averaged over one gram of tissue.

The highest SAR value reported under this standard during product certification for use when properly worn on the extremity (hands/feet/ankle/wrist).

ELECTROMAGNETIC COMPATIBILITY

Guidance and manufacture's declaration - electromagnetic emissions

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments.

Warning: Don't be near the active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Table 1

Guidance and manufacture's declaration - electromagnetic emissions

Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not Applicable
Voltage fluctuations and flicker IEC 61000-3-3	Not Applicable

ELECTROMAGNETIC COMPATIBILITY

Table 2

Immunity test	IEC60601-1-2 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact; ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 4 kV, ± 8 kV, ± 15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not Applicable	Not Applicable
Electrical fast transient/burst IEC 61000-4-5	Not Applicable	Not Applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not Applicable	Not Applicable
Power frequency magnetic field IEC 61000-4-8	30 A/m 50 Hz / 60 Hz	30 A/m 50 Hz / 60 Hz
Conducted RF IEC61000-4-6	Not Applicable	Not Applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80% AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80% AM at 1 kHz

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

ELECTROMAGNETIC COMPATIBILITY

Table 3

Guidance and manufacture's declaration - electromagnetic emissions

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communication s equipment)	Test Immunity MHz	Band MHz	Service	Modulation	Modulation Power (W)	Distance (m)	Immunity Test Level (V/m)	Compliance Level (V/m)
	385	380-390	TETRA 4000	Pulse modulation b) 18Hz	1, 8	0, 3	27	27
	450	430-470	GMRS 460, FRS 460	FM c) ± 5 kHz deviation 1 kHz sine	2	0, 3	28	28
	710	704-787	LTE Band 13,17	Pulse modulation b) 217 Hz	0, 2	0, 3	9	9
	745							
	780							
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18 Hz	2	0, 3	28	28
	870							
	930							
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1,3,4,25; UMTS	Pulse modulation b) 217 Hz	2	0, 3	28	28
	1845							
	1970							
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation b) 217 Hz	2	0, 3	28	28
	5240	5100-5800	WLAN 802.11a/n	Pulse modulation b) 217 Hz	0, 2	0, 3	9	9
	5500							
	5785							

SPECIFICATIONS

Name	Digital Weight Scale
Model Number	210-1113C
Display System	Digital LCD
Scale Dimensions	12.6" x 12.6" x 1.12" / 320 x 320 x 28.5mm
Scale Weight	4.7 lbs./2.16 kg (without battery)
Measuring Unit	Pound (lb) / Kilogram (kg) / Stone (st lb)
Measuring Range	11 lb – 440 lb / 5 kg – 200 kg
Division	0.1kg/0.2lb
Accuracy	5-100kg, 0.05kg/0.2lb; 100.1-200kg, 200±0.5kg
Operating	Temperature: 5°C to 40°C / 41°F to 104°F Relative Humidity: ≤90% RH Atmospheric pressure: 860hPa to 1060hPa
Storage	Temperature: -20°C to 60°C / -4°F to 140°F Relative Humidity: ≤90% Atmospheric pressure: 500hPa to 1060hPa
Power Source	6V 4 AA batteries
Battery Life	Approximately 650 measurements
Auto-On	Sense On Technology
Auto-Off	After 10 seconds of non-use
Mode of Operation	Continuous Operation
Accessories	Instruction Manual; Four (4) AA Batteries

Specifications are subject to change without notice

ONE-YEAR LIMITED WARRANTY

Congratulations on your purchase of a Weight Scale. Your Weight Scale is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Weight Scale will be free from defects in materials and workmanship under normal consumer usage for a period of one year for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Weight Scale for commercial purposes or subjecting the Weight Scale to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Weight Scale.

If the Weight Scale fails to conform to this limited warranty, return the Weight Scale postage prepaid to: Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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