LANCING DEVICE PACKAGE INSERT

Model HH-XIII-T REF: 210-7110LD

INTENDED USE

The RPM PRO Lancing Device is used with the RPM PRO Lancets to draw a drop of capillary blood from the fingertip for blood glucose testing or other testing. It is intended to be used on a single patient age 6 months and up. And it should not be shared.

LANCET PENETRATION DEPTH RANGE The number 1-5 at the Depth Setting Window is a depth gauge; it indicates the depth that the lancet will go into the skin. The lancet penetration depth range is at 0.02-0.08 inch with number 1 being the shallowest and 5 being the deepest. It is for reference only when you use a RPM PRO Lancing Device with a RPM PRO Lancet. If you use another manufacturer's lancets, adjust the depth setting according to skin status and blood volume needed.

LEVEL	LEVEL 1	LEVEL 2	LEVEL 3	LEVEL 4	LEVEL 5
Penetration Depth Range	0.02"	0.04"	0.06"	0.07"	0.08"
Deviation ± 0.01 inch					

CLEANING AND DISINFECTING

Clean and disinfect before and after each use.

1. Wash hands with soap and water, and dry thoroughly.
2. Wipe the entire lancing device surface and the inside of the lancing device cap with a Super Sani-Cloth® to clean it.
3. Use a new Super Sani-Cloth® and repeat step 2 to disinfect it. Make sure to keep cleaning the surface and the inside of the cap for 2 minutes.
4. Wash hands with soap and water, and dry thoroughly.

NOTE:

- Failure to follow the instructions may damage the lancing device and stop it from working properly.
- Do not get any moisture into openings.
- If blood gets on the lancing device, clean and disinfect it before anyone else touches it.
- Always use the same cleaning product for both cleaning and disinfecting.
- If there is residual soil on the lancing device, repeat step 2 and step 3 above.
- A separate Super Sani-Cloth® is suggested to be used for cleaning and disinfection.

INSTRUCTIONS FOR USE

Wash your hands before and after each blood sample collection with soap and warm water, or wipe your finger with an alcohol pad. Dry your hands completely before lancing your finger.


NOTE: Make sure there is no soap, alcohol or lotion on the puncture site. Residual alcohol may lead to inaccurate readings.

Read the instructions carefully before using the lancing device.

1. Remove the device cap and connecting collar by holding and turning the collar counterclockwise then pulling.
2. Insert a lancet into the lancet holder by sliding the lancet into the holder until it snaps in place.
3. Twist the lancet cap off the lancet. (Save the lancet cap for lancet removal. Described in step 9.)
4. Put the cap and collar back on the lancing device by reattaching and turning clockwise to tighten.
5. Hold the lancing device firmly and turn the device cap to adjust the lancet penetration depth. Choose from settings 1 through 5 in the depth level window.

Note: The smaller the number the shallower the puncture and less painful. Lower settings will be best for children and most adults. Larger numbers are for deeper punctures; these settings sometimes necessary for those with thicker skin. Start with a small number setting and increase incrementally until you find the correct setting for a proper blood sample.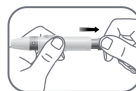


6. Pull the rear handle back until it clicks to load the lancet device.


7. Hold the lancing device against the side of a fingertip and press the release button to prick the skin and obtain a drop of blood. If necessary, gently massage the pricked finger until a drop of blood forms.

8. Apply the blood sample to the test strip following the proper test strip instructions.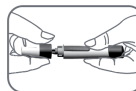

9. Remove the device cap and connecting collar (like in step 1), place the lancet cap on a hard surface, and push the lancet into the lancet cap.

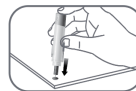10. Slide the lancet ejector forward to release the used lancet. Make sure to dispose of the lancet in an appropriate container, in accordance with local regulations. NEVER use your fingers to pull a used lancet out.

11. Put the device cap and connecting collar back on the lancing device (like in step 4).

CAUTIONS:

1. Do not immerse the device into any liquid.
2. Do not point the lancing device at you or other people.
3. Clean and disinfect the entire lancing device surface and the lancing device cap carefully to prevent infection and contamination.
4. Do not use the lancet if the lancing device falls to the ground or surfaces with a lancet inserted. Pick it up with care, discard the lancet and insert a new lancet before use.
5. For use only on a single patient. Do not use it on more than one patient. Improper use of lancets and the lancing device can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested.
6. If the lancing device is being operated by a second person who is providing testing assistance to the patient, the lancing device should be cleaned and disinfected prior to use by the second person. The second person must wear protective gloves for all the steps when providing testing assistance.
7. If the lancing device has come into contact with blood, it can transmit infections. The lancing device and lancets must not be used by other people, even by family members. Healthcare professionals must not use them to obtain blood from different people.
8. If the lancing device or the lancet was dropped or exposed to other mechanical stress, check it for damages or cracks. Do not use the lancing device or lancets if they are damaged.
9. If the lancing device falls with a lancet inserted, the lancet may become loose in the lancet holder, you cannot use it anymore to obtain blood. In rare cases, the lancet could protrude from the opening in the cap and you could injure yourself on it. Therefore, do not touch the front of the cap. If the lancing device falls, pick it up with care.

WARNINGS:

1. Read the instructions carefully before use.
2. For single use only. Use a new Lancet each time you test. Discard the used lancet after each use. Do not re-use. Do not leave the used lancet in the lancing device.
3. Do not use the lancet if the Lancet Cap is already removed or broken.
4. Do not use lancets beyond the expiration date printed on the box.
5. If an individual is under the age of 18, the lancet and lancing devices must be used by an adult or under the supervision of the adult.
6. Use with caution whenever the Lancet Tip is exposed.
7. Consult a healthcare professional before using the lancing device and selecting the setting for the first time on children to prevent any injury.
8. Keep out of reach of children.
9. Dispose of used lancets and follow local regulations for proper disposal to prevent injury from the lancets.
10. It is strongly recommended to use the RPM Pro Lancing Device and the RPM Pro Lancets for the best compatibility and proper performance. If you use the RPM Pro Lancing Device with another brand of lancets, make sure the lancets are compatible the RPM Pro Lancing Device. Consult a health care professional before use.
11. When you insert the lancet, please do not press the release button if the device cap is removed.

STORAGE CONDITIONS:

The products should be stored in a dry, well-ventilated room. Keep away from any corrosive gas.

WARRANTY

The service life of the RPM PRO lancing device is no more than 5000 times or 10 years, whichever comes first. A malfunctioning lancing device should be returned to your supplier for replacement.

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

For urgent matters,
please contact your health care professional.
www.veridianhealthcare.com

Distributed by: Veridian Healthcare
1175 Lakeside Drive, Gurnee, IL 60031
Made in China
#931633 04/25
©2025 Veridian Healthcare
PN:

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