
# Insect Bite and Sting Helper

Model 59-029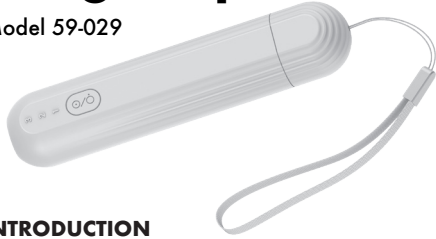

## INTRODUCTION

The HealthWise Insect Bite and Sting Helper offers relief from itching, irritation and swelling from bug bites and stings by applying brief, direct heat soon after a bite.

Many insect bites and stings result in painful and swollen sting and bite marks. These bites/stings often cause severe itching. The Insect Bite and Sting Helper offers very effective relief for insect bites and stings through heat. The toxins that can cause itching and swelling are generally made up of protein molecules. Proteins change their molecular structure when a certain temperature limit is exceeded, and therefore often lose their effect. As a result, the itching subsides and your symptoms improve; it's often noted that the area heals more comfortably.

Intended for use by person's over 12; safe for children 12 years of age and younger with adult supervision.

**NOTE:** Read all instructions carefully before use. The following basic precautions are needed when using an electrical product.

**CAUTION:** Failure to read and observe all precautions could result in personal injury or equipment damage.

## GENERAL CAUTIONS:

- Improper care or use of your Insect Bite and Sting Helper may result in injury, damage to the unit or ineffective treatment. Following these instructions will ensure the device's efficacy and long life.
- This product is not intended to diagnose or treat any health problem or disease and should not be used as a substitute for the consultation and advice of a physician or other medical professional.
- Only use this device in the operating conditions indicated in the Specifications section of this instruction guide.

- If stored outside of operating conditions, allow to warm to operating temperature before use.
- Do not disassemble the unit or attempt to repair device; substitution of a component different from that supplied may result in measurement error and will void manufacturer warranty.
- Never immerse in water to clean the unit; if the device gets wet, discontinue use immediately
- Do not use device if it is damaged or has been dropped.

## OPERATING CAUTIONS:

- Persons' sensitivity varies; if device feels too hot, if redness or irritation occurs, discontinue use immediately.
- Do not use while charging.
- Persons with diabetes may have reduced sensitivity; consult with your healthcare provider prior to use.
- Do not use on persons with disabilities, children under 3 years of age, people with reduced sensitivity to heat.
- Do not use near eyes, on eyelids, mouth or on mucus membranes.
- Do not use on acutely or chronically diseased skin.
- Do not use if you have a fever.
- Do not use if you have sensory impairment that reduces the feeling of pain.
- Do not use over areas of the skin with lotions, creams or gels.

## CHARGING:

This device includes a rechargeable lithium battery (3.7v) and USB cable compatible with common USB wall adapters.

- Slide the battery cover away from the end of the unit for access to the power adapter port.
- During charging, the three indicator lights will blink in sequence.
- When fully charged, all lights will be solid.
- Low battery — Level indicator light will flash while heating; device must be charged soon.
- To preserve lithium battery life, it is suggested that this unit be charged every 3 months even when not in use.

## INSTRUCTIONS FOR USE:

This device offers 6 levels of heat intensity. **3/2/1** – at top of device near POWER button. **H/L** – under the power access cover.

- First time use default is level 1, L (low). After first use, device will default to current setting with each subsequent use.
- Skin sensitivity varies; start with lowest setting to determine best treatment for an individual's skin type and avoid risk of injury.

- Press and hold POWER button to turn unit on.
- Level light will illuminate.
- Press and release POWER button to toggle through heat levels.
- When the device is ready / heated to the proper level (several seconds) it will give an audible beep.
- Quickly press the tip to the bug bite and hold. The device will automatically shut off when treatment is complete, approximately 5-7 seconds.
- Consult the HEATING TEMPERATURE AND TIME chart below for a detailed temperature guide.

**NOTE:** Wait at least five minutes before a second application to the same bite site. Do not exceed 5 applications on the same site within one hour.

## CARE AND MAINTENANCE:

- Do not use any liquids on the device; use a soft, dry cloth to clean the device as needed.
- If needed, use a cloth dampened with 75% diluted isopropyl alcohol to wipe and allow to air dry.
- Check for damage prior to every use.
- Always charge battery when low battery is evident.

## PRODUCT SPECIFICATIONS

### HEATING TEMPERATURE AND TIME

LOW TEMP MODE:	Position 1: 104°F (40°C); heating time 10 sec
104°F ~ 111.2°F (40°C ~ 44°C)	Position 2: 107.6°F (42°C); heating time 10 sec
	Position 3: 111.2°F (44°C); heating time 7 sec
HIGH TEMP MODE:	Position 1: 113°F (45°C); heating time 7 sec
113°F ~ 122°F (45°C ~ 50°C)	Position 2: 116.6°F (47°C); heating time 5 sec
	Position 3: 122°F (50°C); heating time 3 sec

## TROUBLESHOOTING:

PROBLEM	POSSIBLE CAUSE	SOLUTION
Device does not get hot / not hot enough	Device is not in indicated environment temperature range Device is working abnormally	Use device only in indicated environment temperature range Contact Customer Care
Device does not power on or does not last long	Low or depleted battery Device is working abnormally	Charge with included cable Contact Customer Care
The battery will not charge after extended charge time	Device is working abnormally	Contact Customer Care

## SYMBOLS:

These symbols may appear on your device, instructions or packaging; may vary by make and model.

SYMBOL	MEANING	SYMBOL	MEANING
	Manufacturer		Manufacture Date
	Serial Number		Use By Date
	Environmental Protection		USB
	Temperature Range		Límite de humedad
	Atmospheric Pressure Range		Hacia arriba
	Fragile		Mantener seco
	Power		Reciclable
	Batch No.		

Storage Temperature: -40°F ~ 140°F (-40°C ~ 60°C) Humidity 0-95%  
 Operating Temperature: 41°F ~ 104°F (5°C ~ 60°C) Humidity ≤75%  
 Working Voltage: DC 3.6V - 4.2V  
 Working Current: ≤ 250mA  
 Power Supply: Rechargeable lithium battery (3.7V)  
 Battery Life: Charge, discharge minimum of 500 times  
 Adapter specifications: Output Voltage: DC 5V; Output Current: 1000 mA  
 Unit Dimensions: 111 mm (L) x 24 mm (W) x 22 mm (H)  
 Unit Weight: ~ 45 g

Specifications are subject to change without notice.

**Toll-Free Customer Care Help Line:**  
**1-866-326-1313**  
**Monday – Friday 8:30 a.m. – 4:30 p.m. CST**

HealthWise®  
 Manufactured for Veridian Healthcare  
 1175 Lakeside Drive Gurnee, IL 60033  
 Made in China  
 #93-1635 08/24  
 ©2024 Veridian Healthcare, LLC

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