R.P.M. PRO™

GETTING STARTED

Consult included manual for more
instructions, warnings & cautions

BLOOD PRESSURE MONITOR

MODEL 210-2111B

TIPS FOR SUCCESSFUL BLOOD PRESSURE MONITOR USE

- Take care when applying upper arm cuff; a too tight or loose cuff during measurement may result in inaccurate measurements.
- If possible, take the readings at the same time each day and avoid eating or drinking prior to measuring.
- Wait 20 minutes between readings before attempting a second measurement.
- If your arm is outside of the range printed on the arm cuff, consult your healthcare provider for advice on how best to monitor your blood pressure.

GETTING STARTED



- Follow the included paperwork for pairing with App or other monitoring service before use.
- Apply upper arm cuff following instructions on the following page.
- Be seated with your feet flat on the floor with arm resting on a table and device resting on a flat surface.
- Never eat or talk during measurements.
- When ready, press START / STOP button to begin measurement; monitor will automatically inflate and deflate, taking your measurement and storing results in app history.
- Review the included app and manual for aid in interpreting your results. Variations in results should be discussed with your healthcare professional.

CUFF APPLICATION

Included Adult Cuff fits arm circumference: 8.6" – 17.7" (22 – 45 cm)

Included XXL Adult Cuff fits arm circumference: 15.7" – 20.5" (40 – 52 cm)

- Remove any constrictive clothing or jewelry that may interfere with cuff placement.
- Wrap the cuff snugly around your left arm.
- Position the cuff's bottom edge approximately 0.5" ABOVE the elbow joint. Position the air tubing over the brachial artery (on the inside of your arm).
- You should be able to insert one finger between your arm and cuff.
- If for any reason you are unable to or should not use your left arm, modify the instructions for cuff application to the right arm. Your physician can help you identify which arm is best for your health condition.

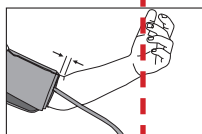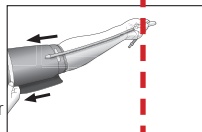BODY POSITION

1. Be seated with your feet flat on the floor and your elbow resting so your wrist will be level with your heart.
2. After you are positioned, insert the cuff air tubing into the monitor; never hold the monitor during readings.


MEASUREMENT

3. Press START/STOP button to power on; measurement will begin automatically.
4. Monitor will inflate and deflate, taking your measurement and storing results in measurement history. This should take approximately one minute.


RESULTS & PAIRING

5. If the APP is open, the monitor will automatically attempt to pair with your app while active. If the APP is not open, readings will store and sync the next time the device and app are connected.
6. Remove cuff; the monitor will automatically shut off after one minute of non-use.


SIMPLE STEP INSTRUCTIONS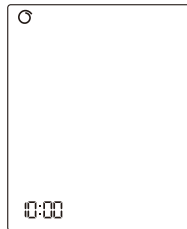


1. Press **START/STOP**; cuff wrap icon will appear.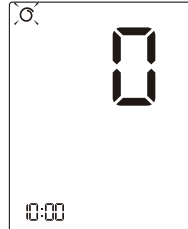

2. '0' will briefly show before measurement begins.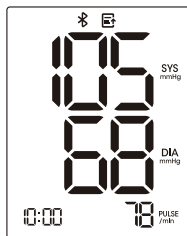

3. After measurement, results will appear.

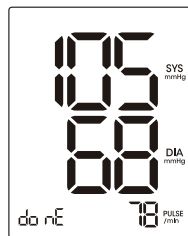4. Transmission successful when BT & transmission symbols disappear and says 'done'.

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST
www.veridianhealthcare.com

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