R.P.M. PRO™

FETAL DOPPLER BT-ENABLED

Model 210-9110B

THIS QUICK START GUIDE IS INTENDED AS A SUPPLEMENT TO THE INSTRUCTION MANUAL. PLEASE READ THE COMPLETE MANUAL PRIOR TO FIRST USE.

HOW TO USE

If using an app or monitoring software, ensure App is downloaded and Bluetooth is enabled before use. This device may be used without connecting to an App.


1. Position yourself flat or very reclined with comfortable access to your bare abdomen.
2. Power ON the doppler.
3. Apply a generous amount of ultrasound gel to the end of the doppler or directly onto your abdomen.
4. Starting at the center of your abdomen near the pelvic bone, gently glide the probe towards your belly button, slowly radiating out until the heartbeat is detected. Consult the guide for optimal location based on current pregnancy week.
5. If after a few minutes, you cannot locate the heartbeat, you may need to wait for the fetus to make growth progress or change position. Always discuss concerns with your healthcare professional.

PLEASE READ THESE INSTRUCTIONS CAREFULLY TO ENSURE PROPER USE AND SAFE OPERATION OF YOUR FETAL DOPPLER

TIPS FOR SUCCESSFUL USE OF FETAL DOPPLERS:

- Suitable for 12 weeks or later in pregnancy; earlier use may not detect the heartbeat.
- Use a generous amount of gel to help limit audible static; apply more as needed.
- Movement of doppler should be very slow.
- Ideal time for heartbeat detection: within 1 hour of eating, within 30 minutes of getting up, within 30 minutes of bedtime.
- Be patient and calm; note that you may detect your own heartbeat in addition to the fetus.

FETAL HEART LOCATION

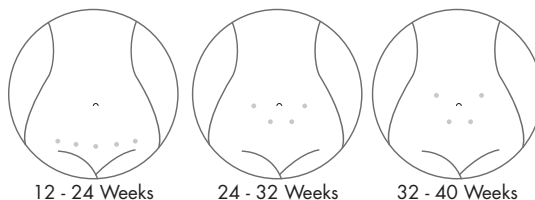Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST
For urgent matters, please contact your health care professional.
www.veridianhealthcare.com

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©2025 Veridian Healthcare, LLC